

# BEAM® ADD A PANEL ASSEMBLY INSTRUCTION

1. Connect both halves of your BEAM® Screen ADD A PANEL by inserting the ends of the smaller tubes into the ends of the larger tubes. (Keep both hinges on the same side when assembling)
2. Take your assembled BEAM® Screen and separate the top and bottom sections at the end to which you are adding a panel.
3. Slide the assembled BEAM® ADD A PANEL Screen hinges onto the open ends of your screen.
4. Reconnect the top and bottom halves of your original Screen.
5. Your Screen with the added panel is now ready for use and can be folded with the other sections.



☒ 800 GLEN AVENUE • MOORESTOWN, N.J. 08057

☎ 1.800.257.BEAM (2326) • LOCAL 1.856.359.2231

☎ FAX: 1.856.359.2249 • E-MAIL: [info@omnimedbeam.com](mailto:info@omnimedbeam.com)

CHARTING SYSTEMS, NARCOTIC CABINETS, PRIVACY SCREENS, STANDS, CARTS AND CUSTOM PRODUCTS  
PRODUCTS TO HELP PEOPLE . . . HELP PEOPLE