theratherm®

Instruction Manual

English - Español - Français

English	1
Español	17
Français	35

TABLE OF CONTENTS

Theratherm® Moist Heating Pad Therapy	2
LCD Display	3
Button Function	4
How to Use the Device (°C unit)	5
How to Use the Device (°F unit)	7
Treatment	9
“Caution: Burn or Skin Injury”	10
Product Specifications	11
Classification: Class II Equipment / Type BF Applied Part	12
UL Mark and Checking the Contents	13
Cleaning of Cotton Cover and Cleaning of Controller	14
Warranty	15

***** Time/30 min. & Temp./65°C are presented by the manufacturer*****

Theratherm® Moist Heating Pad Therapy

The application of moist heat is a commonly prescribed therapy in medical practice today. Moist heat is especially useful in treating pain related to muscle strain and tension which results in muscle spasm.

Theratherm Moist Heating Pad's high-grade intensive ceramics generates far-infrared to penetrate deep into the body for better physical therapy.


The LCD display performs multiple functions, to set-up temperature and time, but also to lock the display in order to avoid any accidental button pressing.

A warning signal will be displayed on the screen if the product malfunctions.

The heating pad is intended for physiotherapy, so EN IEC 60601-2-35 is not applicable.

For normal treatment, 15 minutes on a cycle is sufficient. We recommend a total treatment duration of about 30 minutes.

LCD Display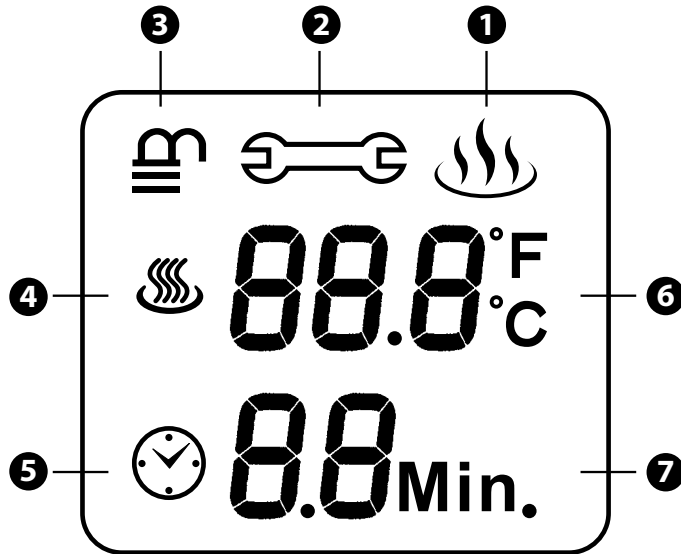

Emblem

- ① Heating
- ② Malfunction
- ③ Lock
- ④ Temperature
- ⑤ Time


Setup

- ⑥ Real-temp/Setup temperature
- ⑦ Real-time/Setup time

Button Function

⊙ Time range: 1~60 minutes

⊙ Temp range: 45~75°C / 113~166°F
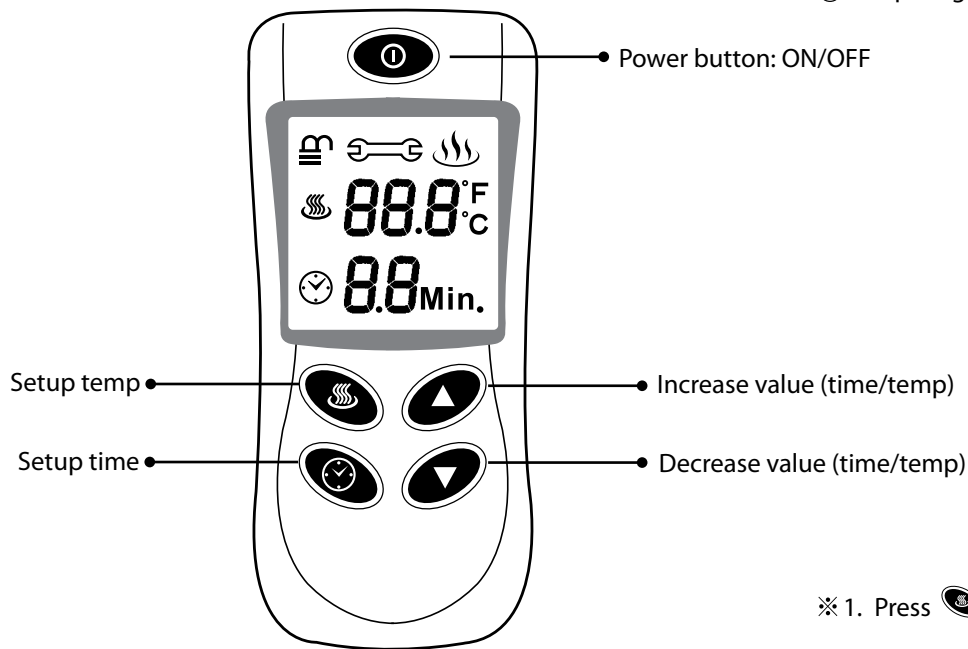


※ 1. Press  switch to °C or °F.


※ 2. Press and hold  3 seconds to lock up or unlock.

How to Use the Device (°C unit)


1. Fit the heating pad into the cotton cover sheet. NOTE: Always make sure a **clean** cotton cover sheet is on the heating pad.
2. Plug into the outlet of 90-130V (40-70 Hz)
3. After making sure that the skin is free from dirt and sweat, place the pad on the area of the body that you wish to treat.
4. Press the “ON/OFF” button “beep” sounds means power ON. The LCD will be displayed full segment (P.1) and the ambient temperature and last-setting time will light up in the indicator panel (P.2).
5. Press the temp “🔥” button to select the temperature. (P.3).
6. Press the time “🕒” button to select the time. (P.4).
7. Press the “▲/▼” button to increase or decrease values.
8. Continue to press in order to sustain increasing or decreasing of parameter. Range: temperature : 45-75°C, time : 1-60 min.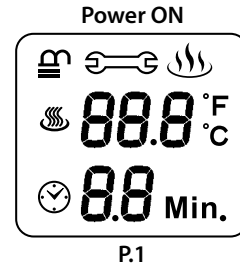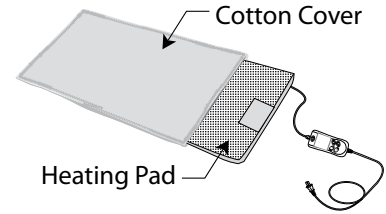


P.1

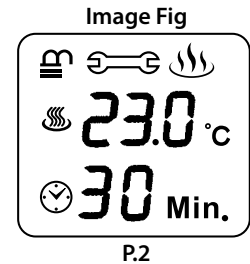P.2

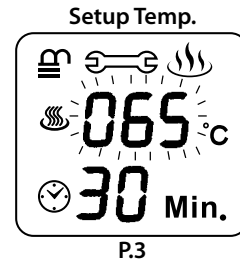P.3



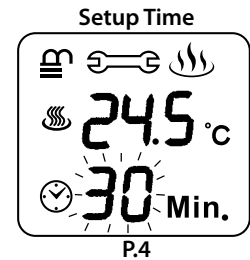
P.4

How to Use the Device (°C unit) – *continued*



9. Press the time “🕒” button until “beep” sound means the setting has been locked (press 3 seconds). **(P.5)**.
10. Press the temp “🔥” button to select °C or °F.
11. When finish the setting and after 5 seconds the LCD will display real-time of temperature and time. **(P.6)**.
12. When press “🔥” button the temperature figure will be flashing, you can set the number by “▲/▼” buttons.
13. When you press “🕒” button the time figure will be flashing, you set the number by “▲/▼” buttons.
14. After treatment completed, press the “ON/OFF” button to switch power OFF. (It will be turned off automatically when the time is up).
 - After you have finished your treatment, unplug from the outlet.
 - For normal treatment, 15 minutes on a cycle is sufficient. We recommend a total treatment duration of about 30 minutes.

NOTE: If malfunction signal is displayed, stop use. **(P.7)**. A symbol will be displayed on the LCD, then the buzzer will alarm continually for 5 seconds, and heat output will automatically shut off.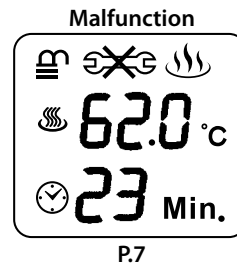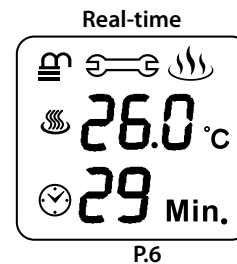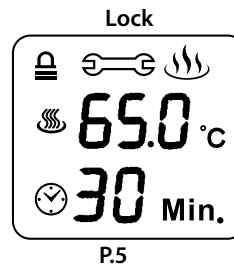


How to Use the Device (°F unit)


1. Fit the heating pad into the cotton cover sheet. NOTE: Always make sure a **clean** cotton cover sheet is on the heating pad.
2. Plug into the outlet of 90-130V (40-70 Hz)
3. After making sure that the skin is free from dirt and sweat, place the pad on the area of the body that you wish to treat.
4. Press the “ON/OFF” button “beep” sounds means power ON. The LCD will be displayed full segment (P.1) and the ambient temperature and last-setting time will light up in the indicator panel (P.2).
5. Press the temp “🌀” button to select the temperature. (P.3).
6. Press the time “🕒” button to select the time. (P.4).
7. Press the “▲/▼” button to increase or decrease values.
8. Continue to press in order to sustain increasing or decreasing of parameter. Range: temperature : 113-166°F, time : 1-60 min.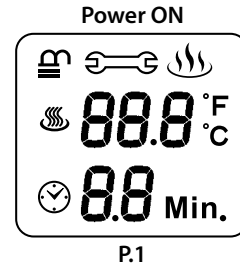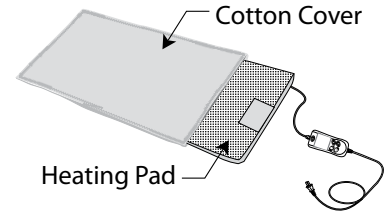


P.1

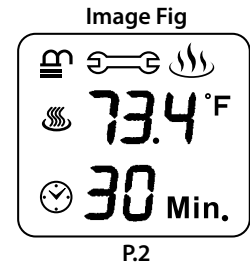P.2

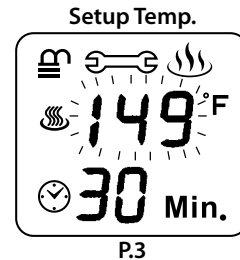P.3



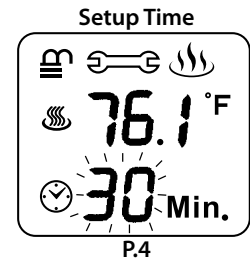
P.4


How to Use the Device (°F unit) – *continued*

9. Press the time “🕒” button until “beep” sound means the setting has been locked (press 3 seconds). **(P.5)**.
10. Press the temp “🔥” button to select °C or °F.
11. When finish the setting and after 5 seconds the LCD will display real-time of temperature and time. **(P.6)**.
12. When press “🔥” button the temperature figure will be flashing, you can set the number by “▲/▼” buttons.
13. When you press “🕒” button the time figure will be flashing, you set the number by “▲/▼” buttons.
14. After treatment completed, press the “ON/OFF” button to switch power OFF. (It will be turned off automatically when the time is up).
 - After you have finished your treatment, unplug from the outlet.
 - For normal treatment, 15 minutes on a cycle is sufficient. We recommend a total treatment duration of about 30 minutes.

NOTE: If malfunction signal is displayed, stop use. **(P.7)**. A symbol will be displayed on the LCD, then the buzzer will alarm continually for 5 seconds, and heat output will automatically shut off.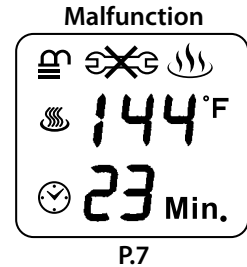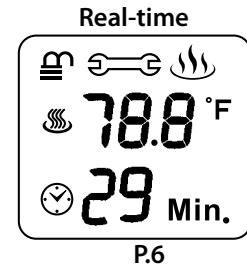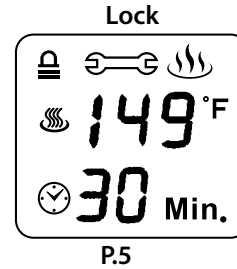


Treatment

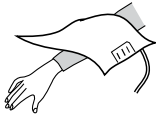
Arm


Knee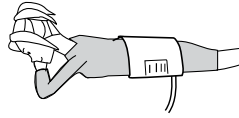

Waist

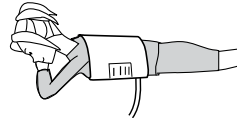Back

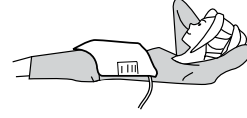Abdomen

Neck & Shoulder

Important

- Read all instructions.
- Never use this Heating-Pad **without** a clean cotton cover.
- Keep Heating-Pad out of reach of children.
- Use of this Heating-Pad is not intended as a substitute for medical care. Please consult with your physician.
- Do not expose the product to strong shocks and do not drop it.
- Do not twist or draw out the electric wire. Do not use the cord as a handle.
- Do not store the unit where exposed to direct sunlight, high temperatures, high humidity, dust, etc.
- Do not use gasoline, paint thinner, benzine, or other strong solvents.
- Do not disassemble or modify the unit.
- Degree of protection against harmful ingress of water: Ordinary Equipment.

CAUTION: Burn or Skin Injury

1. Check skin under pad frequently to avoid burning and blistering.
2. Unplug when not in use.
3. For normal treatment, do not use the pad over 30 minutes at high temperature 46-75°C / 114-166°F settings.
4. Display board should not be crushed or rammed.
5. Do not use while sleeping. Do not sit on or crush pad. Do not wet.
6. Do not use on an infant.
7. This pad is not to be used on or by an invalid, sleeping or unconscious person or a person with poor blood circulation unless carefully attended, check skin under pad frequently.
8. Do not use on area of insensitive skin.
9. Never use pad without cover in place.
10. Carefully examine inner cover before each use, discard the pad if inner covering shows any signs of deterioration.
11. Do not use pins or other metallic means to fasten this pad in place.
12. Never pull this pad by the supply cord and do not use the cord as a handle.
13. Consult with your physician before you use this device.

Classification: Class II Equipment Type BF Applied Part

Operating Condition:	Temperature	: 20-40°C / 68-104°F
	Humidity	: 30-75%RH
Storage Condition:	Storage Condition	: Temperature : 10-50°C / 50-122°F
	Humidity	: 10-90%RH
Transportation Condition:	Transportation Condition:	: Temperature : 10-70°C / 14-158°F
	Humidity	: 10-90%RH


****Do not expose the Heating-Pad to temperature extremes (below 10°C/50°F or over 50°C/122°F) nor excessive humidity (>90%RH).**

*****Equipment not suitable for use in the presence of a flammable anaesthetic mixture with air or with oxygen or nitrous oxide.**

UL Mark and Checking the Contents

This product is compliant to the related Medical regulations including :

EMC : IEC60601-1-2
Electrical safety : IEC60601-1
MDD (93/42/EEC) 0197

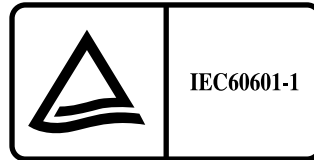The heating-pad is intended for physiotherapy, not for warm keeping so EN IEC 60601-2-35 is not applicable.

UL Mark

Checking the contents.

Make sure the cotton cover sheet is included in the package. If it is missing or damaged, please contact your local dealer.


Cleaning of Cotton Cover

- Machine wash in mild soap.
- Hand wash in lukewarm or cold water with a mild detergent.
- Dry clean.

NOTE: The cotton cover sheet is a replaceable accessory. Replace it when they fail to moisten properly or damage or too dirty.

! Do not bleach.

! Rinse well short spin.

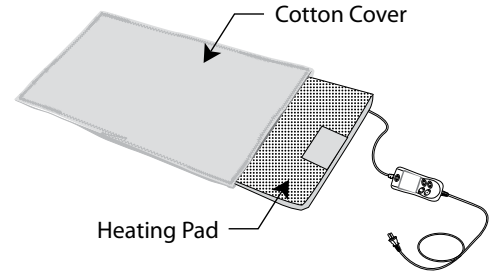Cleaning of Controller

- Please use a soft cloth slightly moistened with water, then lightly wipe the surface.
- ! Do not use any type of solvent, such as thinner or benzene.**

Warranty

DJO, LLC (“Company”) warrants that the Theratherm Moist Heating Pad (“Product”) is free of defects in material and workmanship. This warranty shall remain in effect for one year (12 months) from the date of original consumer purchase. If this Product fails to function during the one year warranty period due to a defect in material or workmanship, Company or the selling dealer will repair or replace this Product without charge within a period of thirty (30) days from the date on which the Product is returned to the Company or the dealer. All repairs to the Product must be performed by a service center authorized by the Company. Any modifications or repairs performed by unauthorized centers or groups will void this warranty.

This Warranty Does Not Cover:

- Replacement parts or labor furnished by anyone other than the Company, the selling dealer or a certified Company service technician.
- Defects or damage caused by labor furnished by someone other than Company, the selling dealer or a certified Company service technician.
- Any malfunction or failure in the Product caused by product misuse, including, but not limited to, the failure to provide reasonable and required maintenance or any use that is inconsistent with the Product User Manual.

COMPANY SHALL NOT BE LIABLE IN ANY EVENT FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES.

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

To Obtain Service From Company or the selling dealer under this warranty:

1. A written claim must be made within the warranty period to the Company or the selling dealer. Written claims made to the Company should be sent to:

DJO, LLC
1430 Decision St
Vista, CA 92081 USA
Phone: 1-800-592-7329 USA
Phone: 1-423-870-2281 or 1-317-406-2250
Fax: 1-317-406-2014

and

2. The Product must be returned to the Company or the selling dealer by the owner along with a copy of purchase receipt.

This warranty gives you specific legal rights and you may also have other rights which vary from location to location. The Company does not authorize any person or representative to create for it any other obligation or liability in connection with the sale of the Product. Any representative or agreement not contained in the warranty shall be void and of no effect.

THE FOREGOING WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

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