one year limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for one year from the date of the original consumer purchase.

This device was built to exacting standards and carefully inspected prior to shipment. This One Year Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



Wenzelite Re/hab

© 2006 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

trotter bus transport option

Item # TR 8029



ш



operating instructions

The Trotter has passed a dynamic crash test done with a 130 lb. front-facing occupant tested at 30 mph (48km/h). It was tested in accordance with Annex A of ANSI/RESNA WC19.

- Bus Transit Tie Downs should be used with restraint systems that conform to SAE J2249 requirements.
- Must be used in a vehicle equipped for for ward-facing tie down during transport.
- Remove mounted accessories; such as, trays and respiratory equipment and secure separately.

installation



- Insert two bolts in each bracket and then through the black tapered washers.
- Mount brackets at designated holes (A) on Trotter frame.
- 3. Install black flat washer.
- 4. Install and tighten nut.

NOTE: The two front brackets mount on the outside of the Trotter frame and the two rear brackets mount of the inside of the Trotter frame. All brackets should face outward.