


Vestibulator II® Safety Note

Tumble Forms 2 Deluxe Vestibulator II frame has a working load of 300 lbs.

Tumble Forms 2 Vestibulator II should be used only with Tumble Forms 2 Accessories including the following:

2772P - Platform Swing 2772D - Flexi Disk 2772H - Roll Swing 2772N - Prone Net Swing 2772S - Net Swing with Positioning Seat 5568-23 - 36" Horse Roll 5568-24 - Free Swing 5568-25 - 5' Platform Swing

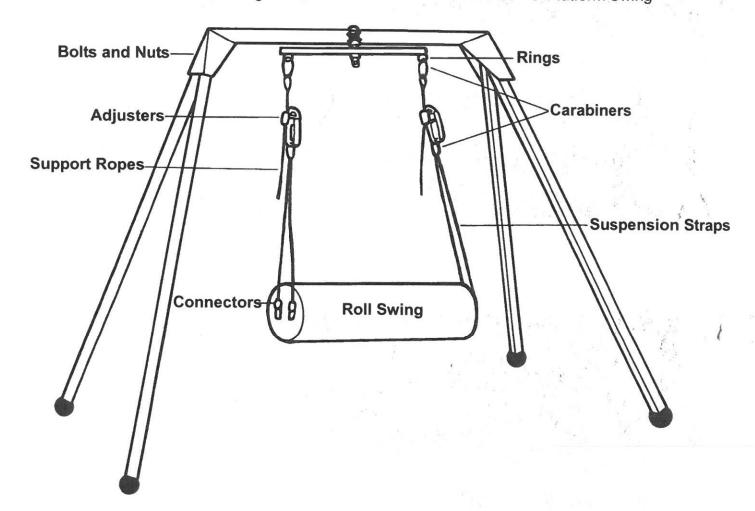

Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your client's safety, don't take a chance.

Recommended for use under the supervision of a healthcare professional.

Manufactured for:

MADE IN THE USA

LIT 47 REV D

PERFORMANCE HEALTH


W68 N158 Evergreen Blvd. Cedarburg, WI 53012 For Replacement Parts


Call toll free: **1-800-323-5547** In Canada call 1-800-655-9200





Vestibulator II®

Assembly Instructions

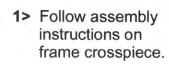


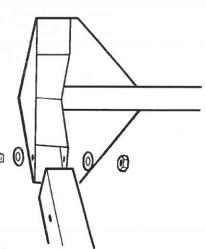

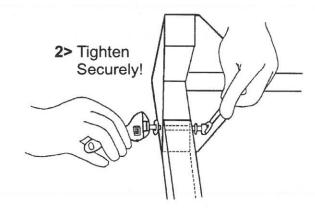


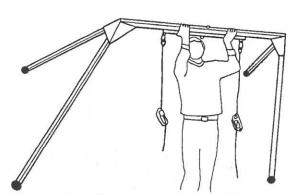

3> Lift into upright position.

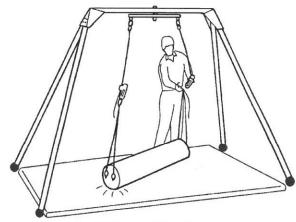

4> With mat in place attach appropriate swing.

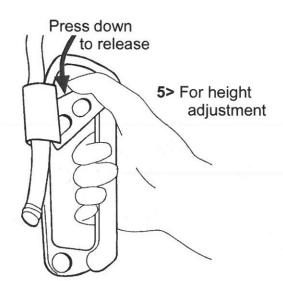




6> Adjust height and position client



Vestibulator II® 36" Horse Roll

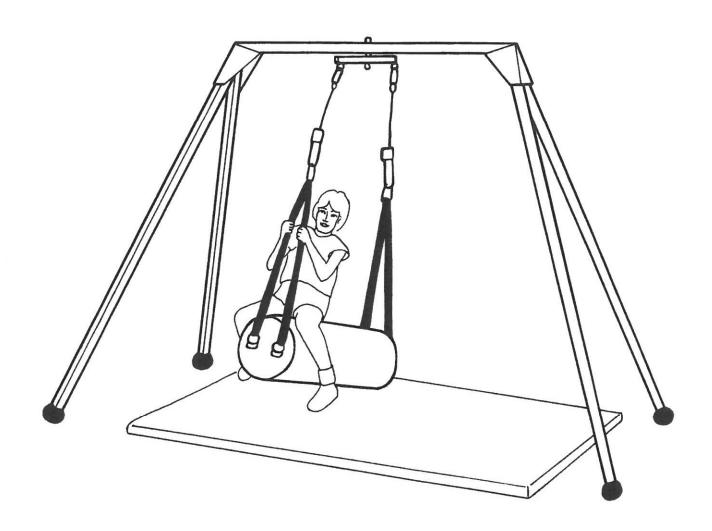

36" Horse Roll Swing Safety Notes

The Tumble Forms 2® 5568-23 Horse Roll Swing should only be used with the Tumble Forms 2® Vestibulator II, III and IV frames. The maximum weight limit for any combination of client and therapist performing activities on the suspended roll swing should not exceed 250 lbs. Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your client's safety, don't take a chance. Recommended for use under the supervision of a healthcare professional.

Manufactured for:

MADE IN THE USA

LIT 46G

PERFORMANCE HEALTH

W68 N158 Evergreen Blvd. Cedarburg, WI 53012 For Replacement Parts

Call toll free: **1-800-323-5547** In Canada call 1-800-655-9200



Vestibulator II® Flexi Disk



Flexi Disk Safety Notes

The Tumble Forms 2® 2772D Flexi Disk should only be used with the Tumble Forms 2® Vestibulator II, III and IV frames. The client weight should not exceed 150 lbs. Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your client's safety, don't take a chance. Recommended for use under the supervision of a healthcare professional.

Manufactured for:

MADE IN THE USA

LIT 46C

PERFORMANCE HEALTH

W68 N158 Evergreen Blvd. Cedarburg, WI 53012 For Replacement Parts

Call toll free: 1-800-323-5547 In Canada call 1-800-655-9200



Vestibulator II® Platform Swing



Platform Swing Safety Notes

The Tumble Forms 2® 2772P Platform Swing should only be used with the Tumble Forms 2® Vestibulator II, III and IV frames. The client weight should not exceed 150 lbs. Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your client's safety, don't take a chance. Recommended for use under the supervision of a healthcare professional.

Manufactured for:

MADE IN THE USA

LIT 46B


PERFORMANCE HEALTH

W68 N158 Evergreen Blvd. Cedarburg, WI 53012 For Replacement Parts

Call toll free: **1-800-323-5547** In Canada call 1-800-655-9200



Vestibulator II® Roll Swing

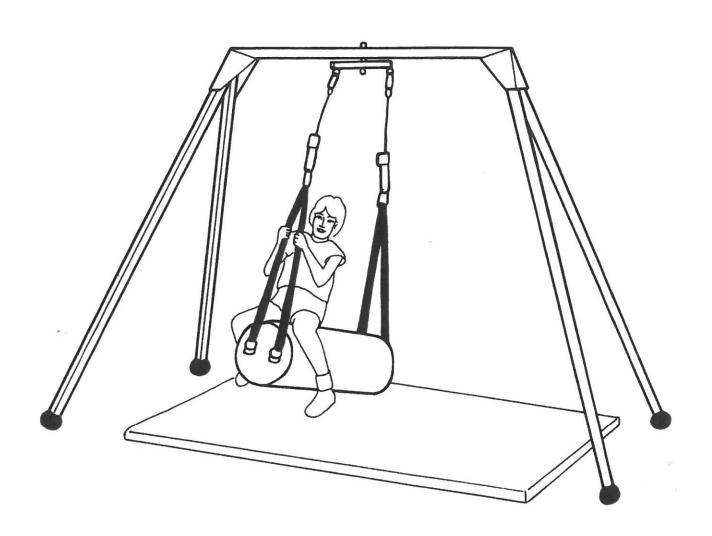

Roll Swing Safety Notes

The Tumble Forms 2® 2772H Roll Swing should only be used with the Tumble Forms 2® Vestibulator II, III and IV frames. The maximum weight limit for any combination of client and therapist performing activities on the suspended roll swing should not exceed 250 lbs. Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your client's safety, don't take a chance. Recommended for use under the supervision of a healthcare professional.

Manufactured for:

MADE IN THE USA

LIT 46A

PERFORMANCE HEALTH

W68 N158 Evergreen Blvd. Cedarburg, WI 53012 For Replacement Parts


Call toll free: 1-800-323-5547 In Canada call 1-800-655-9200



Vestibulator II® Net Swing

Set up Instructions

- 1> Connect one carabiner snap ring to each ascender.
- 2> Recline the Feeder Seat® in the net with the lower front edge of the net covering the lower front edge of the Feeder Seat® The back of the seat should be fully supported by the net as shown.
- 3> Lower the unit, using the ascenders, to a point where your client has easy access. Use the positioning straps to maintain your clients posture.

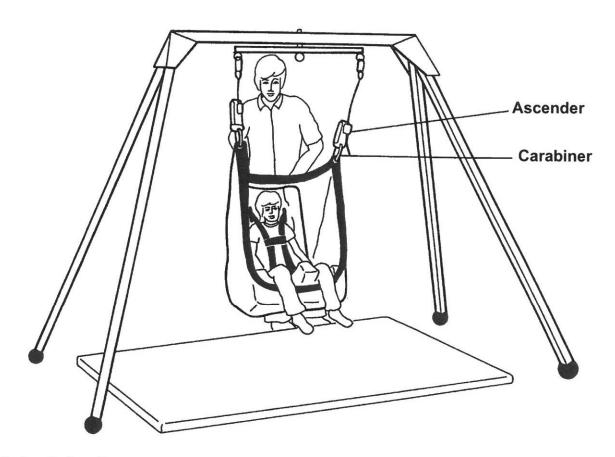

Net Swing Safety Notes

The Tumble Forms 2® 2772S Net Swing should only be used with the Tumble Forms 2® Vestibulator II, III and IV frame. The client should not exceed 100 lbs in weight. Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your client's safety, don't take a chance. Recommended for use under the supervision of a healthcare professional.

Manufactured for: MADE IN THE USA LIT 46

PERFORMANCE HEALTH

W68 N158 Evergreen Blvd. Cedarburg, WI 53012

For Replacement Parts

Call toll free: **1-800-323-5547** In Canada call 1-800-655-9200



Prone Net

CAUTION

We are committed to providing you with quality products. However, we cannot take responsibility for your ceiling support system installation or maintenance of your equipment. If you are unsure of your ceiling support system, be sure to consult your building's maintenance supervisor or a rigging specialist, who can be found in the Yellow Pages.

Please check your equipment regularly, use floor mats, and if there is any question of your client's safety, don't take a chance.

We welcome your comments, so please feel free to contact us at:

Manufactured for:

Tarabiar ou Tor.

PERFORMANCE HEALTH

W68 N158 Evergreen Blvd. Cedarburg, WI 53012 MADE IN THE USA

LIT 45

For Replacement Parts

Call toll free: **1-800-323-5547** In Canada call 1-800-655-9200



Vestibulator II[®] Prone Net Swing



Prone Net Swing Safety Notes

The Tumble Forms 2® 2772N Prone Net Swing should only be used with the Tumble Forms 2® Vestibulator II, III and IV frames. The client weight should not exceed 150 lbs. Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your client's safety, don't take a chance. Recommended for use under the supervision of a healthcare professional.

Manufactured for:

MADE IN THE USA

LIT 46D

PERFORMANCE HEALTH"


W68 N158 Evergreen Blvd. Cedarburg, WI 53012

For Replacement Parts

Call toll free: 1-800-323-5547 In Canada call 1-800-655-9200



Vestibulator II® 36" Horse Roll

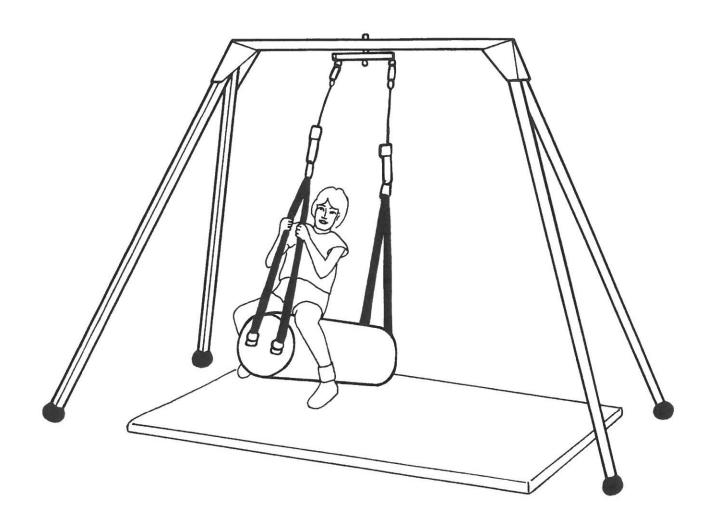

36" Horse Roll Swing Safety Notes

The Tumble Forms 2® 5568-23 Horse Roll Swing should only be used with the Tumble Forms 2® Vestibulator II, III and IV frames. The maximum weight limit for any combination of client and therapist performing activities on the suspended roll swing should not exceed 250 lbs. Use floor mats. Never leave your client unattended. Please check your equipment regularly and replace any parts that show excessive wear. If there is any question of your client's safety, don't take a chance. Recommended for use under the supervision of a healthcare professional.

Manufactured for:

PERFORMANCE HEALTH

W68 N158 Evergreen Blvd. Cedarburg, WI 53012

MADE IN THE USA

LIT 46G

For Replacement Parts

Call toll free: 1-800-323-5547 In Canada call 1-800-655-9200