

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INDICATIONS: Mild MCL and/or LCL sprains, Mild arthritis, Post arthroscopic meniscectomy, and/or Mild global instabilities

- 1)** Unfasten and loosen the straps.
- 2)** Wrap product around leg and fasten over the front of the leg. Position the sleeve so that the hinges are parallel to each other, and the top of the circular cut-out aligns with the top of the knee cap. Note: Adjust posterior (back) straps for a snug fit.
- 3)** Fasten the anterior (front) thigh and calf support straps.

REMOVAL: To remove, unfasten straps and front flaps.

USE AND CARE:

Hand wash in cold water with mild detergent. Air dry. Avoid direct heat or sun. This product is intended for use on a single patient.

CAUTION: Support should be worn as directed.

WARNING: If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately. Never wear the support over an open wound.

WARNING: Neoprene supports should not be worn by individuals with known susceptibility to dermatitis or allergies to neoprene by-products.