

ActivaDose® Lead Wire


For use only with **ActivaDose®**,
ActivaDose® II and **I-Wave**
iontophoretic dose controllers.

Best for use on **Trivarion®** and **I-Trode**
snap-style iontophoresis electrodes.

Package contains one (1) **ActivaDose®**
Lead Wire

Order No. 001-5A

See reverse for instructions

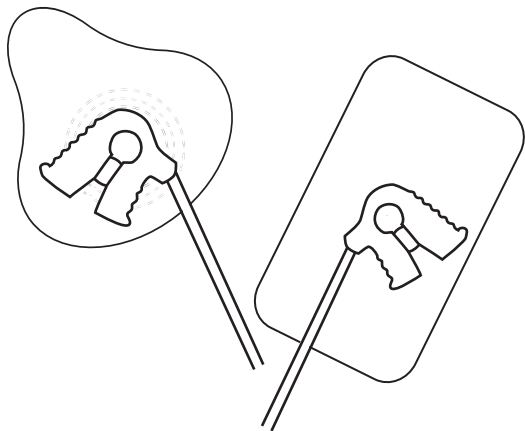

Manufactured for:

ActivaTek Inc.

2734 South 3600 West, Unit F

Salt Lake City, UT 84119 USA

Phone & Fax: 1-800-680-5520

Email: admin@activatekinc.com

www.activatekinc.com

P/N 001-83 Rev A
A/W 001-143 Rev A

How to Use the ActivaDose[®] Lead Wire

IMPORTANT: Use the **ActivaDose[®]** Lead Wire only with the **ActivaDose[®]**, **ActivaDose[®] II** and **I-Wave** Dose Controllers.

1. Plug the **ActivaDose[®]** lead wire connector into the connector port on the dose controller.
2. Attach the lead wire to the appropriate connector snaps on the drug and dispersive pad (ground electrode). Be sure the clips are attached correctly and follow polarity guidelines.
3. Please refer to the dose controller manual and the drug delivery electrode directions for important additional information.



Manufactured for:

ActivaTek Inc.

2734 South 3600 West, Unit F

Salt Lake City, UT 84119 USA

Phone & Fax: 1-800-680-5520

Email: admin@activatekinc.com

www.activatekinc.com