



- Decubitus prevention
- Air Floatation
- 20g Vinyl, Heat Sealed Construction
- Anti-Bacterial
- Large air valve
- Easy to inflate or deflate
- Wipe clean vinyl
- Safety straps
- Pre-inflated
- Meets CA 117



| Reorder# | Description                            | Size      | Color  | Capacity | Warranty | UM  |
|----------|--|-----------|--------|----------|----------|-----|
| 756210   | Air Lift Seat Cushion only             | 17" x 17" | Blue   | 250 lbs. | 1 Yr     | Ea. |
| 756215   | Air Lift Seat Cushion w/LSII Cover     | 17" x 17" | Blue   | 250 lbs. | 1 Yr     | Ea. |
| 756219   | Optional Cushion Cover w/Safety Straps | 17" x 17" | Blue   | 250 lbs. | 1 Yr     | Ea. |
| 756225   | Air Lift Seat Cushion only             | 19" x 19" | Grey   | 250 lbs. | 1 Yr     | Ea. |
| 756230   | Air Lift Seat Cushion w/LSII Cover     | 19" x 19" | Grey   | 250 lbs. | 1 Yr     | Ea. |
| 756234   | Optional Cushion Cover w/Safety Straps | 19" x 19" | Grey   | 250 lbs. | 1 Yr     | Ea. |
| 756235   | Air Lift Seat Cushion only             | 22" x 28" | Green  | 450 lbs. | 6 Mo.    | Ea. |
| 756237   | Air Lift Seat Cushion w/LSII Cover     | 22" x 28" | Green  | 450 lbs. | 6 Mo.    | Ea. |
| 756239   | Optional Cushion Cover w/Safety Straps | 22" x 28" | Green  | 450 lbs. | 6 Mo.    | Ea. |
| 751580   | Optional Hand Pump                     | Universal | Purple | N/A      | 6 Mo.    | Ea. |

**Indication for use:**

Comfort, prevention and healing therapy through stage IV pressure ulcers. Deep tissue injury protection.

**Inspection, Inflation and Deflation:**

1. A properly inflated cushion will appear to be half full.
2. Place the individual on the cushion in his/her normal sitting position.
3. Slide your hand between the cushion and surface of the chair, and feel for the lowest bony prominence.
4. If the bony prominence is felt, remove the patient and add air as needed.
5. If the bony prominence isn't felt no action is required.
6. Optional air pump; open valve, place pump inside valve and use pump action to inflate. Remove pump and close valve cover.
7. To deflate cushion, open valve cover and squeeze air out by pressing on upper surface. Close valve cover when finished.

**Installation:**

1. Place cushion on wheelchair, valve side down, with tie straps towards back of wheelchair and secure cushion to back of wheelchair with straps.
2. Optional cover; open zipper, place cushion inside cushion cover with flange toward the back of the cover and with the valve facing the bottom (Cover, blue side up, black side is bottom) close zipper, adjust safety straps around chair, and connect side release buckle. Cover can be wiped clean with spray and wipe cleaning solution, avoid bleach.

**Easy Care:**

1. Clean cushion with spray and wipe cleaning solution, avoid bleach. Do not launder cushion.
2. Do not launder cushion cover.

**Caution: Keep cushion away from open flame and cigarettes and avoid sharp objects from making contact with the cushion.**