



Cardinal Health™ Monoject™ Hypodermic Safety Needles

Instructions for use



1 Select the appropriate Monoject™ Hypodermic Safety Needle for your procedure. All Monoject™ Safety Needles are ISO color-coded.



2 Open package using aseptic technique by peeling apart the plastic and paper tab, then remove safety needle.



3 Secure the needle by holding the needle and twisting the syringe clockwise to fully seat the needle. Remove protective needle sheath.



4 Draw up medication and administer injection using aseptic technique in accordance with your facility's protocol.

Instructions for disposal



5 Lock the safety shield using either of the activation methods.



6 An audible and tactile click will verify the safety device has been activated.



7 Once safety shield is locked, immediately dispose of needle and syringe in an approved sharps container.



For more information, visit cardinalhealth.com/monoject, scan the QR code to view product guides or call customer service at **1.800.964.5227**

