

# Delta-Lite® Plus

Use the Glove of Your Choice and Customize  
the Set Time to Your Individual Needs

Delta-Lite® Plus is constructed of a 100% fiberglass substrate and specially formulated polyurethane resin with built-in slip agent. The built-in slip agent allows you to use the glove of your choice and the resin formula allows you to customize the set time to your individual needs.

## Faster Set Time For Experienced Clinicians and Less Difficult Applications

- 1) Submerge cast tape in room temperature water (Max 77°F / 25°C) for 5 seconds.
- 2) While submerged squeeze the cast tape 2-3 times using firm pressure. The more water that is forced into the roll, the faster the set time.
- 3) Remove from water and give 2-3 firm squeezes to remove water. The more water removed, the faster the set time.

## Slower Set Time For Less Experienced Clinicians and More Difficult Applications

- 1) Submerge cast tape in room temperature water (Max 77°F / 25°C) for 5 seconds.
- 2) Do not squeeze cast tape while submerged.
- 3) Remove from water and shake to remove excess water. The more water that remains within the roll, the slower the set time.



# Delta-Lite® Plus

## The Trusted Rigid Solution

### Your new standard of casting.

Delta-Lite® Plus is our newest fiberglass cast tape that combines a fiberglass substrate and advanced water activated polyurethane resin system which provides tack free properties. The tack free properties allow ease of application and molding using standard examination gloves.

### Features and Benefits

**Tack free resin** - Use exam glove of your choice. Improves unrolling tension, improving ease of cast application

**Rigid finished cast** - Creates long-lasting, light-weight cast

**Smooth surface** - Less abrasive finished cast enhances patient comfort while it reduces clothing snags

**Good conformability** - Makes casting application easier with change of anatomy

**Outstanding molding** - Produces an anatomically correct cast

**Excellent lamination** - Creates a strong bond between layers ensuring a strong finished cast

### Ordering Information

Order Number	Size	Color	Unit of Measure
7345800	1 in. x 2 yds.	WHITE	10 rolls / box
7345801	2 in. x 4 yds.		10 rolls / box
7345802	3 in. x 4 yds.		10 rolls / box
7345803	4 in. x 4 yds.		10 rolls / box
7345804	5 in. x 4 yds.		10 rolls / box
7345845	2 in. x 4 yds.	BLACK	10 rolls / box
7345846	3 in. x 4 yds.		10 rolls / box
7345847	4 in. x 4 yds.		10 rolls / box
7345835	2 in. x 4 yds.	LIGHT BLUE	10 rolls / box
7345836	3 in. x 4 yds.		10 rolls / box
7345837	4 in. x 4 yds.		10 rolls / box
7345838	5 in. x 4 yds.		10 rolls / box
7345860	2 in. x 4 yds.	PURPLE	10 rolls / box
7345861	3 in. x 4 yds.		10 rolls / box
7345862	4 in. x 4 yds.		10 rolls / box
7345855	2 in. x 4 yds.	PINK	10 rolls / box
7345856	3 in. x 4 yds.		10 rolls / box
7345857	4 in. x 4 yds.		10 rolls / box
7345830	2 in. x 4 yds.	DEEP RED	10 rolls / box
7345831	3 in. x 4 yds.		10 rolls / box
7345832	4 in. x 4 yds.		10 rolls / box
7345820	2 in. x 4 yds.	DEEP BLUE	10 rolls / box
7345821	3 in. x 4 yds.		10 rolls / box
7345822	4 in. x 4 yds.		10 rolls / box
7345823	5 in. x 4 yds.		10 rolls / box
7345825	2 in. x 4 yds.	DEEP GREEN	10 rolls / box
7345826	3 in. x 4 yds.		10 rolls / box
7345827	4 in. x 4 yds.		10 rolls / box
7345828	5 in. x 4 yds.		10 rolls / box
7345865	2 in. x 4 yds.	YELLOW	10 rolls / box
7345866	3 in. x 4 yds.		10 rolls / box
7345867	4 in. x 4 yds.		10 rolls / box
7345875	2 in. x 4 yds.	ORANGE	10 rolls / box
7345876	3 in. x 4 yds.		10 rolls / box
7345877	4 in. x 4 yds.		10 rolls / box
7345850	2 in. x 4 yds.	MIXED PACK*	10 rolls / box
7345851	3 in. x 4 yds.		10 rolls / box
7345852	4 in. x 4 yds.		10 rolls / box

\*Mixed pack: Contains 2 Dark Blue, 2 Light Blue, 2 Deep Red, 2 Purple and 2 Pink.

Colors and availability subject to change.

Please refer to the product label and / or package insert for full instructions on the safe use of these products.



BSN medical Inc. 5825 Carnegie Blvd. Charlotte, NC 28209-4633  
 Tel. (+1) 704 554 9933 Fax (+1) 704 358 4558 www.bsnmedical.com  
 To order toll-free: (+1) 800 552 1157

Questions? Email: [deltasolutions@bsnmedical.com](mailto:deltasolutions@bsnmedical.com)