



Economy Digital Floor Scale

Item #081588813

Please read instructions before use.

Instructions:

- Install the battery
- Place the scale on the floor and make sure all corners touch the floor for stability
- Tap the scale gently to power it on; the LCD display will show "8"
- Stand on the scale; LCD display will show weight result within seconds
- After 30 seconds, the scale will automatically power off to save battery power (To zero the scale, step on it 3 times. The third time it will read accurate.)

Product Information:

- Weight capacity of scale is 400 lbs (182 kg)
 - Item takes one CR2032 Lithium Battery
 - Item use is for body weight measure only
 - Do not use harsh chemicals to clean this item – use gentle cleanser only
 - Stand on unit with both feet on at same time to avoid falling and injury
 - Use the item on a flat surface; uneven surfaces such as carpet may skew results
 - Do not submerge item in water
 - Do not jump on scale; this may cause damage to the item
- To change from kg to lb, on backside is a tab, press to change

Warning Indicators

- If the LCD display reads "Err" this means the weight exceeds weight limit of scale.
- If the LCD display reads "LO" this means the battery is low – please replace.



Sammons Preston
A Patterson Medical Company
1000 Remington Blvd., Suite 210
Bolingbrook, IL 60440-5117
Patterson Medical Company
All rights reserved.
1-800-323-5547
www.sammonspreston.com