

P.O. Box 3417 Abillene, TX 79604

Telephone: 325.677.4177 Toll Free: 800.637.8436 Facimile: 325.677.7217 Web: www.convaquip.com

Model SS-H-05C / SS-H-08C

Handle Step Assembly Instructions

- 1. Place the platform on a flat, even surface so that all four rubber leg tips are flush with the surface and the platform is stable. One person should maintain even pressure (sitting or standing) on the platform throughout steps 2-4 below.
- 2. Grasp the handle at the U-bend and position open ends over round tube sections attached to the platform so that both the snap buttons and the holes are facing you.
- 3. Depress both snap buttons and simultaneously push down slowly on the handle until the snap buttons have just cleared the edge of the lower round tube sections.
- 4. Push down on the U-bend (top) of the handle until both snap buttons have clicked into place in the holes.