

Re-Order #	Description	Size	Unit
914366	Back Support & 16" Gel Foam X-Cushion	Back 16"x16" Seat 16"x16"	Set
914368	Back Support & 18" Gel Foam X-Cushion	Back 18"x16" Seat 18"x16"	Set
914372	Back Support with pocket for 16" Cushion	Back 16"x16" Seat 16"x16"	Each
914374	Back Support with pocket for 18" Cushion	Back 18"x16" Seat 18"x16"	Each
914443	Back Support & 16" Gel Foam X-Cushion w/Coccyx Cut-Out	Back 16"x16" Seat 16"x16"	Set
914444	Back Support & 18" Gel Foam X-Cushion w/Coccyx Cut-Out	Back 18"x16" Seat 18"x16"	Set
751026	Replacement Gel-Foam X-Cushion	16"x16"	Each
751027	Replacement Gel-Foam X-Cushion	18"x16"	Each



**Purpose**

Typically, pressure sores are formed when capillary blood flow is impeded which occurs when an individual sits on a hard surface. In addition, the same exposure to pressure sores also exists in the clavicle or back area. In a standard wheelchair scenario, a cushion is used without a foam back support. Also, many cushions aren't provided with tie straps on the cushion, therefore allowing the cushion and or patient to slide off the wheelchair.

This product was designed to provide pressure relief in both the back and seat and is available in two setups, namely with the gel cushions provided in model #914368, 914366, 914443, 914444 and the other setup with a foam back without the cushions (914374, 914372), but will accept an existing wheelchair cushion sized 18"x16"x4" or smaller or 16"x16"x4 or smaller.

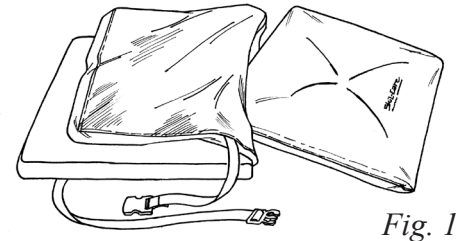


Fig. 1

In the event that a wheelchair cushion is already available at the facility, models number's 914372 and 914374 can be purchased with only the foam back support and seat pocket which can accept any wheelchair cushion shown above. See Figure 1.

**Easy to Use**

**Model numbers 914368, 914366, 914443 and 914444**

Place the seat on the chair with the cushion corresponding to the seat with the non skid surface facing down, and the foam support against the backrest of the wheelchair. Open the side release buckle and place around the top of the chair.

**Model numbers 914374 and 914372**

Place the available cushion in the seat pocket facing seat up. Note that the non skid surface is the bottom. Open the side release buckle and place around the top of the chair.

**Easy Care**

With the seat cushion and backrest foam insert removed the stretch cover is launderable using temperatures below 180°F(82°C). If bleach is used, assure that it is diluted or mild. The Gel X-Cushion provided in models 914368, 914366, 914443 and 914444 are heat sealed and can be cleaned with mild detergent or spray and wipe cleaner. See Figure 2.

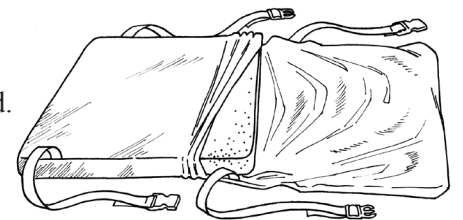


Fig. 2

***This product is guaranteed for 6 months.***