## Rolyan\* Neoprene Pull-On Thumb Support:

A952-40 through 49

## Rolyan\* Neoprene Wrap-On Thumb Support:

A952-50 through 59

Be sure to note which support you are using and follow the instructions that apply to that support. Unless otherwise stated, these instructions apply to both supports.

### INDICATIONS

- May be used for tendonitis and other repetitive motion injuries to the thumb
- May be used for soft tissue injuries, arthritis, bursitis, and general muscle soreness

### CONTRAINDICATIONS

- Not for immediate treatment following fractures
- Not for people who have an allergic reaction to neoprene, or who have a tendency to develop dermatitis
- Not for use over open wounds and infected areas

### PRECAUTIONS

- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Be careful that the support does not fit so tight that circulation is impaired.

### CARE AND CLEANING

- Wash frequently to remove body oils and other soil.
- Hand wash in warm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure the support is completely dry before reapplying it.
- Do not use ointments or oils under the support.

- Do not dry clean or expose to heat or sunlight for extended periods of time.

### Rolyan Neoprene Wrap-On Thumb Support

- Fasten hook-and-loop closure prior to washing support.

### INSTRUCTIONS FOR USE

The edge of the palmar area of the support is cut deeper than the edge of the back area.

### Rolyan Neoprene Pull-On Thumb Support

Slide the thumb into the tubelike thumb covering and pull the support onto the hand.

### Rolyan Neoprene Wrap-On Thumb Support

Slide the thumb into the tubelike thumb covering. Attach the straps on the back of the hand.

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