INDICATIONS

- To increase upper extremity range of motion, strength, and coordination
- To improve eye/hand coordination
- To improve balance and trunk control
- To improve the ability to compensate for hemianopsia and overcome unilateral neglect
- To improve the ability to cross the midline

PRECAUTIONS

- **This equipment should be used only with the instruction and supervision of a healthcare professional.**
- **If unusual discomfort occurs, use should be discontinued.**
- **Use a stable table top.**


CARE AND CLEANING

- The hard surfaces can be cleaned with a hard-surface disinfectant. The straps can be wiped with soap and warm water. Be sure the unit is completely dry before using it.
- Lubricate the casters periodically with a light grease to maintain free motion. In addition, apply a few drops of oil to the bearings.

PROCEDURE

Suggested use is shown in **A**.

1. The hand support section adjusts for the right or left hand and for a variety of forearm lengths as follows:
 - The hand support section tilts to the left for right-hand use and to the right for left-hand use. To change it, release the hook-and-loop fastener and turn the hand support.
 - The elbow should be fully supported by the skateboard. To adjust for forearm length, remove the hand support and extend it.
2. Fasten the straps on the hand support over the MCPs and the first phalanges.

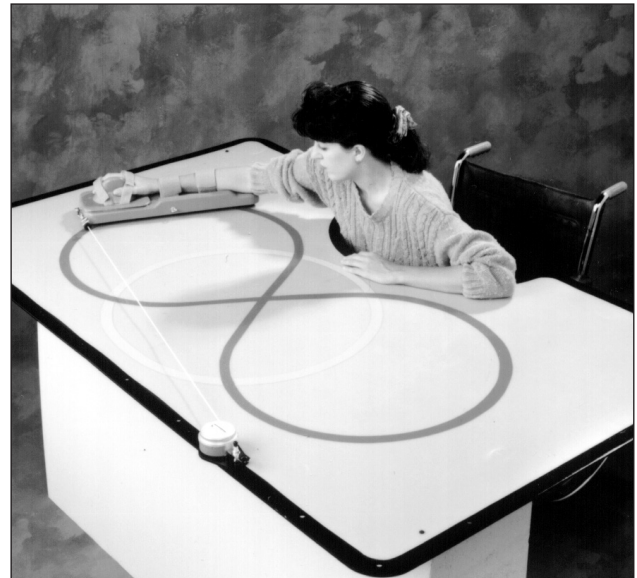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Distributed by:
Performance Health Supply Inc.
28100 Torch Parkway, Ste. 700
Warrenville, IL 60555, USA
Phone: 800-323-5547
Fax: 800-547-4333

Manufacturer:
Performance Health Supply Inc.
W68 N158 Evergreen Blvd.
Cedarburg, WI 53012, USA
Phone: 800-228-3693

Distributed by:
Performance Health Canada, Inc.
6675 Millcreek Drive, Unit 3
Mississauga, ON L5N 5M4
Phone: 800-665-9200
Fax: 877-566-8611

Authorized European Representative
Performance Health International, LTD.
Nunn Brook Road
Huthwaite, Sutton-in-Ashfield
Nottinghamshire
NG17 2HU, UK
International
Phone: +44 (0) 1623 448 706
Fax: +44 (0) 1623 448 784