## Rolyan® TakeOff® Thumb Support:


A952-712 and A952-713

## Rolyan® TakeOff® Custom Thumb Support:


A952-700 and A952-701

## Rolyan® TakeOff® Gel Shell® Thumb Support:


A952-710 and A952-711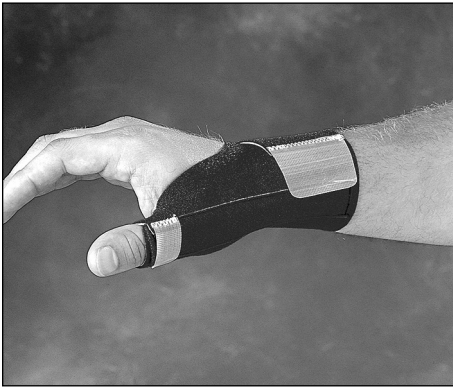

A

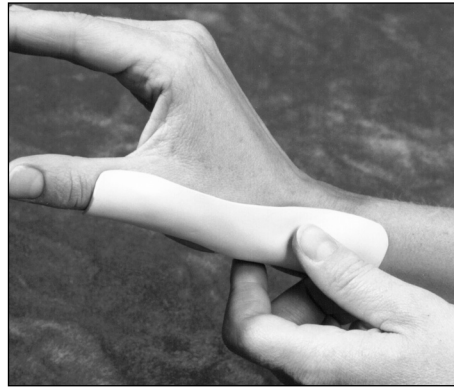B

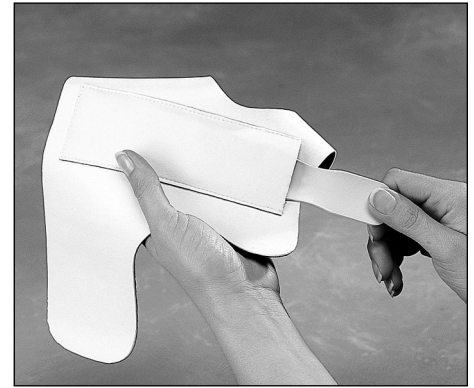C

*Be sure to note which brace you are using and follow the instructions that apply to that brace. Unless otherwise stated, these instructions apply to all of the braces listed.*

### INDICATIONS

- To help protect the thumb to allow healing following injury

#### TakeOff Thumb Support

- May be used for arthritis and tendonitis and other repetitive motion injuries of the thumb

#### TakeOff Custom Thumb Support

#### TakeOff Gel Shell Thumb Support

- May be used for conservative treatment for deQuervain's syndrome

#### TakeOff Gel Shell Thumb Support

- May be used to decrease pain and hypertrophic scarring following surgery and suture removal

### CONTRAINDICATIONS

- Not for severe sprains, immediate treatment following fractures, or if rigid support is required
- Not for use directly over open wounds and infected areas
- May not be for people with circulatory problems

### PRECAUTIONS

- If **unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.**
- **The brace should fit snugly and comfortably without interfering with circulation.**

#### TakeOff Custom Thumb Support

#### TakeOff Gel Shell Thumb Support

- **This brace is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.**
- **After heating splinting material, always check its temperature before applying it to the patient.**

D

E

### CARE AND CLEANING

Prior to cleaning, remove optional stay and Gel Shell pad from brace and fasten hook closures.

#### Fabric splint

- For extended use, soak the fabric for five minutes in a solution of mild detergent and cold water. Rinse the detergent out completely. Do not wring. Allow to air dry thoroughly before reuse.

#### Gel Shell pad (TakeOff Gel Shell Thumb Support only)

- Hand wash in warm water (hot water may yellow the pad) and mild soap. Rinse thoroughly. Press gently to remove excess water and let it air dry. Be sure the pad is completely dry before reapplying it.

#### Polyform® Light stay (TakeOff Custom and Gel Shell Thumb Supports only)

The stay will lose its shape in temperatures over 135°F (57°C). It should be kept away from sources of heat such as ovens, hot water, open flames, and sunny car windows.

### INSTRUCTIONS FOR USE

#### TakeOff Thumb Support

1. Place the brace on a flat surface with the foam (white) side up.
2. Position the hand on the brace so the wider end of the brace is at the wrist and fasten the long strap around the wrist.
3. Be sure the stay is lined up with the thumb nail and fasten the short strap around the thumb.

4. A properly applied TakeOff Thumb Support is shown in **A**.

#### TakeOff Custom Thumb Support

#### TakeOff Gel Shell Thumb Support

1. **Optional stay:** Position the indented part of the stay between the MCP and CMC joints. The stay should fit along the radial aspect of the thumb, wrist and partial forearm (**B**). The distal end should lie proximal to the IP joint. If necessary, the stay can be molded for a more exact fit by heating it for 30 seconds in water heated to 150° to 160°F (65–70°C). Place the stay in the pocket in the brace (**C**).
2. **Optional Gel Shell pad (TakeOff Gel Shell Thumb Support only):** Fasten the back of the pad to the surface of the pocket for the stay (**D**). Remove the protective plastic sheet from the surface of the gel pad.
3. Place the brace on a flat surface with the foam (white) side up. Position the hand on the brace so the wider end of the brace is at the wrist and fasten the long strap around the wrist. Fasten the short strap around the thumb.
4. A properly applied TakeOff Custom Thumb Support and TakeOff Gel Shell Thumb Support as shown in **E**.

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**Patterson Medical Ltd.**