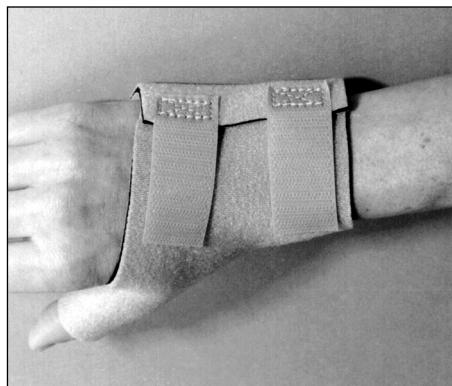
Rolyan® TAP® (Tone and Positioning) Splint:

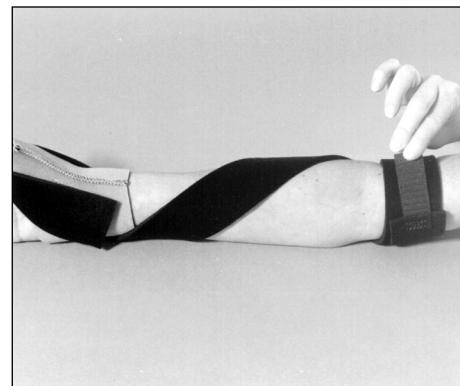
A834-54 through 67


A


B


C

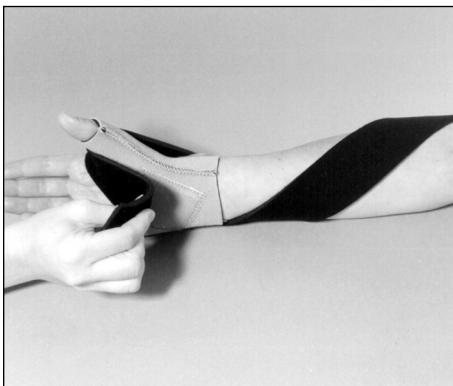D

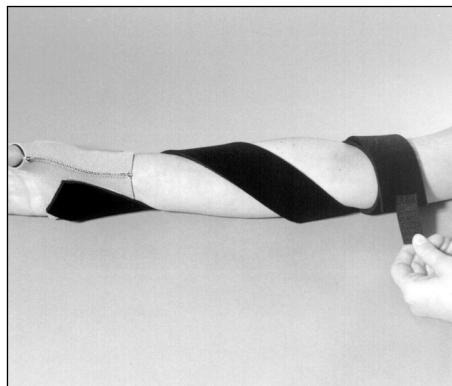E

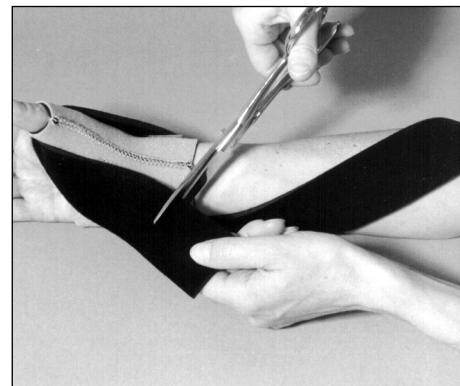F

INDICATIONS

- For functional positioning of the forearm and hand

CONTRAINDICATIONS

- Not for severe hypertonicity
- Not for use with severe edema
- Not for people who have an allergic reaction to neoprene or who have a tendency to develop dermatitis
- Not for use over open wounds and infected areas
- May not be for people with circulatory problems

PRECAUTIONS

- This splint is to be fitted initially by a healthcare professional who is familiar with the purpose for which it is prescribed. The healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners, care providers involved in the patient's care, and the patient.
- If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.
- Be careful that the splint is not so tight that circulation is impaired.

CARE AND CLEANING

- Wash frequently to remove body oils and other soil.
- Fasten closures. Hand wash fabric in warm water and mild soap. Rinse thoroughly. Do not wring. Dry at room temperature. Be sure the splint is completely dry before reapplying it.

- Do not use ointments or oils under the splint.
- Do not dry clean or expose to heat or sunlight for extended periods of time.

APPLICATION

Applying glove

1. Open straps.
2. With hook pad on volar surface, put glove onto hand and close straps (**A** and **B**).
3. If necessary, the glove can be trimmed. Check the following areas for proper fit:
 - It should clear the distal palmar crease.
 - It should clear the MCPs on the dorsum of the hand.
 - It should cover the ulnar border of the hand.

Do not trim the thumb through the stitching.

Applying splint so forearm is in supination

4. Using the end that does not have the hook tab, attach the strap to the hook pad on the volar surface of the glove. Wrap the strap through the thumb web space, over the dorsum of the hand, around the ulnar aspect of the wrist, and over the volar surface of the middle of the forearm (**C**).
5. Wrap the strap around the arm either distal or proximal to the elbow (not directly over it) until the hook tab can be fastened to the strap to form a cuff (**C**).
6. Adjust the distal end of the strap for final fit by lifting the strap off the hook and gently stretching the strap for the appropriate force of pull and desired amount of supination. Reattach the strap to secure it (**D**).
7. If necessary, trim the length of the strap (see instruction 12).

Applying splint so forearm is in pronation

8. Using the end that does not have the hook tab, attach the strap to the hook pad on the volar surface of the glove. Wrap the strap toward the ulnar side of the wrist and around the mid-point of the dorsum of the forearm. Bring it to the volar surface of the forearm (**E**).
9. Wrap the strap around the arm either distal or proximal to the elbow until the hook tab can be fastened to the strap to form a cuff (**E**).
10. Adjust the distal end of the strap for final fit by lifting the strap off the hook and gently stretching the strap for the appropriate force of pull and desired amount of pronation. Reattach the strap to secure it.
11. If necessary, trim the length of the strap (see instruction 12).

Trimming strap

12. If the splint will be used for only supination or only pronation, trim the length of the strap after the final fit (**F**). If using the splint alternately for pronation and supination, trim the strap during the final fit in supination only to avoid over-trimming it.

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Patterson Medical Ltd.