

 **Manufactured for:**  
Dynarex Corporation  
10 Glenshaw Street  
Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)

Made in China


# Aluminum Crutches



## OPERATING INSTRUCTIONS

**PLEASE READ THROUGH THE FOLLOWING INSTRUCTIONS BEFORE USING, ASSEMBLING OR INSTALLING THIS PRODUCT.**

1. Height: Adjust height by pressing both push-buttons, rotating foot-piece, and pulling down until desired height is obtained. Rotate foot-piece again until both push-buttons snap back into place.
2. Hand grip: Adjust the hand grip to fit with a slight bend at the patient's elbow
3. Safety check: With crutches positioned per diagram below, confirm patient can lean forward comfortably while resting on the underarm supports. Patient should not have to stand uncomfortably erect. Ensure both push-buttons are firmly in place and hand grip wing nuts are tightened securely.

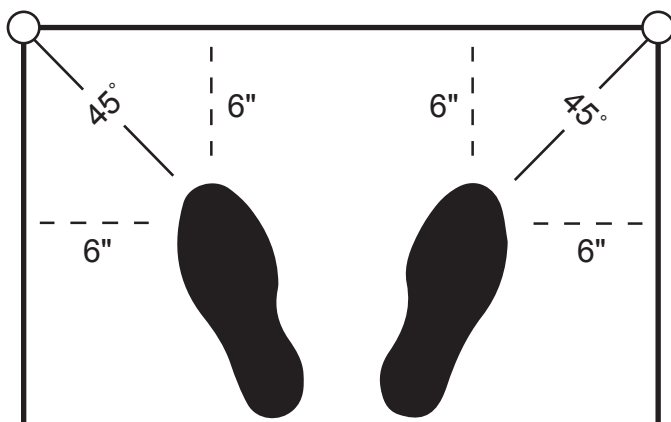

## GENERAL CARE

### CHROME AND METAL PARTS

Wipe chrome parts with a clean, soft cloth at least once a week. All metal parts can be cleaned with auto wax or similar products. If exposed to moisture, dry immediately. Do not use an abrasive cleaner that will scratch the finish.

### HAND GRIPS AND UNDERARM PADDING

Use only a clean towel dampened with cool water when cleaning hand grips and underarm padding. Confirm that the grips are dry before using. NEVER use ammonia-based cleaning solutions or soap on hand grips and underarm padding. This may cause the hand grips and underarm padding to loosen or crack.

### RUBBER TIPS

Periodically check the rubber tips for rips, tears, cracks or worn tread. If any of these conditions are found, contact your health representative immediately.

 **Manufactured for:**  
**Dynarex Corporation**  
**10 Glenshaw Street**  
**Orangeburg, NY 10962**  
**USA • [www.dynarex.com](http://www.dynarex.com)**

Made in China

# Aluminum Crutches



## **WARNING**

**DO NOT ATTEMPT TO ADJUST OR OPERATE CRUTCHES WITHOUT READING ALL INSTRUCTIONS CAREFULLY.**

- Follow your healthcare provider's instructions for proper use, and weight bearing on your affected leg.
- Make sure that you are correctly fitted, all push-buttons are fully engaged, all screws are tightened, and hand grips are attached securely.
- Keep your wrists straight when using crutches.
- Carry your full weight on your hands, not your underarms.
- Use extra caution on wet and slippery surfaces.
- Do not exceed the maximum weight capacity or serious injury could result.

## **LIFETIME LIMITED WARRANTY**

Your Dynarex product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions: Problems arising from normal wear; Problems arising from failure to adhere to the product instructions; Problems arising from misuse, negligence, accident or improper operation, maintenance or storage; Problems arising from modifications or unauthorized repairs, parts or attachments; Products where the serial number has been removed or defaced; Problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.



### **SYMBOL GLOSSARY**

For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)

 **WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

R200708