

## LIMITED LIFETIME WARRANTY

Your Dynarex product is warranted to be free of defects in materials and workmanship for the lifetime of the product for the original purchaser. This item was built to exacting standards and carefully inspected prior to shipment. This Limited Lifetime Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

The Warranty shall not apply under the following conditions: problems arising from normal wear; problems arising from failure to adhere to the product instructions; problems arising from misuse, negligence, accident or improper operation, maintenance or storage; problems arising from modifications or unauthorized repairs, parts or attachments; products where the serial number has been removed or defaced; problems with non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement. Dynarex shall not be liable for any consequential or incidental damages whatsoever. Dynarex shall repair or replace defective products at its option. The foregoing warranty is exclusive and in lieu of other express warranties, if any, including the implied warranties of merchantability and fitness of a particular purpose. The remedy for any violation of the implied warranty shall be limited to repair or replacement of the defective product pursuant to the terms contained herein.

**⚠ WARNING:**  
Cancer and Reproductive Harm  
[www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)




**SYMBOL GLOSSARY**  
For an explanation of symbols  
used in Dynarex packaging, visit  
[dynarex.com/symbols.php](http://dynarex.com/symbols.php)

R181115

Reorder Nos. 10131-10134  
10136-10138

 Manufactured for:  
Dynarex Corporation  
10 Glenshaw Street  
Orangeburg, NY 10962  
USA • [www.dynarex.com](http://www.dynarex.com)  
Made in China

# Cane

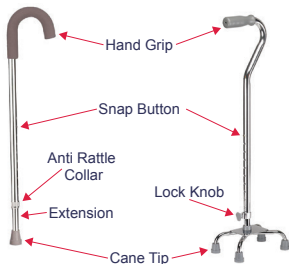

## FOR USERS TO READ BEFORE USE

For maximum safety, these precautions must be taken when using this product. DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment – otherwise, injury may occur.

## BEFORE EACH USE MAKE SURE:

1. Use only as directed by a health care provider.
2. Length must be correctly adjusted to user's height.
3. Spring button must be engaged and fully protruding through the adjustment hole and the collar must be tightened.
4. Check that tips are not worn, ripped, cracked, or have excessive wear. If any of these conditions exist, replace the rubber tip immediately.
5. The cane handle should be fully inserted into the hand grip.
6. Do not use if the user's weight is over 250 lb. for canes, 300 lb. for quad canes, and 500 lb. for bariatric canes.
7. Canes are designed to provide support, increased stability and assistance while walking. The canes are NOT designed to support the total weight of individuals.
8. Do not use if user has any handicaps and/or disabilities or is taking any medication, etc. that a doctor, therapist and/or seller believes could cause interference with the safe use of the product.
9. Do not use if there are any loose components, rattles, cracks, dents or other uncustomary characteristics in the metal, plastic, etc.
10. Do not subject the cane to sudden impact or jerks.
11. Use the utmost care when using a cane. Watch out for potential safety hazards including, but not limited to, slippery, uneven or soft surfaces and objects in your path.

## HEIGHT ADJUSTMENT

1. Loosen the anti-rattle collar. Depending on the model, either turn the collar counterclockwise to loosen, turn the lock knob to loosen, or lift the plastic lever on the collar.

2. Press the snap button so that it enters the shaft of the cane.
3. Slide extension up or down until snap button protrudes through an adjustment hole on handle end of cane.  
**NOTE:** An audible "CLICK" will be heard when snap button securely protrudes through the adjustment hole.
4. Repeat steps 2-3 until cane is at height determined. PROPER CANE HEIGHT below.
5. Tighten the anti-rattle collar.

## PROPER CANE HEIGHT

For a more comfortable fit, ensure shoe height is the same as shoe style worn most frequently when using cane:

1. Stand as upright as possible on a firm, flat surface and let your arm hang down naturally by your side.
2. Adjust cane so top portion of handle touches inside of wrist where it meets palm.

## ADJUSTING QUAD CANE FOR RIGHT/LEFT USE

Quad canes (Reorder Nos. 10136-10138) can be adjusted for both left or right hand use by following these easy steps:

1. Loosen the lock knob.
2. Depress the spring button while rotating the handle 180 degrees.
3. Engage the spring button in the hole which accommodates the user's height.
4. Retighten the lock knob. **NOTE:** *the 45 degree angled legs on quad canes should be placed farthest from the user's body when the cane is being used.*

## CLEANING

Clean the grips with a mild soap, detergent or household cleaner. Do not use an organic solvent (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, gasoline or turpentine) that may dissolve the plastic and cause the grip to loosen.