

Reorder No. 10364

 Manufactured for:
Dynarex Corporation
10 Glenshaw Street
Orangeburg, NY 10962
USA • www.dynarex.com
Made in China

Elongated Raised Toilet Seat



USER WEIGHT LIMIT: 300 lb.

INCLUDED IN BOX



Elongated Raised Toilet Seat



Screw
x2



Wing Nut
x2




Washer (A)
x2




Washer (B)
x2

INSTALLATION

1. Remove your toilet seat.

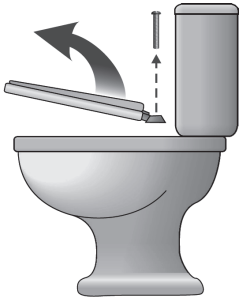

2. Place your Elongated Raised Toilet Seat on top of the riser, placing your toilet seat on top of the riser, lining up the holes of the riser and the seat with the holes on the toilet frame.

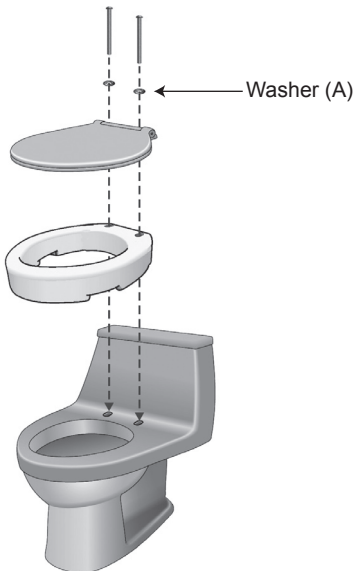

3. Put the enclosed bolts thru the holes, attaching the Elongated Raised Toilet Seat to the toilet frame with the nuts provided.



WARNING! Before using your Elongated Raised Toilet Seat, push on the unit firmly to ensure that it is securely attached.



SYMBOL GLOSSARY

For an explanation of symbols used in Dynarex packaging, visit dynarex.com/symbols.php

WARNING:
Cancer and Reproductive Harm
www.P65Warnings.ca.gov