

# Rolyan® Kwik-Form™ Plus Universal Hand Orthosis

7879-00 Rolyan Kwik-Form Plus Universal Hand Orthosis, Small, Regular  
7879-01 Rolyan Kwik-Form Plus Universal Hand Orthosis, Medium, Regular  
7879-02 Rolyan Kwik-Form Plus Universal Hand Orthosis, Medium, Firm  
7879-04 Rolyan Kwik-Form Plus Universal Hand Orthosis, Large, Firm



## Indications

Provides static, functional positioning to wrist, hand and fingers. For mild to moderate hypertonicity.

## Contraindications

Not to be used over open wounds.  
Not to be used for severe hypertonicity.

## Precautions

Orthosis needs to be fitted initially by healthcare professional familiar with the purpose for which it is prescribed.

Healthcare professional is responsible for providing wearing instructions and precautions to other healthcare practitioners or care providers involved in patient's care.

If unusual swelling, skin discoloration or discomfort occurs, use should be discontinued and a healthcare professional consulted.

Be careful that the straps are not so tight that they interfere with circulation.

Single patient use only.

This splint is not heat moldable.

## Instructions for Use

Perform passive range of motion to determine the best position for the patient's hand.

If thumb stabilization is necessary, adjust the thumb portion to support the thumb in a therapeutic alignment.

Adjust the splint to fit the patient. It may be necessary to use the edge of a stable, hard surface to bend and contour the splint.

To prevent ulnar deviation or radial drift, bend the wings on either side of the fingers.

Bend and fit the wrist to a proper position.

When proper shape is achieved, open the hook and loop closures. Apply the orthosis to the extremity; then adjust and set the strap in position, using the shape matching system.

Maintain a two-finger gap under the straps. This will prevent pressure on the skin.

Check every two hours for redness and swelling or follow facility's splint wearing protocol.

Evaluate periodically, per physician's orders, for usefulness and compatibility of the orthosis. It could be necessary to perform adjustments to the splint periodically.

## Washing Instructions

Remove cover from frame. Engage all hook and loop closures. Cover can be hand washed or machine washed on gentle cycle (in laundry bag provided) with mild soap and water. Air dry.

Wipe down frame with mild soap and water. Reassemble after splint is completely dry.

## Limited Product Warranty

Patterson Medical warrants that all products shall be free from manufacturing defects for a period of one year from the date of shipment. OTHER THAN THE FOREGOING WARRANTY, PATTERSON MEDICAL MAKES NO REPRESENTATION OR WARRANTY OF ANY KIND, EXPRESS OR IMPLIED, AS TO MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR ANY OTHER MATTER RELATING TO THE PRODUCT. Customer must notify Patterson Medical of any manufacturing defect within five days of discovery of the defect. Upon a determination by Patterson Medical that the product is defective, Patterson Medical sole and exclusive obligation shall be to provide customer with a replacement product or a full refund of the purchase price, at the sole discretion of Patterson Medical, and to reimburse customer for any return freight costs. In no event shall Patterson Medical be liable for any other costs or expenses including lost profits or direct, indirect, consequential, incidental or punitive damages. The warranty set forth herein shall govern all purchases, notwithstanding any inconsistent terms and conditions of customer's purchase order.



**Distributed by:**  
**Patterson Medical**  
28100 Torch Parkway, Ste. 700  
Warrenville, IL 60555  
Phone: 800-323-5547  
Direct Phone: 630-393-6000  
Fax: 800-547-4333  
Direct Fax: 855-480-1946

**Manufactured for:**  
**Patterson Medical**  
W68 N158 Evergreen Blvd.  
Cedarburg, WI 53012  
Phone: 800-323-5547  
Fax: 800-547-4333

**Authorized European Representative**  
**Patterson Medical, Ltd.**  
Nunn Brook Road  
Huthwaite, Sutton-in-Ashfield  
Nottinghamshire, NG17 2HU, UK

**United Kingdom**  
Phone: 08444 124 330  
Fax: 08448 730 100  
International  
Phone: +44 (0) 1623 448 706  
Fax: +44 (0) 1623 448 784

© September 2013  
Patterson Medical  
Products, Inc.  
99104.009