**ENGLISH**  
BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

**INTENDED USER PROFILE:**  
The user profile describes for whom a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use.

**INDICATIONS:** Helps alleviate pain associated with plantar fascitis. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**CONTRAINDICATIONS:** N/A

**WARNINGS AND PRECAUTIONS:**

- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- Do not use this device if it was damaged and/or the package has been opened.

**NOTE:** Contact manufacturer and competent authority competent in case of a serious incident arising due to usage of this device.

**APPLICATION INSTRUCTIONS:**

**NOTE:**

- The splint should be applied with the knee bent to maximize the angle of joint dorsiflexion.
- Before applying the splint overnight, apply the splint for one hour then remove. Check for any irritation or discoloration of the foot or leg. If there is irritation or discoloration, the splint may need to be adjusted. Refer to the Modifying the Night Splint section for instructions.
- The Night Splint may be worn at night while reclining or sitting.
- Non-skid material on bottom allows for limited ambulation.
- 1) Place the foot into the splint. Ensure that the heel is securely seated and that the big toe is within the confines of the splint.
- 2) Wrap the instep strap over the center of the ankle joint and secure in place using a hook closure.
- 3) Once the instep strap has been applied, straighten the leg out. Check that the length of the instep strap is correct.
- NOTE: If the heel rides up the splint, the instep strap is too loose. If the calf contracts, the instep strap may be too tight. Adjust the instep strap and recheck the tightness.
- 4) Once the instep strap is fit comfortably, secure the remaining straps along the foot and leg.

**NOTE:** Ensure that the instep strap is well padded to prevent irritation. Cotton socks can be worn under the splint to provide additional protection. (Fig. 1)

**INSERTION OF WEDGE:** With the splint in place, slide the thin end of the foam wedge (blue side up) between the foot and splint. The thin end of the foam will rest under the heel and the thick end will rest under the big toe.

**PRECAUTION:** Do not apply the foam wedge until instructed to do so by a physician or other health care professional.

**MODIFYING:**

- 1) Remove the cover, from the insert by folding the toe portion of the insert against the back portion of the insert, and sliding the insert out of the cover.

**NOTE:** If any part of the splint is too tight, manually spread the width of the insert.

- 2) Heat the insert using a heat gun or hair dryer. Direct the heat towards the point of the insert that needs adjustment.

**CAUTION:** Overheating the plastic may cause the plastic to melt.

- 3) Once the plastic is warmed up it becomes moldable and can be adjusted to any position desired.

**CAUTION:** Do not heat the plastic while the patient is wearing the splint. Allow the insert to cool down before use. Do not apply heated insert to either the cover or the skin.

- 4) Replace the cover, on the insert, by folding the toe portion of the insert against the back portion of the insert and sliding the insert into the cover.

**USE AND CARE:**

- Remove cover before washing by folding the toe portion of the insert against the back portion of the insert, and sliding the insert out of the cover. Once dry, replace the cover by folding the toe portion of the insert against the back portion of the insert, and sliding the insert into the cover.
- Hand wash the cover in warm water using mild soap. Allow the cover to air dry at room temperature. Do not dry the cover using a washing machine to clean the cover. Do not dry the cover using a heat gun.

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

**FOR SINGLE PATIENT USE ONLY.**

**NOT MADE WITH NATURAL RUBBER LATEX.**

**NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.**

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