

limited lifetime warranty

Your Drive branded product is warrantied to be free of defects in materials and workmanship for the lifetime of the product for the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2014 Medical Depot, Inc. All rights reserved.
Drive is a trademark of Medical Depot, Inc.
Port Washington N.Y. 11050 USA Made in China

economy I.V. pole
with removable top
4 hook, 4 leg

item # 13029



EU Authorized Representative
Drive Medical LTD
Ainley's Industrial Estate
Elland, West Yorkshire,
United Kingdom HX5 9JP

www.drivemedical.com


A large, vertical version of the Drive logo, with the word "drive" written in a bold, white, lowercase sans-serif font. A small yellow square is positioned above the letter "i". A trademark symbol (TM) is located at the top right of the word.

assembly instructions

- 1 Overlay top leg (A) with pre-assembled pin (F) over corresponding holes in the middle of the bottom leg (B)
Hammer the pin (F) into bottom leg and making sure that it's flush with the top leg.
- 2 Place lock washer (D) onto bolt (E) and install from the bottom side of the base into bottom of post (C).
- 3 Snap the hook (G) over the top end of post (C) until the snap pins pop through the two holes in the tube portion of the hook.

parts list

code	description	qty
a.	Top Leg (w/insert)	1
b.	Bottom Leg	1
c.	Post	1
d.	Lock Washer	1
e.	Lock Washer	1

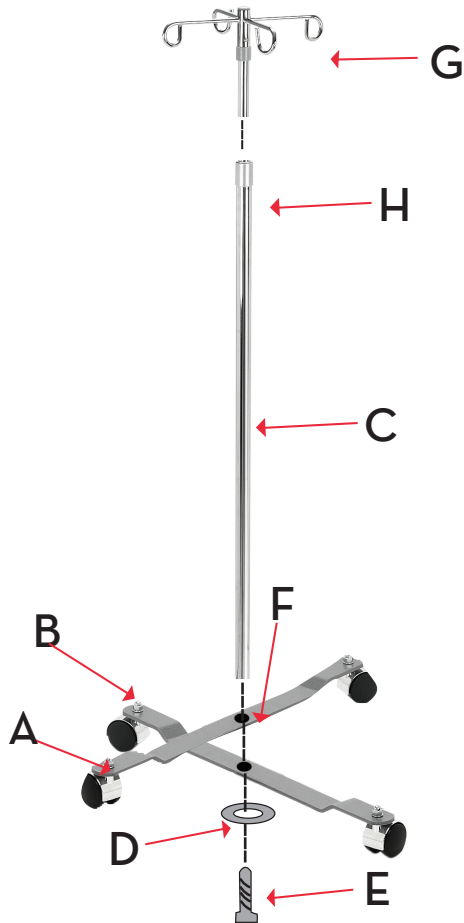

driveTM