

Limited Warranty

Foot Specialists Since 1885, PediFix guarantees your satisfaction with every product.

This product is warranted to be free from defects in material and workmanship for one (1) year from the date of original purchase.

The UV lamps carry a warranty of 60 days. These warranties do not cover normal use, wear and tear, breakage or abuse. If this product is determined to be defective, PediFix will replace the product at no charge.

For replacement, return the product with proof of purchase to: PediFix, Inc., 281 Fields Lane, Suite 1, Brewster, NY 10509 USA


This warranty does not cover damage due to acts of nature, accident, misuse, abuse, negligence or modification of, or to any part of the product.

This warranty does not cover damage due to improper operation, maintenance, or attempted repair by anyone other than the manufacturer. This warranty does not cover products sold AS IS or WITH FAULTS. Proof of purchase on the form of a bill of sale, receipt or invoice, which is evidence that that unit is within the warranty period, must be presented to obtain warranty service.

All returns should be sent to PediFix. Send only the product and proof of purchase to the address above. You may also contact our customer service department at 1-800-PEDIFIX (733-4349), Monday thru Thursday, 9 AM to 5 PM, Fridays, 9 AM to 2:00 PM, Eastern Time.

Warranty disclaimers: Any implied warranties arising out of this sale, including but not limited to the implied warranties of description, merchantability and fitness for a particular purpose, are limited in duration to the stated warranty coverage. In no event shall PediFix, Inc. (PediFix) be liable for loss of use of this product or for any indirect, special, incidental, or consequential damages, or costs, or expenses incurred by the consumer or any other user of this product, whether due to a breach of contract, negligence, strict liability in tort or otherwise. PediFix shall have no liability for any personal injury, property damage of any kind resulting from the use of this product. Some states do allow limitations on how long an implied warranty lasts or the exclusion or limitation of consequential or incidental damages, so the above limitation may not apply to you.

Legal remedies: This warranty gives you specific legal rights and you may also have other rights that vary from state to state.

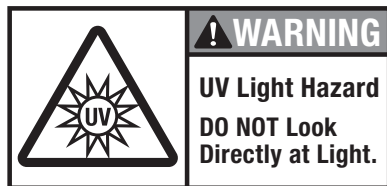

Made in China exclusively for the PediFix® Footcare Company

PediFix, Inc.
281 Fields Lane, Suite 1, Brewster, NY, 10509 U.S.A

Questions? Comments? Replacement Bulbs?
Call 1-800-PEDIFIX (733-4349)

Email info@pedifix.com
Visit www.pedifix.com




Scan for more information!



UltraViolet Shoe Sanitizer

User Manual

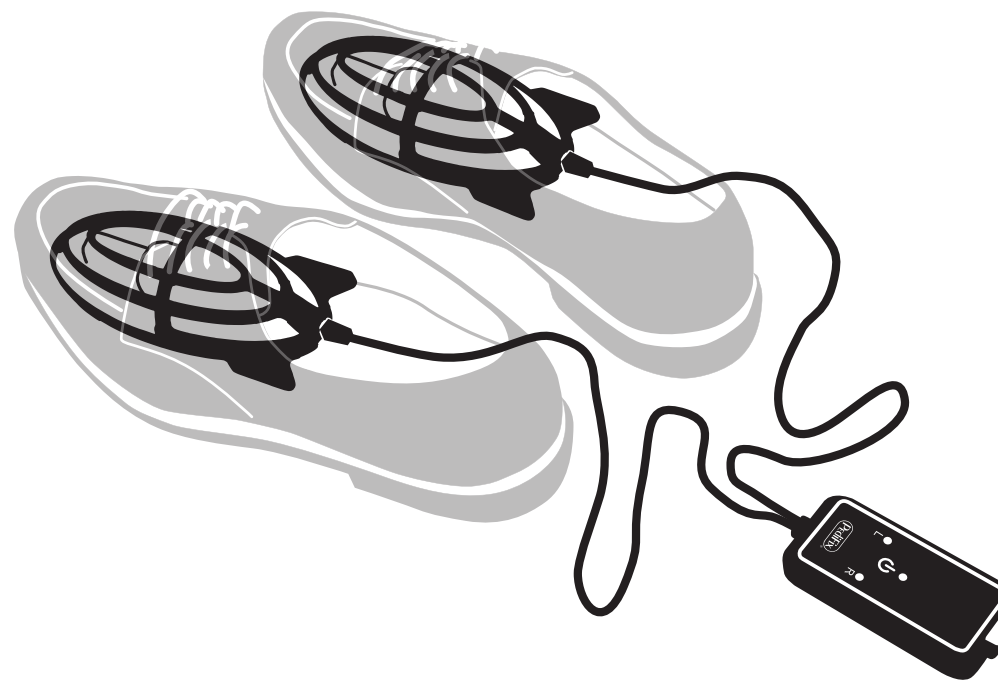

UltraViolet Shoe Sanitizer

User Manual

Table of Contents

Important Safeguards	3
Introduction	5
Product Benefits	6
Contents	7
Production Specifications	7
Using the UV Sanitizer	8
Important Safety Notice	10
Disclaimers	10
Troubleshooting	11
Warranty Information	12

IMPORTANT SAFEGUARDS

When using electrical appliances, basic safety precautions should always be followed carefully.

PLEASE READ ALL INSTRUCTIONS BEFORE USING THE PRODUCT SAFETY GUIDELINES – DEFINITIONS

For your SAFETY and PROTECTION, it is very important for you to read and understand this instruction manual before use. Please pay particular attention DANGER, CAUTION, and WARNING sections as they are designed to call particular attention to potential dangers when using any electrical product or appliance.

DANGER: Indicates an imminently hazardous situation which, if not avoided, will result in serious injury or death.

WARNING: Indicates a potentially hazardous situation which, if not avoided, could result in serious injury or death.

CAUTION: Indicates a potentially hazardous situation which, if not followed, may result in minor or moderate injury.

DANGER: TO REDUCE THE RISK OF ELECTRIC SHOCK:

- Always unplug the Sanitizer from an outlet when not in use.
- Do not operate/drop near water or while bathing or swimming.
- If the device falls into water, unplug IMMEDIATELY before reaching for the device.
- Do not use if cord is damaged.
- Keep the cord away from heated surfaces.
- Do not insert any objects into any openings.
- UV lamp is fragile, use with caution.

WARNING: TO REDUCE THE RISK OF DAMAGE TO YOUR EYES, ELECTROCUTION, FIRE OR PERSONAL INJURY:

- KEEP OUT OF REACH OF CHILDREN
- Only use as intended. Use according to this manual's instructions without tampering or modifying the device in any way.
- Close supervision is necessary when any appliance is used by or near children and people with special disabilities.
- Do not allow the device to be used as a toy.
- To avoid UV exposure to eyes, always insert Sanitizers into shoes, and place shoes with Sanitizer into protective Shoe Bags.
- Do not look directly at the lamp.

- Shock hazard. To protect against electrical shock, do not put unit or power supply cords in water or other liquids.
- Do not use if device is damaged, not working properly or has broken lamps.
- Do not operate in the presence of explosives and/or flammable fumes, smoke or liquids.
- Do not use outdoors or on wet surfaces.
- Do not handle the device with wet hands.
- Do not use on wet or damp shoes. Allow shoes to dry before using.
- Do not abuse the cord. Never carry the device by the cord or yank to disconnect from an outlet. Instead, gently grasp the electrical plug and pull to disconnect.
- Never carry device by its ultraviolet light or mishandle the bulb.
- Do not allow the cord to hang over the edge of a table or counter or touch hot surfaces.
- Use the charger only in a standard electrical outlet (100V-240V)
- Use only as described in this manual. Use only with manufacturer's recommended attachments and materials.

CAUTION: TO REDUCE THE RISK OF COMMON INJURY OR HEALTH PROBLEMS:

- THIS DEVICE USES ULTRAVIOLET LIGHT (UV) TO KILL HAMFUL BACTERIA
- Do not look directly at the light.
- Never turn on the device until it is placed inside the shoe and covered completely by the protective bags.
- Do not remove the device from the shoe while it is still turned ON.
- If the device does not automatically turn off after approximately 15 minutes, the timing mechanism may be malfunctioning. Turn off the device, disconnect the power source and contact us before you attempt to reuse at 1-800-PEDIFIX.
- The device is not recommended for use by people with reduced mental, physical, and sensory capabilities. Extreme caution should be used along with careful supervision.
- If the device is dropped and/or if the ultraviolet light is damaged or broken, discontinue use.
- The unit should be placed inside protective bags at all times when in use.

SAVE THESE INSTRUCTIONS

Introduction

Congratulations and Thank You for your purchase of our UltraViolet Shoe Sanitizer

Ultraviolet light technology has been used to fight germs in hospitals, nursing homes, to clean water, air and other environments for years. This same, clinically-proven technology is now available to safely clean and disinfect shoes.

The UV Shoe Sanitizer is designed to use the germicidal power of UV light to clean the surface areas inside your shoes. No chemicals are used in this process. The power of UV light, along with a short period of time, is all that is needed.

UV Shoe Sanitizing technology has been clinically tested and proven safe and effective.

UV Shoe Sanitizer:

- Sanitizes shoes in just 15 minutes.
- Scientifically proven to destroy microorganisms.
- Effective on the microbes that can cause toenail fungus, athlete's foot, shoe and foot odor.
- Non-toxic. Uses no hazardous or toxic chemicals, sprays, or drugs to kill bacteria, fungi, and germs.
- ONE SIZE FITS ALL to treat every shoe in your home.
- Approved and recommended for use by physicians.
- Easy, safe, and convenient to use.

Carefully read this manual to enhance your experience with this innovative product. We look forward to hearing from you and welcome your comments and suggestions. To contact us, please:

Call: 1-800-PEDIFIX

Visit: www.PediFix.com

E-mail: info@pedifix.com

Write: PediFix, 281 Fields Lane, Suite 1, Brewster, NY 10509

Product Benefits

Why Do Your Feet/Shoes Stink?

Blame it on the bacteria. They live on our skin and thrive in dark, damp places such as the insides of your shoes.

When bacteria inside your socks and shoes metabolize or ‘eat’ sweat, they create Acetic, Butanoic and 3-methylbutanoic acids. It is these organic compounds that smell so terrible.

Some people’s feet smell worse than others because they sweat more profusely. If you’re one of the unlucky, your feet are home to bacteria called *Micrococcus Sedentarius* that produce more than just stinky organic acids — they also produce sulfur compounds that smell awful, similar to rotten eggs.

Use Your UV Shoe Sanitizer Daily for Best Results

The combination of sweat, bacteria and fungi can cause toenail fungus, athlete’s foot, and foul odor. Even if these conditions are treated, your shoes can be a common source of re-infection.

We recommend that you change the UV lamps in your Sanitizer every 2 years to guarantee maximum effectiveness in killing harmful microbes in your shoes.

How the UV Shoe Sanitizer works

Your UV Shoe Sanitizer kills germs that reside inside shoes with a special germicidal ultraviolet light (UVC).

With the first treatment, the Sanitizer destroys up to **99.9%** of targeted microorganisms, including the bacteria and fungi that can cause toenail fungus (onychomycosis), athlete’s foot (tinea pedis) and offensive foot and shoe odor.

Contents

- 2 Shoe Sanitizers (1 pair)
- 2 Protective Shoe Bags
- 1 User Manual
- 1 A/C Power Adapter
- 1 UV Warning Card
- 1 Product Registration Card

Product Specifications:

- Working voltage: 10V
- Power Adapter:
Input 100-240V, 50/60 Hz, 0.56A Ma., Class II, Output 20 VDC, 600 mA
- UV bulb: C Class, 3 Watt, Ozone-FREE
- UV wavelength: 253.7 nm

The UV bulbs are designed to last for 2000 hours when the UV Sanitizer is used as directed.

Using Your UltraViolet Shoe Sanitizer

Read All Instructions Before Using This Product

Your UV Sanitizer comes with important safety warnings, to prevent exposure to the UVC light. The following is a quick reference on how to safely operate your UV Shoe Sanitizer.

Use the UV Shoe Sanitizer only as directed. To prevent errors when using the Sanitizer, please follow these directions:

General Use Instructions:


Step 1

Insert Right and Left Sanitizer completely into Right and Left shoes, with bulb and flat-side facing down.




Step 2

Place each shoe with Sanitizer into a Protective Shoe Bag. Close Shoe Bag completely with drawstrings.

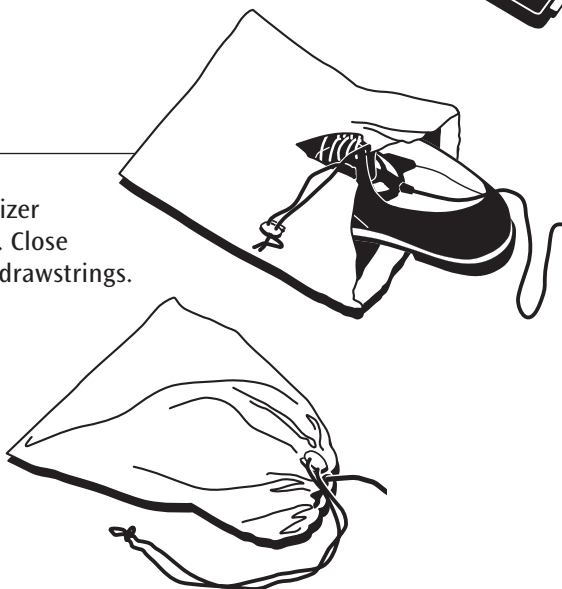

Step 3

Connect the A/C adapter to the Sanitizer On-Off Control Switch. Plug the A/C adapter into a power supply.

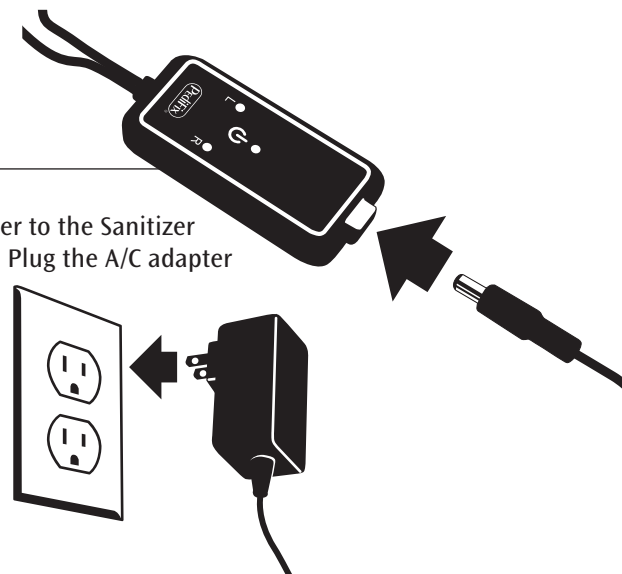


Step 4

Press ON/OFF button to begin shoe treatment. The Red power light will illuminate. Note 1 minute delayed start until the Right and Left Blue UV indicator lights illuminate. Sanitizer lamps are now ON.

Step 5

Sanitizer turns off automatically after 15 minutes. Only after Blue indicator lights and UV lamps are OFF it is safe to remove shoes from Bags and Sanitizer from shoes.

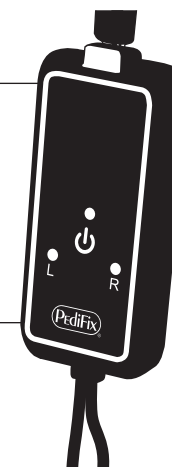

Safety warnings:

Do not remove shoes from Shoe Bags or Sanitizer from shoes while the device is on. Caution: Bulbs are Hot During and After Use — Do Not Touch

If you need to remove a Sanitizer from a shoe during the treatment process, press the ON/OFF button to turn device off. Unplug device. It is safe to remove the Shoe Sanitizer when the ON/OFF button is not illuminated.

Important Safety Notice

DO NOT look at the light from the ultraviolet Shoe Sanitizer when it is turned on; it emits UV light which can be harmful to eyes and skin. You can tell when the Shoe Sanitizer light is on because the ON/OFF button will be glowing solid red, and the individual Sanitizer lights will glow blue.

LAMP CONTAINS MERCURY (Hg). The lamp contains a tiny metal ball which you may hear inside the lamp. This is normal; the lamp is not broken. Since mercury is toxic please dispose of the lamp in accordance with local disposal laws. For more information, see: www.lamprecycle.org/res-statelaws.shtml.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.

WARNING: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

Disclaimers

- The UV Shoe Sanitizer destroys up to 99.9% of targeted microorganisms. Targeted microorganisms are those that come in direct unobstructed contact with the UV light.
- The Shoe Sanitizer does not treat medical conditions; it is intended to provide a clean and healthy environment for feet inside shoes.
- The UV light in the Shoe Sanitizer may discolor the interior of the shoe.
- UV light may accelerate the degradation of certain materials.

Troubleshooting

Situation:

If your device doesn't start after one minute delay:

- Check that Sanitizer cords are connected to A/C adapter.
- Check if A/C adapter is plugged in properly and the RED light on the A/C adapter is ON.
- If you checked this step and your device is still not on, please call 1-800-PEDIFIX for technical assistance.

Situation:

If your device doesn't turn off:

- The device will turn off automatically after a 15 minute treatment cycle.
- If the Sanitizer does not turn off, unplug the device, wait two minutes and try again.
- If the problem persists, call 1-800-PEDIFIX for technical assistance.

Situation:

If either of the two Blue Indicator lights blink, the UV bulb is broken or not screwed in fully:

Check bulb. If tight, and the Blue Indicator light still blinks, the UV bulb needs to be replaced.


Installing or Replacing a Lamp

Using a soft dry cloth, hold the front of the UV lamp. Remove the existing bulb. Replace with a new bulb every two years or when the existing bulb no longer works. Repeat this step for the second Shoe Sanitizer if necessary.

If the problem remains with the lamp, e.g. the right shoe sanitizer was not working and after switching the lamps the left shoe sanitizer is not working, then the lamp needs to be replaced.

NOTE: After turning the shoe sanitizer on, it will take approximately 60 seconds for this error indicator to occur. This is the time needed to ignite the UVC lamp.

Replacement Bulbs and Bags

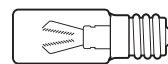

Replacement Bulb:
Order Part Number P3411

Replacement Bag:
Order Part Number P3412

