

# DynaSorb™

## Super Absorbent Dressing

### Non-Adherent




#### Description

DynaSorb® Super Absorbent Dressing, Non-Adherent is a sterile dressing for use on heavily exuding wounds, consisting of an air permeable and waterproof nonwoven layer, a super absorbent pad layer and a hydro-penetrating nonwoven layer. The triple-layered construction facilitates fluid management to provide the optimal moist wound environment which leads to the promotion of faster wound closure. The absorbent pad provides superior absorption and locks exudates away. This minimizes the risk of maceration and damage to peri-wound skin.

#### Indications

DynaSorb® Super Absorbent Dressing, Non-Adherent is indicated for management of moderate to high exuding wounds, including: leg ulcers, pressure ulcers, diabetic foot ulcers and dehisced surgical wounds.

#### Contraindications

1. Do not use this product on dry or low exuding wounds, eyes, mucous membranes or in wound cavities because the dressing swells during absorption. Do not use on third-degree burns.
2. DynaSorb® Super Absorbent Dressing, Non-Adherent should not be used on persons with a known allergy or hypersensitivity to the dressing or its components.

#### Instructions for Use

1. After wound is cleaned and sterilized, the appropriate size dressing is selected to allow for sufficient space to the wound edge.
2. DynaSorb® Super Absorbent Dressing, Non-Adherent can be used both as a primary dressing or as a secondary dressing. When used as a primary dressing, DynaSorb® Super Absorbent Dressing, Non-Adherent must be in close contact with the wound bed to ensure wound fluid is absorbed. Secure accordingly with bandage, tapes or compression.
3. The frequency of changing dressing depends on the amount of exudates.
4. Dressing removal can be facilitated by moistening the dressing with saline solution.
5. Dispose of as normal waste. If contaminated, follow local regulations.

#### Warnings

Do not cut the dressing, the cover must remain intact.

#### Shelf Life

Dressings expires 2 years from date of manufacture.

#### Specifications

| Reorder No. | Size                          | Dressing Core                       |
|-------------|-------------------------------|-------------------------------------|
| 3087        | 2 in. × 2 in. (5 cm × 5 cm)   | 3 in. × 3 in. (7.5 cm × 7.5 cm)     |
| 3088        | 4 in. × 4 in. (10 cm × 10 cm) | 4.72 in. × 4.72 in. (12 cm × 12 cm) |

**Manufactured for:**  
**Dynarex Corporation**  
**10 Glenshaw Street**  
**Orangeburg, NY 10962**  
**USA • www.dynarex.com**

Made in China



**SYMBOL GLOSSARY**  
 For an explanation of symbols used in Dynarex packaging, visit [dynarex.com/symbols.php](http://dynarex.com/symbols.php)

R190326

Copyright © 2019 Dynarex Corporation. All rights reserved.