

### **INSTRUCTIONS FOR USE:**

1. Wash your hands.
2. Put on face mask.
3. Put on gloves.
4. Remove old dressing.
5. Carefully open the alcohol swabstick package. Apply to area around catheter using circular motion going from the catheter out. **DO NOT GO OVER ONE AREA TWICE WITH THE SAME SWABSTICK!** Repeat with remaining swabsticks.
6. Open PVP iodophor swabstick package. Apply as #5 above. Repeat with all three swabsticks.
7. Allow PVP iodophor to dry (blot with sterile 4" × 4" if necessary).
8. The transparent dressing may be applied directly over the insertion site. For cosmetic reasons the dressing may be covered by a gauze pad. If desirable, a small compact dressing may be applied under the transparent dressing (use the dressings provided as you wish.)
9. Apply the transparent dressing as instructed.