

Re-Order Number	Depth	Units
703115	3"	Each
703116	3.75"	Each
703117	4.75"	Each



Original  
**Skil-Care™**  
CORPORATION  
Design



### **Purpose**

Laterally stabilized the resident's head and places it in a position that promotes forward eye contact. This headrest is designed and universally sized to fit all Skil-Care Reclining Wheelchair Backrests as well as extended wheelchair and geri-chair backrests made by other manufacturers.

### **Easy to Use**

Position headrest on wheelchair or geri-chair backrest and bring straps behind backrest. secure buckle and tighten strap to prevent headrest from slipping.

### **Easy Care**

The cover removes for easy laundering. Do not wash or dry the cover at temperature in excess 180°F (82°C). Do not use bleach. Do not launder foam.