HOW TO ASSEMBLE YOUR




Item # 8801 Capacity not to exceed 250 lb








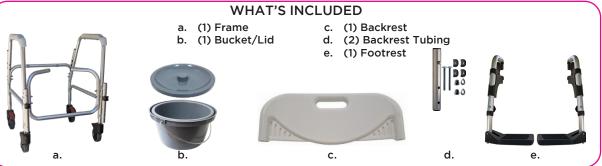

Attach the two (2) backrest tubings (d) to the backrest (c) by guiding the tubing through the holes on each side of the plastic backrest (see Figure 1). Ensure each push button is fully engaged after hearing it "click" into place.

Attach the two (2) backrest tubings to the rear of the frame by guiding the tubing through the holes on each side of the plastic backrest (see Figure 1). Ensure each push button is fully engaged after hearing it "click" into place.

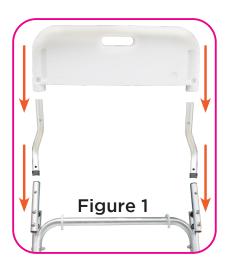

Attaching Footrest

First determine the left and right side by folding down the footplate and placing them in front of the seat. The footplates should be facing each other and directly in front of the seat.

To put on footrests: Place footrest onto the designated brackets towards the side (90 degrees from the front of the chair), swing the footrest towards the front until you hear it click into place (see Figure 2). Repeat on other side.

To take off footrests: Push the small black lever and allow the footrests to swing out 90 degrees, grab the top of the footrest and lift directly upwards to remove.











Figure 2

Turn the knob couterclockwise and remove with the washer and screw from the footrest. Slide the footplate up or down and align the holes together (see Figure 3).



Figure 3

Once you have determined the correct height, insert the screw into the hex screw cut out on the front of the footrest. Ensure that the hex screw head fits into the hex cut-out on the frame. Insert the washer and the knob. Turn the knob clockwise and fully tighten (see Figure 4).



Figure 4

- Before using the chair, lock all four (4) wheels by depressing the hanging end of the red lever above the wheel. Once the user is seated and stable, unlock the wheels by depressing the raised end of then red lever, and roll the commode to the desired location. (see Figure 5). NOTE: Be especially careful and do not move too quickly over any threshold, mat/carpet, or other uneven surfaces to avoid tipping the user forward.
 - 1. Lock the wheels before the user seated in the chair attempts to stand up.
 - 2. After use, remove the bucket, discard waste and sanitize.

LOCK UNLOCK

Figure 5

Removing Bucket



Figure 6

Slide the bucket in or out from the rear of the commode (see Figure 6).



Figure 7

Lift the seat up all the way (see Figure 7).



Figure 8

Grab the handle and lift the bucket to remove (see Figure 8).

WARNING

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. NOVA Medical Products shall have no liability for any damage or injury resulting from improper assembly or misuse of this product.

- Make sure the backrest is secure (push button locked completely into hole) before using
- DO NOT use if chair is damaged. Check frequently for cracks, damage, or looseness.
- Use on a level surface
- DO NOT stand on chair
- This product is rated for persons weighing up to **250 lb** when properly assembled.

HOW TO USE

Before sitting, lock the wheels by pushing down the red lever on all 4 wheels. When the user seated is stable, unlock the wheels by popping up the red lever and roll the commode to the desired location - be especially careful over any threshold, mat, or uneven surface to not tip the user forward by moving too guickly. When you have reached your location, lock the wheels again before the user attempts to stand up. Bucket with handle and lid are included and easily slides in and out from the rear of the seat or lift the seat up and lift the bucket out...

CARE AND MAINTENANCE

Use a mild detergent and warm water to clean. Dry with a soft cloth to avoid moisture build-up.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame Limited Lifetime Warranty Non-Metal/Metal parts One Year Limited Warranty

(Rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse. modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.

CONTACT US! We're here to help



info@novajoy.com 1800 557 6682

novaiov.com



