



ShoeLift™

PATIENT APPLICATION

BEFORE USING THIS DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USE/INDICATIONS:

Intended to compensate for leg length changes while wearing a fracture walking brace. Recommended for use on a securely fastened closed toe shoe or sneaker.

CONTRAINDICATIONS: None

WARNINGS AND PRECAUTIONS:

Always wear a securely fastened shoe. Remove the ShoeLift™ before operating a motorized vehicle. Always exercise caution and care when using the ShoeLift. Improper use or installation of the ShoeLift could result in injury. If you experience any pain, swelling, sensation changes, or any unusual reactions while using this product, consult your medical professional immediately.

APPLICATION INFORMATION:

PREPARATION

- A) Select the appropriate size.

SHOELIFT APPLICATION

- 1) Unfasten and loosen the top strap.
- 2) While seated, place your uninjured foot over your opposite knee. Place the front of the ShoeLift™ cage over the toe of your shoe.
- 3) Now pull the elastic cage toward the heel of the shoe and adjust the side straps of the cage and secure over the sole of the shoe. Pull the heel tab up over the heel of the shoe.
- 4) Fasten the hook and loop strap securely over the top of shoe with the ProCare® name facing out.

CLEANING INSTRUCTIONS:






Hand wash in cold water using mild soap, or wipe with a damp cloth. Air dry.

**NOT MADE WITH NATURAL RUBBER LATEX
SINGLE PATIENT USE ONLY**

WARRANTY:

DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THIS DEVICE IS ONLY ONE ELEMENT IN THE OVERALL TREATMENT PROGRAM ADMINISTERED BY A MEDICAL PROFESSIONAL. THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

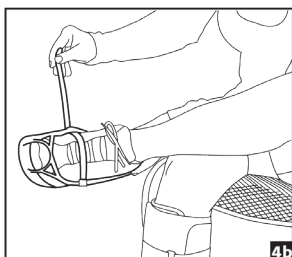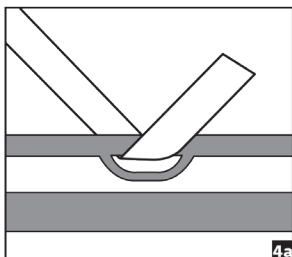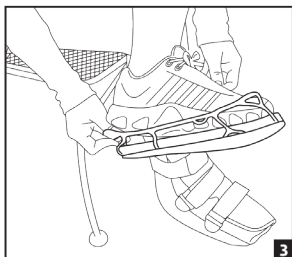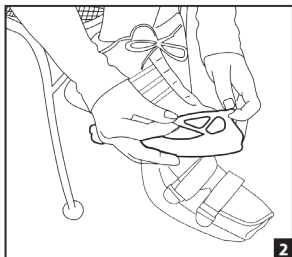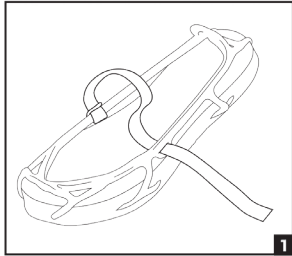

DJO, LLC
 T 800.336.6569 D 760.727.1280 F 800.936.6569
 1430 Decision Street | Vista, CA 92081 | U.S.A.
DJOglobal.com