

Wood Finish Care and Cleaning Options

Hand Applied Lacquered Finish and Oil Stains

Wood Products and Furniture with Lacquer Finish - Care and Cleaning Tips

The type of finish on the wood, not the type of wood, determines how the custom wood furniture care and cleaning can be done. Excessive dampness, dryness, heat, or cold can damage wood furniture. Sunlight can change the color. Rubber or plastic mats should not be used unless marked safe for wood, since some may soften or stain the finish. Wipe up spills at once to prevent spots that require refinishing.

Wood Product Cleaning

Vacuuming with a dusting brush attachment gently removes dust from surfaces, preventing buildup. If no vacuum cleaner, use a clean soft cloth, turning it often, or soft paper towels to pick up dust. Dust furniture before vacuuming floors. If the finish is water resistant, a barely dampened towel or cloth will pick up dust.

Clean with Mild Soap

The best way to clean lacquer is to use a mild soap and warm water. A mix of one teaspoon of soap to a half gallon of water is sufficient. Wring your cloth out so it's only damp, not dripping wet. Wipe the lacquer down avoiding circular motions. Once it's cleaned, you want to take a soft dry cloth and wipe any excess water away. Remember water can be death to lacquer finishes if left alone. You also need to make sure the cloth has no foreign material that can scratch the surface. Using harsh household cleaners will pull the finish off as well. Avoid using any cleaner that has bleach or ammonia in it.

Guide to Wood Lacquer Finishes

Regularly dust with a soft, dry cloth. Never use oiled or treated cloths on lacquered finishes. Some finishes can be wiped with a damp (not wet) cloth, followed at once by rubbing with a dry cloth, (test first in inconspicuous spot) to remove fingerprints and smudges. Apply liquid wax on occasion to maintain a closs.

When severe or potentially damaging soiling occurs, use a solvent base furniture cleaner with care. Apply with soft cloth in one hand, and wipe at once with soft cloth in other hand, doing only a small area at a time. An oil soap (Murphy's) may be satisfactory. Protect with liquid wax or polish to maintain a gloss.

Disinfectants

If required by patient use, the wood products may be cleaned with Cavicide or Protex wipes. Do not let liquid stand on the surface, and dry after use. SEE NEXT PAGE.

CAUTION: Stools & Steps With Anti-Slip Treads

Steps and Stools with Black Anti-Slip Treads may ONLY be cleaned using a stiff synthetic or nylon bristle brush (non-metallic) and soapy water, never use solvents. A basic nylon brush is perfect. Nylon bristles will provide effective cleaning force, removing dirt from in between the peaks while being tough enough to withstand the abrasive surface.



Coronavirus Disease 2019 (COVID-19) and Metrex Surface Disinfectants

March 13, 2020

Protocols for Disinfection Efficacy on COVID-19

As of the date of this writing, there is currently no EPA recognized test protocol to evaluate disinfection efficacy against this specific novel coronavirus virus strain, SARS-CoV-2 (Severe Acute Respiratory Syndrome Coronavirus 2), which causes COVID-19 (Coronavirus Disease 2019). Therefore, there is no EPA-registered surface disinfectant that bares a label claim against SARS-CoV-2 as of the date of this writing.

All coronavirus strains are enveloped viral particles that belong to the same virus family of *Coronaviridae*. Enveloped viral particles are typically more susceptible to chemical disinfectant formulations than are other pathogens¹.

CDC's interim recommendations for infection prevention and control now state that the use of EPA-registered, hospital-grade disinfectants are appropriate for SARS-CoV-2 in the healthcare settings.² The following Metrex products meet this guideline:

- o CaviWipes
- o CaviCide
- CaviWipes1
- o CaviCide1
- o CaviWipes Bleach
- CaviCide Bleach

The CDC also refers to the EPA's List N that lists EPA-registered disinfectants products with Emerging Viral Pathogens and Human Coronavirus claims for use against SARS-CoV-2. This list is actively being updated by the EPA to include additional disinfectants that meet EPA's criteria.³

Refer to any updates on the EPA's List N here:

www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2

The following Metrex products have been included onto the EPA's List N:³

- CaviCide⁸
- CaviWipes1⁴
- CaviCide1⁵
- CaviWipes Bleach⁶
- CaviCide Bleach⁷



The table below gives an overview of the information discussed above:

	Emerging Viral Pathogen	EPA-registered, Hospital-	
	Claim / Contact Time	Grade Disinfectant /	
		Contact Time	
CaviWipes	-	YES / 3 minutes	
CaviCide	-	YES / 3 minutes	
CaviWipes1	YES / 3 minutes	YES / 1 minute	
CaviCide1	YES / 3 minutes	YES / 1 minute	
CaviWipes	YES / 3 minutes	YES / 3 minutes	
Bleach			
CaviCide Bleach	YES / 3 minutes	YES / 3 minutes	

REFERENCES:

- Sattar, S. "Hierarchy of Susceptibility of Viruses to Environmental Surface Disinfectants: A Predictor of Activity
 Against new and Emerging Viral Pathogens". Journal of AOAC International. 2007. Vol 90.6.
 https://www.researchgate.net/publication/5657319 Hierarchy of Susceptibility of Viruses to Environmental Surface Disinfectants A Predictor of Activity Against New and Emerging Viral Pathogens. Accessed 3.3.2020.
- Interim Infection Prevention and Control Recommendations for Patients with Confirmed Coronavirus Disease 2019
 (COVID-19) or Persons Under Investigation for COVID-19 in Healthcare Settings.
 https://www.cdc.gov/coronavirus/2019-ncov/infection-control/control-recommendations.html?CDC_AA_refVal=https%3A%2F%2Fwww.cdc.gov%2Fcoronavirus%2F2019-ncov%2Fhcp%2Finfection-control.html. Accessed 3.11.2020.
- 3. List N: Disinfectants for Use Against SARS-CoV-2. https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2. Accessed 3.11.2020
- 4. CaviWipes1 Master Label from retrieved from EPA's Pesticide Product and Label System. https://iaspub.epa.gov/apex/pesticides/f?p=PPLS:8:::NO::P8 PUID,P8 RINUM:504032,46781-13. Accessed 3.3.2020.
- CaviCide1 Master Label from retrieved from EPA's Pesticide Product and Label System. https://iaspub.epa.gov/apex/pesticides/f?p=PPLS:8:::NO::P8_PUID,P8_RINUM:503807,46781-12. Accessed 3.3.2020.
- 6. 7. CaviWipes Bleach Master Label from retrieved from EPA's Pesticide Product and Label System. https://iaspub.epa.gov/apex/pesticides/f?p=PPLS:8:::NO::P8 PUID,P8 RINUM:521508,46781-14. Accessed 3.3.2020.
- CaviCide Bleach Master Label from retrieved from EPA's Pesticide Product and Label System.
 https://iaspub.epa.gov/apex/pesticides/f?p=PPLS:8:::NO::P8 PUID,P8 RINUM:522158,46781-15
 Accessed 3.3.2020.
- 8. CaviCide Master Label from retrieved from EPA's Pesticide Product and Label System. https://iaspub.epa.gov/apex/pesticides/f?p=PPLS:8:::NO::P8 PUID,P8 RINUM:27348,46781-6. Accessed 3.3.2020.
- 9. Xu et. al., <u>Systematic Comparison of Two Animal-to-Human Transmitted Human Coronaviruses: SARS-CoV-2 and SARS-CoV</u>, *Viruses* **2020**, 12, 244, doi:10.3390/v12020244.

CaviCide™ & CaviWipes™

The brand you've trusted for over 25 years has just added a new member to its family.
Introducing... CaviWipes 220ct.
Same great brand, even better value.



The CaviWipes & CaviCide family of products provides time savings, convenient packaging, ease of use, and intermediate-level disinfection in various sizes. CaviWipes & CaviCide are recommended for use on hard, non-porous surfaces in healthcare settings, and doing so may help alleviate your important surface disinfection concerns.

Plus.. Now, you don't have to change out the canister as often with the all new CaviWipes 220 count.



The Convenience and Strength of CaviCide & CaviWipes

Fast Action. Kills TB in 3 minutes and MRSA, HIV-1, HBV and HCV in 2 minutes.

Durable. Nonwoven towelettes will not bunch up during use.

Accessibility. The deep well lid and easy access tab provides quick opening and tight closures between wipes.


Saves Time and Inventory. All-in-one cleaner and disinfectant.

Value. Now with even more wipes per canister with the 220 count!



Ordering Information

Part No.	Product Description Qu	antity
10-1090	CaviWipes - 220 wipes per canister NEW	12 canisters/ case
13-1100	CaviWipes - 160 wipes per canister	12 canisters/ case
13-1150	CaviWipesXL - 65 wipes per canister	12 canisters/ case
13-1155	CaviWipesXL Singles - 50 wipes per box	6 boxes/ case
13-1224	CaviWipes in a Flat Pack – 45 wipes per pack	20 packs/ case
13-1175	CaviWipes canister wall bracket	12 brackets/ case
13-1185	Flat Pack Wall Mount	20 mounts/ case

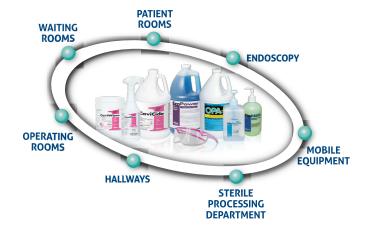

Circle of Care™

The seven critical touch points in Infection Prevention's Circle of Care represent the most critical areas in the healthcare environment in preventing healthcare-associated infections (HAIs). CaviWipes covers every critical touch point. The diversity of uses and benefits makes this product uniquely favorable in the healthcare sector.





USE PROTEX[™] DISINFECTANT SPRAY

- As a disinfectant on hard, nonporous surfaces.
- For disinfecting of ultrasound transducers, probes, mammography compressor plates and other hard nonporous surfaces

Protex Disinfectant Spray is a one-step hospital use disinfectant, bactericidal according to the current AOAC Use-Dilution Test Method, Fungicial according to The AOAC Fungicidal Test and Virucidal* according to the Virucidal qualification modified in the presence of 5% organic serum against:

(Bacteria) Acinetobacter baumannii • Brevibacterium ammoniagenes • Campylobacter jejuni • Community Associated Methicillin-Resistant Staphylococcus aureus—CA-MRSA • Enterobacter aerogenes • Enterococcus faecalis—VRE • ESBL Escherichia coli • Klebsiella pneumoniae • Legionella pneumophilia • Pseudomonas • Salmonella • Serratia marcescens • Shigella dysenteriae • Staphylococcus aureus—MRSA • Staphylococcus aureus Multi-Drug Resistant • Staphylococcusaureus—VISA • Streptococcus pyogenes • Vibrio cholerae

(Viruses) Hepatitis B Virus (HBV)* • Hepatitis C Virus (HCV)* • Herpes Simplex Virus Type 1* • Herpes Simplex Virus Type 2* • HIV1 (AIDS Virus)* • Human Coronavirus* • Influenza Type A / Brazil* • Norwalk Virus* • Respiratory Syncytial Virus* • Rotavirus* • SARS Associated Coronavirus* • Vaccinia*

(Fungi) Trichophyton mentagrophytes • Aspergillus niger

USE IN: USE ON:

Hospitals
 Non- critical medical device surfaces

 Medical Related Facilities
 Nursing Homes
 Non- critical medical/hospital equipment surfaces, walls, counters, metal and stainless

Athletic FacilitiesExercise FacilitiesUltrasonic bathsDefribrillators

For fungicidal activity: Protex Disinfectant Spray is an effective one-step fungicide against Trichophyton mentagrophytes when used on surfaces in areas such as locker rooms, dressing rooms, shower and bath areas and exercise facilities.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

KILLS HIV-1 AND HBV AND HCV ON PRE-CLEANED ENVIROMENTAL SURFACES / OBJECTS PREVIOUSLY SOILED WITH BLOOD / BODY FLUIDS

in health care settings or other settings where there is an expected likelihood of soiling or inanimate surfaces / objects with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS) or Hepatitis B Virus (HBV) or Hepatitis C Virus (HVC).



Just as **TOUGH** as ever. Only *FASTER*.



Eliminates 99.9% of bacteria in 15 seconds. Disinfects in just 4 minutes.

- Effective on 40 different pathogens
- Starts killing on contact
- Cleans AND disinfects in one easy step
- Will not harm most hard, nonporous surfaces
- STAY-PUT adhesive backing adheres to mobile equipment for ULTRA convenience
- Alcohol, bleach, and solvent-FREE





ISO 13485:2003

Protex ULTRA not available for sale in Europe or Canada



Parker Laboratories, Inc.

The sound choice in patient care.™

PROTEX™ ULTRA DISINFECTANT WIPES

A powerful, one-step cleaner/disinfectant effective against a broad spectrum of pathogens, including HIV, H1N1, MRSA and many others. Useful in disinfecting ultrasound transducers, mammography compressor plates, mobile equipment, vinyl exam tables, countertops and other hard, nonporous, non-critical surfaces.

FEATURES

- Effective on 40 different pathogens
- Starts killing on contact
- STAY-PUT adhesive backing adheres to mobile equipment for ULTRA convenience
- Alcohol, bleach and solvent-FREE
- Will not harm most hard surfaces
- Cleans AND disinfects in one easy step

PACKAGE SIZE	UNITS/CASE
60 ct Softpack	2 bx/case
6.5" x 6.0" non-abrasive wipes	
	60 ct Softpack

