

BAILEY

ASSEMBLY INSTRUCTIONS MODEL 310 CHILD/MODEL 311 ADULT INDIVIDUAL STANDING TABLES

Step 1. Layout base (A) with tee nuts on bottom of base.

Step 2. Place cabinet (B) on base (A) using #1200 $\frac{1}{4}$ -20 X $1\frac{1}{2}$ truss hd machine screw (4) each to secure cabinet to base. TIGHTEN DOWN

Step 3. For shipping purposes, one angle is reversed. Remove angle and re-install on outside of cabinet to match existing four angles.

Place tray (C) on top of cabinet (B)

Match angles on cabinet to pre-drilled holes on tray and secure both using #3290 $\frac{3}{4}$ -10 slotted hex washer hd (5) places. TIGHTEN DOWN.

Make sure all hardware and components are secure before placing unit into service.

