# **Assembly Guide**

for the

# IN SHUTTLE BALANCE



If you discover damaged or missing parts, please contact Shuttle Systems support@shuttlesystems.com | 1-800-334-5633 9am-5pm Pacific Time

### **Important Note:**

### PDF versions of these instructions are available at: http://shuttlesystems.com/pages/faq

Keep in mind that the following images pertain to just one of the four available Shuttle Balance models, your parts may appear slightly different than those in the included images but, the required steps remain the same.

If there is any shipping damage please take a photo of the damage and email to **support@shuttlesystems.com** or call **(800)334-5633**.

### **BALANCE ASSEMBLY**

The Balance will come in one box
\* Balance Sport will come in 2 boxes

#### 1. Shuttle Balance

- Two Handrail Tubes
- Two Bottom Frame Tubes
- Wobble Board
- Four Knurled Knobs & Four Steel/Rubber Washers
- Two Steps

#### STEP 1

#### ASSEMBLING THE HANDRAIL TUBES TO THE BOTTOM FRAME TUBES

ITEMS REQUIRED

HANDRAILS TUBES

BOTTOM FRAME TUBES

- 1. Remove all protective packaging from the Handrail Tubes.
- 2. Connect both Handrail Tubes to one side of the Bottom Frame Tubes, aligning the holes. Lift both Handrail Tubes and attach the last remaining Bottom Frame piece. (Pic 1.1 & 1.2)



1.1



1.2

STEP 2	ATTACHING THE WOBBLE BOARD
ITEMS REQUIRED WOBBLE BOARD KNURLED KNOBS BAG	<ol> <li>Place the Wobble Board in the middle of the Frame. Remove the zip-tie and parts bag with four knurled knobs and four steel/rubber washers from the Wobble Board. (Pic 2.1)</li> <li>Detach four chain carabiners (two on each side) from the eyebolts on the Wobble Board and attach them to the eyebolts on the Handrails. (Pic 2.2)</li> <li>Remove each carabiner attached to the black control rope from the eyebolts on the Wobble Board. Attach the top carabiner of the control ropes to the corresponding eyebolt located on the side of the Handrail Frame. Insert the eyebolt into and through the middle of the frame. Tighten down the eyebolt with a Steel/Rubber Washer and Knurled Knob. Attach the last remaining carabiner on the black control ropes to the bottom eyebolt of the frame. (Pic 2.3 &amp; 2.4)</li> <li>Detach the red elasticord carabiners from each other. Attach them to the bottom eyebolts on the frame. The elasticords and control ropes should both be attached to the bottom eyebolt of the frame. Remove the protective packaging from the Wobble Board. (Pic 2.5)</li> </ol>
	I

Pictures are on the next page





2.1








2.5

#### **SECURING THE STEMPS TO THE FRAME**

ITEMS REQUIRED

**STEPS** 

1. Align the velcro on the bottom frame with the velcro on the bottom of the steps. Press the step down to secure to the frame. The Shuttle Logo should be facing out from the Wobble Board. (Pic 3.1)

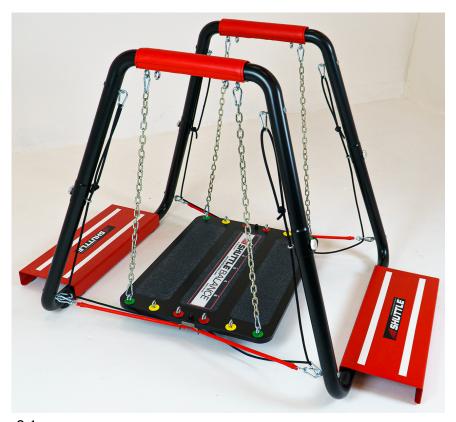

3.1

Finally, you have one of the greatest machines on earth....

Well, at least we'd like to think so - and many of our owners agree.

Therefore, enjoy it!

Leave us a review on our website!

Please visit us at: www.ShuttleSystems.com 1-800-334-5633



Made in the

**USA** 

# Important Information 5 YEAR LIMITED ORIGINAL EQUIPMENT WARRANTY

#### REPLACEMENT PARTS

Replacement parts for SHUTTLE SYSTEMS cardio-muscular conditioning equipment are available directly from SHUTTLE SYSTEMS. To place an order, call 1-800-334-5633. Please have the following information ready: Model name and number, serial number, shipping address, and authorized payment information.

#### REPLACEMENT PART WARRANTY

SHUTTLE SYSTEMS/Contemporary Design Company makes every effort to assure that operating parts meet high quality and durability standards and warrants to the original retail consumer/purchaser of our parts that each such part(s) be free from defects in materials and workmanship for a period of two years from the date of parts purchase. Wear-parts (elasticords, rebound elastics, lock knobs, and torque handles) are warranted for one year.

#### **PROOF OF PURCHASE**

Please retain your dated sales receipt as proof of purchase to validate the warranty period. SHUTTLE SYSTEMS may require reasonable proof of purchase and we suggest you keep your invoice.

#### LIMITED ORIGINAL EQUIPMENT WARRANTY

SHUTTLE SYSTEMS makes every effort to assure that its products meet high quality and durability standards and warrants to the original retail consumer/purchaser of our products that each product be free from defects in workmanship and materials under normal and reasonable use and correct assembly (if assembled by consumer/purchaser), as follows. Warranty does not apply to defects due directly or indirectly to misuse, abuse, negligence or accidents, repairs or alterations outside our facilities or to a lack of maintenance.

- 1. **LIMITED LIFETIME WARRANTY:** Bolted metal frames, stands, towers, kickplate frame and board, and carriage frame.
- 2. For a period of FIVE YEARS: Pulley systems, carriage handles, and wheels.
- 3. For a period of TWO YEARS: Upholstery, headrest, kickplate face, harnesses, & grips.
- 4. For a period of ONE YEAR: Elasticords, rebound elastics, lock knobs, torque handles, and all other parts.

SHUTTLE SYSTEMS LIMITS ALL IMPLIED WARRANTIES TO THE PERIOD SPECIFIED ABOVE FROM THE DATE THE PRODUCT WAS PURCHASED AT RETAIL. EXCEPT AS STATED HEREIN, ANY IMPLIED WARRANTIES OR MERCHANTABILITY AND FITNESS ARE EXCLUDED. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG THE IMPLIED WARRANTY LASTS, SO THE ABOVE LIMITATIONS MAY NOT APPLY TO YOU.

SHUTTLE SYSTEMS shall in no event be liable for death, injuries to persons or property or for incidental, contingent, special or consequential damages arising from the use of our products. In administration of this warranty, SHUTTLE SYSTEMS may at its discretion request that the product or part must be returned for examination, postage prepaid, to our Bellingham facility. If such inspection discloses a defect, SHUTTLE SYSTEMS will either repair or replace the product with a comparable replacement. Neither SHUTTLE SYSTEMS dealers nor retail establishments selling this product have any authority to make any warranties or to promise remedies in addition to or inconsistent with those stated above. SHUTTLE SYSTEMS' maximum liability, in any event, shall not exceed the purchase price of the product paid by the original consumer/purchaser. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.