

BD Integra™ Syringe

with retracting BD PrecisionGlide™ Needle

Quick reference guide



1. Attachment

Attach the BD Integra™ needle into the BD Integra™ syringe with a push and a clockwise twist. If pre-attached, tighten the needle onto the syringe prior to use. Pull the shield straight off the needle.



2. Aspiration

Draw up medication in accordance with your institution's protocol.



3. Injection

Administer medication in accordance with your institution's protocol.



4. Activation

Push the plunger rod to activate the safety mechanism. Depress until you sense two "clicks".

Activation can occur before or after needle is withdrawn from the patient.




5. Needle retraction


The needle will disappear into the syringe and the plunger will be recessed into the barrel when activation is complete.

Discard immediately into an approved sharps container after single use.

Luer needle

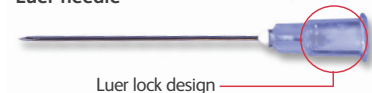

BD Integra™ needle

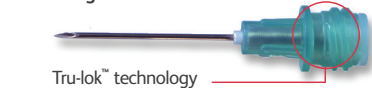

Remember

- If activation is performed **before** needle is withdrawn from patient, do not advance needle while depressing plunger rod.
- If activation is performed **after** needle is withdrawn from patient, minimal splatter may occur. When activating the device outside of the patient, hold the device downward at arm's length and activate while pointing away from self and others. Keep device within sight at all times to avoid accidental needlestick injury.
- **Do not** use this product to draw blood or for venipuncture.
- BD Integra™ needle and syringe **do not** have a standard luer connection. Conventional luer lock needles cannot be used with the BD Integra™ syringe.
- When delivering highly viscous medications, use a 22G or 21G BD Integra™ retracting needle, and inject at a controlled rate.
- Follow your institution's protocol for aseptic technique and skin preparation.
- Follow your institution's policy for safe disposal of all medical waste.