



Cardinal Health™ Monoject™ Magellan™ Insulin and Tuberculin Safety Syringes

Instructions for use



1 Select the appropriate Monoject™ Magellan™ Insulin or Tuberculin Safety Syringe for your procedure.



2 Open package by peeling back paper tabs and remove syringe.



3 Remove protective needle sheath.



4 Draw up medication and administer injection using an aseptic technique in accordance with your facility's protocol.



5 Lock the safety shield using your thumb



or your finger.



6 Verify locked position through audible and tactile "click." Locked position will completely encase needle.



7 Once the shield is locked, immediately dispose of syringe in an approved sharps disposal container.



For more information, visit cardinalhealth.com/monoject, scan the QR code to view product guides or call customer service at **1.800.964.5227**