

# HURRISEAL® SWABS

## DENTIN DESENSITIZER

• SIMPLE TO USE

• CLINICALLY PROVEN

• UNCONDITIONALLY GUARANTEED



### HurriSeal® Dentin Desensitizer

is a simple, one-step, chairside procedure for treating and preventing dentinal hypersensitivity. HurriSeal's unique HEMA, Sodium Fluoride and Benzalkonium Chloride formula acts in seconds to seal dentinal tubules.

#### TREAT SENSITIVITY

HurriSeal Snap -n- Go™ Swabs are perfect for use to treat cervical sensitivity and provide immediate relief lasting up to 9 months.

#### PREVENT SENSITIVITY

To prevent post-procedure discomfort associated with restorative procedures, we recommend HurriSeal in multi-dose 12 mL bottles. Just a quick application after etching is all you need to prevent sensitivity for the lifetime of the restoration.

#### EFFECTIVE

HurriSeal offers chairside relief with only a quick 30-second application.

#### EASY TO USE

No mixing or light-curing. Just a quick and simple chairside application with instant results.

#### RESTORATIONS

Not only does HurriSeal act as a wetting agent, but it also increases the bond strength under restorations.\*

#### SOFT TISSUE FRIENDLY

Unlike products containing glutaraldehyde, HurriSeal is gentle to soft tissue and does not cause gingival irritation when used as directed.

### ORDERING INFORMATION

Box of 2 Snap -n- Go Swabs ..... 0283-0697-11  
Box of 36 Snap -n- Go Swabs ..... 0283-0697-47  
Multi-dose 12 mL bottle ..... 0283-0697-82

**In-office application of a desensitizing agent is an ADA recognized procedure.**

Simply use insurance code D9910 for reimbursement of HurriSeal application.

**After only a few applications, HurriSeal pays for itself!**

**PLACE YOUR ORDER TODAY! QUESTIONS?** Contact your preferred Dental Supplier or call us at 1-800-238-8542, M-F: 8:00 a.m. – 4:30 p.m. ET. [www.beutlich.com](http://www.beutlich.com).

\*As reported in Chairside by: Karl F. Leinfelder, DDS, MS, and Ross W. Nash, DDS. "Solutions for Postoperative Sensitivity." Compendium of Continuing Dental Education, May 1999, pp. 1-8


HurriSeal is a registered trademark of Beutlich® Pharmaceuticals, LLC. HSS 616 0418

**Beutlich**  
PHARMACEUTICALS LLC  
Since 1954

Beautiful Results Without the Sensitivity

# HURRISEAL®

## DENTIN DESENSITIZER

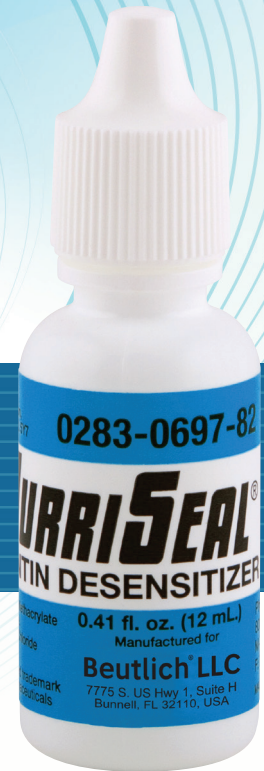

### HurriSeal® Dentin Desensitizer

is a simple, one-step, chairside procedure for treating and preventing dentinal hypersensitivity. HurriSeal's unique HEMA, Sodium Fluoride and Benzalkonium Chloride formula acts in seconds to seal dentinal tubules.

#### TREAT SENSITIVITY

Treat cervical sensitivity and provide immediate relief lasting up to 9 months.

#### PREVENT SENSITIVITY

Prevent post-procedure discomfort associated with restorative procedures. Just a quick application after etching is all you need to prevent sensitivity for the lifetime of the restoration.

#### RESTORATIONS

Not only does HurriSeal act as a wetting agent, but it also increases the bond strength under restorations.\*

#### EFFECTIVE

HurriSeal offers chairside relief with only a quick 30-second application.

#### EASY TO USE

No mixing or light-curing. Just a quick and simple chairside application with instant results.

#### SOFT TISSUE FRIENDLY

Unlike products containing glutaraldehyde, HurriSeal is gentle to soft tissue and does not cause gingival irritation when used as directed.

### In-office application of a desensitizing agent is an ADA recognized procedure.

Simply use insurance code D9910 for reimbursement of HurriSeal application.

**After only a few applications, HurriSeal pays for itself!**



**PLACE YOUR ORDER TODAY! QUESTIONS?** Contact us at 1-800-238-8542, M-F: 8:00 a.m. - 4:30 p.m. ET. [www.beutlich.com](http://www.beutlich.com)

\*As reported in Chairside by: Karl F. Leinfelder, DDS, MS, and Ross W. Nash, DDS. "Solutions for Postoperative Sensitivity," Compendium of Continuing Dental Education, May 1999, pp. 1-8

HurriSeal is a registered trademark of Beutlich® Pharmaceuticals, LLC.

• SIMPLE TO USE

• CLINICALLY PROVEN

• UNCONDITIONALLY GUARANTEED