Moving  
Rehabilitation  
Forward™

# Hydrocollator®

## HEATING UNITS

### User Manual

120 - 220/240 Volt Units

ENGLISH  
DEUTSCH  
FRANÇAIS  
ESPAÑOL  
ITALIANO  
PORTUGUÊS  
NEDERLANDS  
DANSK  
ČESKY  
SVENSKA

# ENGLISH

Foreword ..... 2  
Precautionary Definitions..... 2  
Safety Precautions..... 3-4  
Nomenclature..... 5-7  
Technical Specifications ..... 8-10  
Setup ..... 11  
Operation ..... 11  
Troubleshooting ..... 12  
Accessories ..... 13  
Maintenance ..... 14-15  
Warranty ..... 16  
Replacement Parts ..... I-XII

Hydrocollator® is a registered trademark of DJO, LLC

HotPac™ is a trademark of DJO, LLC

Comet® of Prestige Brands, Inc.

Soft Scrub® of the Dial Corporation

Scotch-Brite™ of 3M

©2009 DJO, LLC Vista, California, USA. Any use of editorial, pictorial, or layout composition of this publication without express written consent from DJO, LLC is strictly prohibited. This publication was written, illustrated, and prepared for distribution by DJO, LLC.

## FOREWORD

Hydrocollator® Heating Units

This manual contains general instructions for operation, precautionary instructions, and maintenance recommendations for the owners and operators of the Hydrocollator SS-2, M-2, SS, E-2, E-1, and M-4 Heating Units. In order to obtain maximum life and efficiency of the Hydrocollator Heating Units, please read this manual thoroughly and follow all instructions prior to operating the unit.

Specifications put forth in this manual were in effect at the time of publication. However, owing to DJO, LLC's policy of continual improvement, changes to these specifications may be made at any time without obligation on the part of DJO, LLC.

## PRECAUTIONARY DEFINITIONS

The precautionary instructions found in this section and throughout this manual are indicated by specific symbols. Understand these symbols and their definitions before operating this equipment. The definitions of these symbols are as follows:

= **CAUTIONS** - Text with a "CAUTION" indicator explains possible safety infractions that could have the potential to cause minor to moderate injury or damage to equipment.

= **WARNINGS** - Text with a "WARNING" indicator explains possible safety infractions that will potentially cause serious injury and equipment damage.

= **DANGERS** - Text with a "DANGER" indicator explains possible safety infractions that are imminently hazardous situations that would result in death or serious injury.

**NOTE:** Throughout this manual, "NOTE" may be found. These notes are helpful information to aid in the particular area or function being described.

## CAUTION

- Read, understand, and practice the precautionary and operating instructions. Know the limitations associated with the Hydrocollator SS-2, M-2, SS, E-2, E-1, M-4 Heating Units. Observe the cautionary and operational decals installed on the unit.
- DO NOT operate this unit in an environment where other devices are being used that intentionally radiates electromagnetic energy in an unshielded manner. Portable and mobile RF communications equipment can affect Medical Electrical Equipment.
- The Hydrocollator Heating Unit generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to other devices in the vicinity. However, there is no guarantee that interference will not occur in a particular installation. Harmful interference to other devices can be determined by turning the Hydrocollator Heating Unit on and off. Try to correct the interference using one or more of the following: reorient or relocate the receiving device, increase the separation between the equipment, connect the equipment to an outlet on a different circuit from that which the other device(s) are connected and consult the DJO, LLC Service Department for help.
- Do not use accessories other than those supplied with the Hydrocollator Heating Unit or recommended by DJO, LLC. The safety of other products has not been established, and their use could result in injury to the patient.
- Never adjust the thermostat too high. The thermostat is extremely sensitive and the slightest adjustment will alter the temperature several degrees. The recommended operating temperature is 71° to 74° C (160° F to 165° F). The temperature of the water should be checked with a thermometer after every adjustment, before using the HotPac. Always allow sufficient time for the water temperature to stabilize.
- The Hydrocollator Heating Unit should not be installed above a surface that is susceptible to casual water damage.
- Always keep the water level over the top of the HotPacs to avoid burning or scorching of the packs.
- Never allow the HotPacs to dry after use.
- Check water level daily as it has a natural loss due to evaporation.
- Clean the tank periodically as described in the maintenance portion of this manual.
- Check unit periodically for signs of leaking and contact service if leaks are noted.
- Always replace Hydrocollator HotPacs as soon as they show signs of wear.
- If the unit is to be left unattended for a period of time, unplug the unit, remove the packs, empty the water, and clean the tank.
- Do not move the Hydrocollator while filled with heated water. Tipping over could result in burns to the user or others.
- Always unplug the Hydrocollator from the power source before attempting to empty water from unit.
- Always use factory authorized replacement parts.
- Always destroy and replace HotPacs which are exposed to contamination: hazardous or cleaning materials, bodily fluids, mold, etc.
- This unit should be operated, transported and stored in temperatures between -40° C to +70° C (-40° F to +158° F) and no requirements for atmospheric pressure, with relative humidity ranging 10% to 100%.
- Treatment time should not exceed 30 minutes.
- DO NOT retrieve HotPacs from tank or replace HotPacs in tank with unprotected hand.
- Avoid direct contact with mounting rack in tank.
- Always wrap HotPac with towel or terry cover before handling or applying to patient.
- Constantly monitor HotPac application to ensure that the skin is not becoming too hot.
- Damage to skin can occur from exposure to extreme heat or cold. Note instructions for proper use.
- DO NOT apply over insensitve skin or in the presence of poor circulation.
- Never lay or sit on top of the HotPac.
- DO NOT use HotPac directly over cuts, abrasions or wounds.
- Consult health care professional before using HotPacs on non-communicative individuals.
- Exercise extreme caution when using HotPacs on non-communicative individuals.

## **WARNING**

- Grounding the Hydrocollator SS, SS-2, E-1, E-2, M-2, and M-4 Heating Units: Make certain the unit is electrically grounded by plugging into a hospital grade electrical outlet with a ground terminal receptacle (U-ground outlet). Follow the National Electrical Code.
- The water temperature in the Hydrocollator is approximately 71° C (160° F), and the water scalding temperature is approximately 49° C (120° F).
- Care must be taken when operating the Hydrocollator adjacent to or stacked with other equipment. Potential electromagnetic or other interference could occur to this or other equipment. Try to minimize this interference by not using other equipment in conjunction with it.
- The Hydrocollator Heating Unit should be placed on a smooth, level surface. The surface should support up to the Filled Weight (weight of heating unit filled with water) found in Technical Specifications pages 8-10 of this manual.
- Use only accessories that are specially designed for the Hydrocollator. Do not use accessories manufactured by other companies on the Hydrocollator. DJO, LLC is not responsible for any consequence resulting from using products manufactured by other companies. The use of other accessories or cables may result in increased emissions or decreased immunity of the Hydrocollator.
- Periodically inspect unit for signs of damage and pitting during regular cleaning intervals, as described in maintenance portion of this manual. Contact service if any signs of damage are discovered.
- DO NOT apply HotPac directly to skin.
- Individuals with circulatory problems should consult with a physician before using this product.
- DO NOT sit or lie on the HotPac.
- DO NOT add chlorine additive to water.
- DO NOT allow HotPac to come in contact with hydrocarbons (e.g., Comet®, Soft Scrub®, etc.).

Risk of explosion if used in the presence of flammable anesthetics.

### Explanation of Symbols

Refer to Instruction Manual/Booklet

Type B equipment

Thermostat Intensity Control

## NOMENCLATURE

Hydrocollator® Heating Units

**WHITE LAMP** (*where applicable*), when illuminated, indicates that main power is connected, the switch is on and unit is operating.

**RED LAMP** (*where applicable*), when illuminated, warns of an overheat/low water condition. Action is necessary to correct the condition.

**GREEN LAMP** (*220/240 volt units*), when illuminated, indicates that main power is connected, the switch is on and unit is operating.


**IMPORTANT:** Some models are fitted with a thermal cut-out thermostat which acts as a “non self-resetting” thermal cut-out. In the unlikely event of a fault condition, this device is designed to open the power circuit between  $88^{\circ}\text{C} \pm 8^{\circ}\text{C}$  ( $190^{\circ}\text{F} \pm 14^{\circ}\text{F}$ ). It is not possible to alter its operating range. The operation of the unit may be restored by depressing the red button after the water/element temperatures have been allowed to cool. If the fault condition persists, please contact DJO, LLC or an authorized agent.

# NOMENCLATURE

Hydrocollator® Heating Units

## Diagram

- |   |                               |
|---|-------------------------------|
| 1. Heating Element                                  | 9. Junction Box (J-Box) Cover |
| 2. Drain  | 10. IEC connector             |
| 3. Overtemp Thermostat                              | 11. Power Cord                |
| 4. Thermostat, Hydraulic                            | 12. Drain Valve (Not Shown)   |
| 5. Bracket (Bracket & Bulb Thermostat A/B Assembly) | 13. Reset Overtemp Button*    |
| 6. Lid  | 14. 6.3 Amp Fuses             |
| 7. 3-inch Caster                                    | 15. Pilot Lamp, Red           |
| 8. Overtemp Light                                   | 16. Pilot Lamp, Clear         |

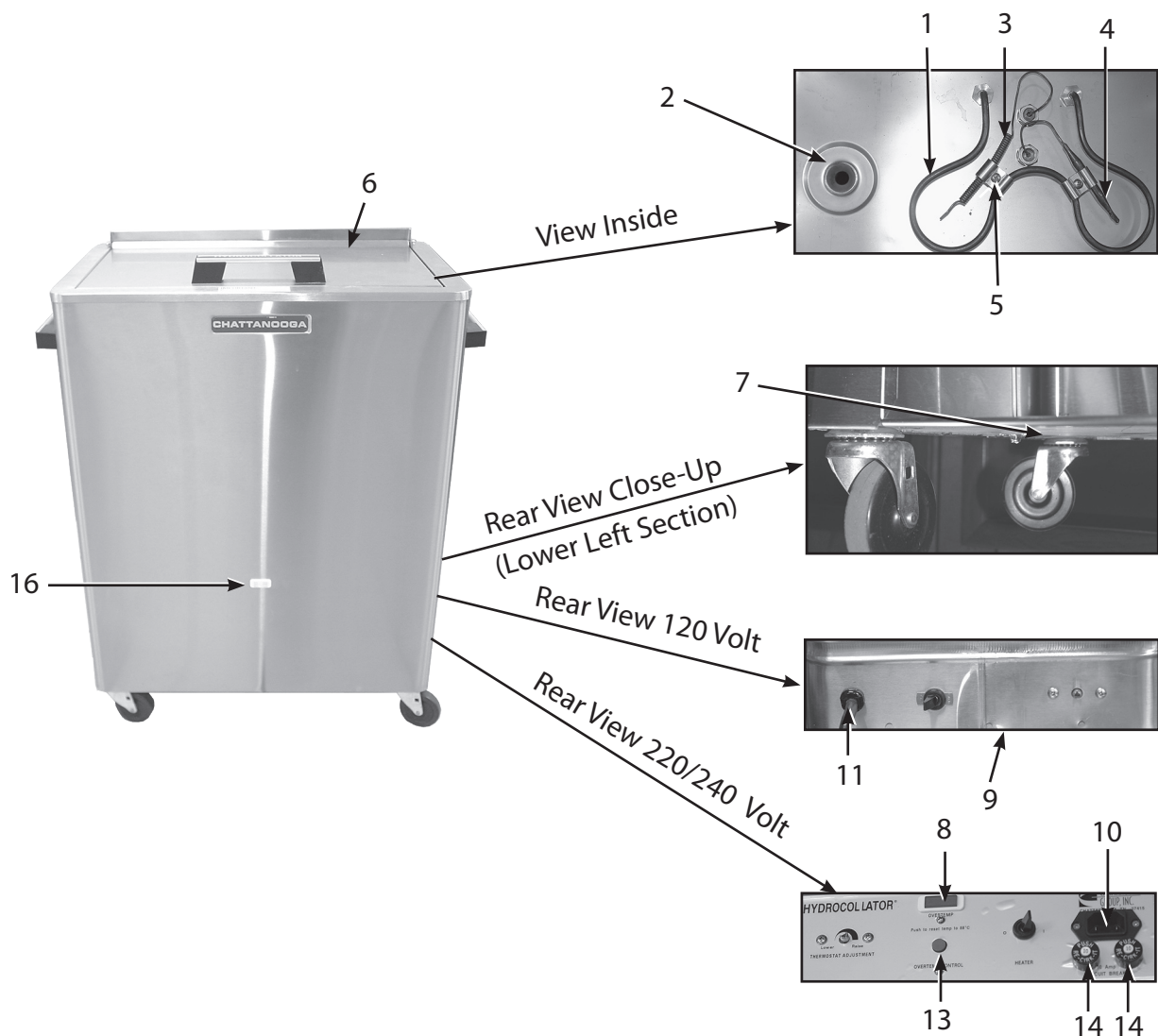**NOTE:** M-4 model shown; models will vary.

\* All 220-240 Volt and M-4 120 Volt have this button


# NOMENCLATURE

Hydrocollator® Heating Units

## Diagram (CONTINUED)

- |                        |                                  |
|------------------------|----------------------------------|
| 17. HotPac             | 23. Side Handle                  |
| 18. Neck Pac Insert    | 24. Thermostat, Hydraulic        |
| 19. Drain, Copper Tube | 25. Thermostat Intensity Control |
| 20. Lid Hinge Nuts     | 26. On/Off Switch                |
| 21. Lid Knob/Handle    | 27. Internal Wiring Harness      |
| 22. Lid Hinge          |                                  |

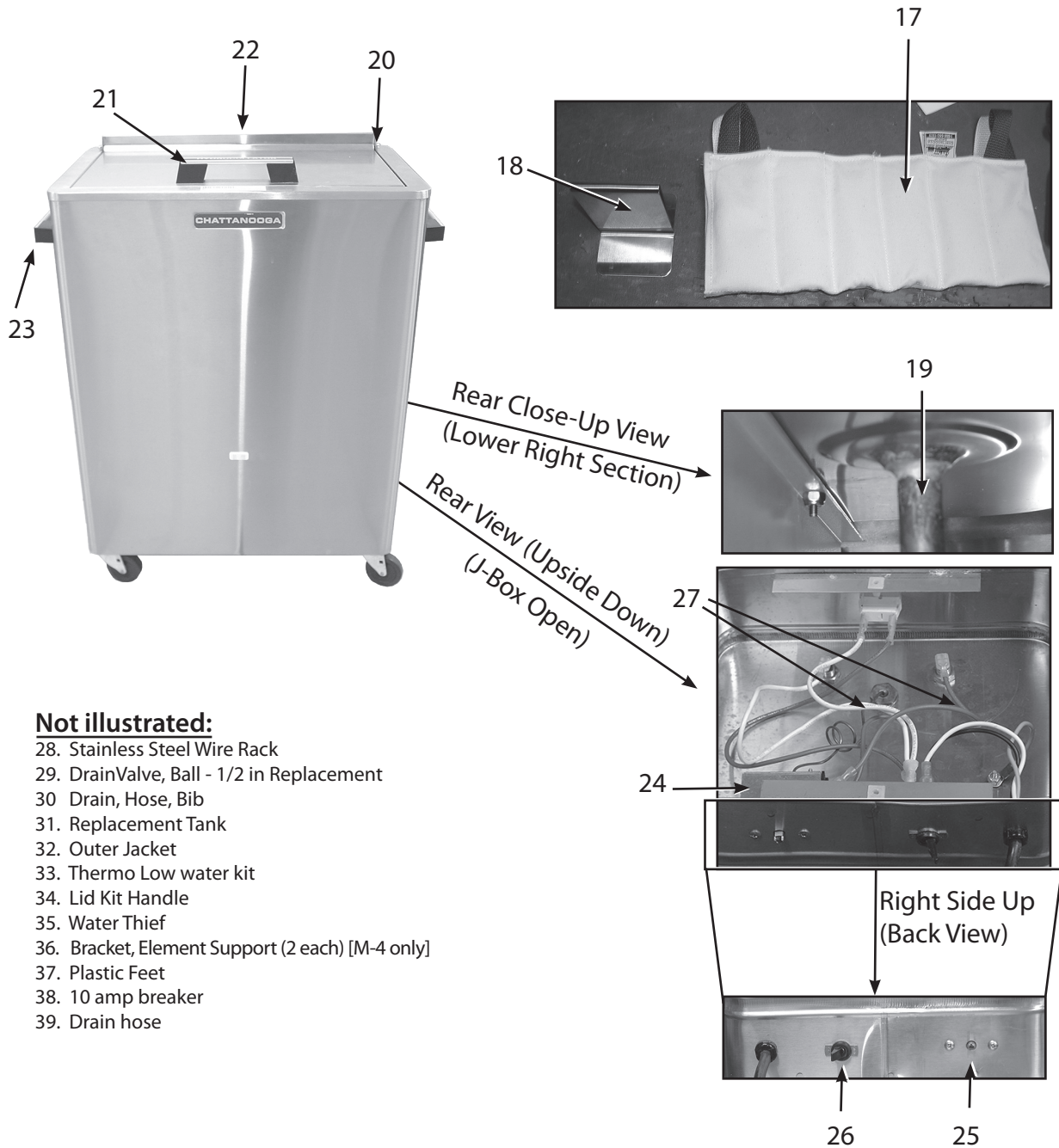## TECHNICAL SPECIFICATIONS

Hydrocollator® Heating Units

### SS-2

Power Supply	110-120 V~ 220-240 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	49.32 Liters
Filled Weight	85 kg (140 lbs)
Temperature Range	71° C - 74° C (160° F - 165° F)
Thermal Cut-out Temp	88° C ± 8° C (190° F ± 14° F)
Heating Up Time	Time to 70° C (160° F) - 4 Hours
Cool Down Time	Time from 70° C (160° F) - 2 Hours
Safety Class	Type B ⚠
Safety Tests	IEC/EN 60601-1
Dimensions	85 cm H x 52 cm L x 39 cm W (33 in H x 21 in L x 16 in W)
Product Weight	30.8 kg (68 lbs)
Shipping Weight	33.6 kg (74 lbs)

**Standard Accessories:** Includes eight Standard size HotPacs.

### M-2

Power Supply	110-120 V~ 220-240 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	68.81 Liters
Filled Weight	110 kg (190 lbs)
Temperature Range	71° C - 74° C (160° F - 165° F)
Thermal Cut-Out Temp	88° C ± 8° C (190° F ± 14° F)
Heat Up Time	Time to 70° C (160° F) - 6 Hours
Cool Down Time	Time from 70° C (160° F) - 3 Hours
Safety Class	Type B ⚠
Safety Tests	IEC/EN 60601-1
Dimensions	85 cm H x 67 cm L x 40 cm W (33 in H x 27 in L x 16 in W)
Product Weight	39 kg (86 lbs)
Shipping Weight	42.2 kg (93 lbs)

**Standard Accessories:** Includes twelve Standard size HotPacs.

## TECHNICAL SPECIFICATIONS

Hydrocollator® Heating Units

### M-4

Power Supply	110-120 V~ 220-240 V~
Rated Frequency	50/60 Hz
Power Consumption	1500 W
Tank Capacity	135.76 Liters
Filled Weight	200 kg (340 lbs)
Temperature Range	71° C - 74° C (160° F - 165° F)
Thermal Cut-out Temp	88° C ± 8° C (190° F ± 14° F)
Heating Up Time	Time to 70° C (160° F) - 8 Hours
Cool Down Time	Time from 70° C (160° F) - 4 Hours
Safety Class	Type B 
Safety Tests	IEC/EN 60601-1
Dimensions	85 cm H x 89 cm L x 50 cm W (33 in H x 35 in L x 20 in W)
Product Weight	61.2 kg (135 lbs)
Shipping Weight	64.9 kg (143 lbs)

**Standard Accessories:** Includes twenty-four Standard size HotPacs.

### SS

Power Supply	110-120 V~, 220-240 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	31.64 Liters
Filled Weight	56 kg (120 lbs)
Temperature Range	71° C - 74° C (160° F - 165° F)
Thermal Cut-out Temp	88° C ± 8° C (190° F ± 14° F)
Heating Up Time	Time to 70° C (160° F) - 3 Hours
Cool Down Time	Time from 70° C (160° F) - 2 Hours
Safety Class	Type B 
Safety Tests	IEC/EN 60601-1
Dimensions	51 cm H x 46 cm L x 41 cm W (20 in H x 18 in L x 16 in W)
Product Weight	23.1 kg (51 lbs)
Shipping Weight	24.5 kg (54 lbs)

**Standard Accessories:** Includes eight Standard size HotPacs.

## TECHNICAL SPECIFICATIONS

Hydrocollator® Heating Units

### E-2

Power Supply	110-120 V~, 220-240 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	43.23 Liters
Filled Weight	60 kg (125 lbs)
Temperature Range	71° C - 74° C (160° F - 165° F)
Thermal Cut-Out Temp	88° C ± 8° C (190° F ± 14° F)
Heat Up Time	Time to 70° C (160° F) - 4 Hours
Cool Down Time	Time from 70° C (160° F) - 2 Hours
Safety Class	Type B 
Safety Tests	IEC/EN 60601-1
Dimensions	51 cm H x 38 cm L x 33 cm W (20 in H x 15 in L x 13 in W)
Product Weight	14.1 kg (31 lbs)
Shipping Weight	15.9 kg (35 lbs)

**Standard Accessories:** Includes two Oversize HotPacs, three Standard size HotPacs and 1 Neck Contour HotPac.

### E-1

Power Supply	110-120 V~, 220-240 V~
Rated Frequency	50/60 Hz
Power Consumption	1000 W
Tank Capacity	15.44 Liters
Filled Weight	25 kg (55 lbs)
Temperature Range	71° C - 74° C (160° F - 165° F)
Thermal Cut-out Temp	88° C ± 8° C (190° F ± 14° F)
Heating Up Time	Time to 70° C (160° F) - 2 Hours
Cool Down Time	Time from 70° C (160° F) - 1 Hour
Safety Class	Type B 
Safety Tests	IEC/EN 60601-1
Dimensions	41 cm H x 33 cm L x 20 cm W (16 in H x 13 in L x 8 in W)
Product Weight	6.4 kg (14 lbs)
Shipping Weight	7.3 kg (16 lbs)

**Standard Accessories:** Includes four Standard size HotPacs.

## SETUP

Hydrocollator® Heating Units

Remove all of the contents inside the Hydrocollator Heating Unit and then replace the wire rack. Make certain the drain valve is closed before filling the tank with water. The tank should be approximately 3/4 full with packs removed. The tank may be filled from any standard type faucet.

Check the voltage rating on the serial decal plate which is located on the back of the unit. To turn the unit on, simply plug the unit into a 120 Volt, 60 Hz or 220/240 Volt, 50-60 Hz outlet as indicated on the decal plate. To turn the unit off, simply unplug from outlet. Do not attempt to use direct current. Follow the procedures indicated in the Precautionary Instructions. Do not attempt to use the unit if it is not properly grounded.

## OPERATION

The Hydrocollator Heating Unit is simple and easy to operate. The controls are located in the rear, near the bottom of the unit. The thermostat controls the temperature of the water. It is located near the bottom of the unit on the control panel. It has been tested and preset at the factory to normal operating temperature. If further adjustment is necessary, it can be easily accomplished by turning the adjusting screw located on the thermostat. ***Turning the outer screw counterclockwise lowers the temperature, while turning it clockwise raises it.***

**NOTE: Do not turn the inner stem under any circumstance! Stem is preset by the manufacturer.**

The Hydrocollator Heating Units operate through the use of a hydraulic capillary-type thermostat and can be left plugged in and turned on to provide a ready supply of heated HotPacs.

The HotPac rack accommodates different sizes and types of packs.

***SS-2 Unit*** – The unit comes complete with eight standard HotPacs.

***M-2 and M-4 Units*** - The M-2 unit comes complete with 12 standard HotPacs. The M-4 comes complete with 24 standard HotPacs.

***SS Unit*** – Holds eight Standard Size packs. Special rack holds two Neck Contour packs.

***E-2 Unit*** – The E-2 unit comes complete with two Oversize, three Standard Size packs and one Neck Contour pack.

***E-1 Unit*** – With the insert in, the rack accommodates four Standard Size packs.

# TROUBLESHOOTING

Hydrocollator® Heating Units

The following are solutions to minor performance problems. Only qualified personnel should service equipment problems.

<b>PROBLEM</b>	<b>POSSIBLE CAUSE</b>	<b>POSSIBLE SOLUTION</b>
Unit plugged in, water does not get hot	<ol style="list-style-type: none"><li>1. Breaker for electric circuit off</li><li>2. Thermostat not functioning</li><li>3. Heating element burned out</li></ol>	<ol style="list-style-type: none"><li>1. Turn breaker on</li><li>2. Replace thermostat</li><li>3. Replace element</li></ol>
Cloudiness of water	<ol style="list-style-type: none"><li>1. Filler from packs is seeping</li><li>2. Too long a period between cleanings</li></ol>	<ol style="list-style-type: none"><li>1. Replace packs</li><li>2. Drain and clean</li></ol>
Packs too hot	<ol style="list-style-type: none"><li>1. Thermostat set too high</li><li>2. Thermostat failure</li></ol>	<ol style="list-style-type: none"><li>1. Adjust thermostat</li><li>2. Replace thermostat</li></ol>
Packs too cool	<ol style="list-style-type: none"><li>1. Thermostat set too low</li><li>2. Thermostat failure</li></ol>	<ol style="list-style-type: none"><li>1. Adjust thermostat</li><li>2. Replace thermostat</li></ol>
Unit boils	<ol style="list-style-type: none"><li>1. Thermostat failure</li></ol>	<ol style="list-style-type: none"><li>1. Replace thermostat</li></ol>
Rust/Discoloration	<ol style="list-style-type: none"><li>1. Contact with ferrous material</li><li>2. Deposits on inside of tank</li><li>3. Too much chlorine in water</li></ol>	<ol style="list-style-type: none"><li>1. Remove the ferrous metal item from the unit (staple, paper clip, safety pin, tongs, etc.) and clean thoroughly. Add a dechlorinating agent if local water has high chlorine content.</li><li>2. Thoroughly clean tank and remove all deposits.</li><li>3. Clean tank and rinse thoroughly. Add a dechlorinating agent if local water has high chlorine content.</li></ol>

Listed below are the Power Cords for each Country, their part numbers, and their dimensions.

Ref.	Description
21284	Euro [18 AWG, 203.2 cm (80 in), Shielded]
78121	US [18 AWG, 203.2 cm (80 in), Shielded]
20971	Australian [18 AWG, 203.2 cm (80 in), Shielded]
20972	Swiss [18 AWG, 203.2 cm (80 in), Shielded]
20973	UK [18 AWG, 203.2 cm (80 in), Shielded]
20974	Danish [18 AWG, 203.2 cm (80 in), Shielded]
20975	Japanese [18 AWG, 203.2 cm (80 in), Shielded]
20976	Indian [18 AWG, 203.2 cm (80 in), Shielded]
20977	Israeli [18 AWG, 203.2 cm (80 in), Shielded]

## Care and Cleaning

The Hydrocollator Heating Unit is equipped with an immersion type heating element and a hydraulic capillary type thermostat which evenly maintains the HotPac temperature in the water and provides a ready supply of heated packs. It is critical to maintain the water level over the top of the HotPac to avoid damage to the heating element, the stainless steel, or the HotPac. Water is constantly lost during operation due to evaporation. Therefore, it is essential that water be added daily. The tank should also be drained, cleaned and inspected systematically, at minimum intervals of every two weeks.

Always unplug the unit from the power source when emptying or cleaning the unit. Drain the unit by the drain valve located at the bottom rear of the unit. Either move the unit to a floor drain or attach the extension hose to it when draining. (E-1 units have no drain and can be emptied by turning unit upside down).

### **WARNING**

- DJO, LLC does not recommend the Hydrocollator Heating Unit be moved while filled with heated water. Tipping over could result in a burn to the user or others. If the Hydrocollator Heating Unit must be moved while filled, **exercise extreme caution!** It is recommended to allow the water to cool below 49° C (120° F) before moving the unit.
- Always unplug the Hydrocollator Heating Unit from the power source before attempting to empty water from unit.

The Hydrocollator Heating Unit is made of high quality stainless steel that, with proper care and maintenance, could easily last a lifetime. Improper care could allow the stainless steel to wear prematurely.

Follow these simple steps to ensure that the Hydrocollator Heating Unit remains in excellent condition:

1. Excessive chlorine can be detrimental to the stainless steel in the Hydrocollator Heating Unit. No bleach or any cleaner with high chlorine content should be used under any circumstance.
2. Clean regularly for optimal service and operation.  
**NOTE: Failure to maintain the unit (including regular cleaning) will void your warranty.**
3. Fill daily with water. Water is constantly lost during operation due to evaporation.
4. Chlorine in regular tap water may be present in high enough concentrations to damage the unit. If you suspect high levels of chlorine in the Hydrocollator Heating Unit, the addition of a dechlorinator is recommended.
5. Certain additives (such as herbal teas, essential oils, etc.) will damage the HotPacs and components of the heating unit, including the stainless steel. The Hydrocollator Heating Unit was designed to heat water and HotPacs only. The addition of any such foreign substance (except dechlorinator) to the water in the unit will void the warranty.
6. Inspect the heating element for pitting and other visible damage during regular cleaning intervals.
7. Inspect the power cord for wear, fraying, and other visible damage during regular cleaning

## To avoid potential for rusting:

1. **Do not** use bleach or cleaners with high chlorine content.
2. **Do not** use paper clips, safety pins, staples, coat hangers, etc., in, on, or near the unit.
3. **Do not** clean the unit with carbon steel wool or a carbon steel brush. Use only soft cloth, Scotch-Brite™ type scouring pads of green, blue, or white abrasiveness, or stainless steel wool if absolutely necessary.
4. **Do not** use abrasive cleaners which tend to disturb the grain in the finish, especially on the exterior of the unit. For exterior cleaning, use only non-abrasive stainless steel polish.
5. Water level is critical: Add water daily, if necessary, to keep it full.
6. Regularly clean and drain the tank (**every two weeks**). Failure to properly maintain the unit will cause premature wear and will void the warranty.
7. Remove all deposits from interior surfaces and parts. The deposits are concentrations of minerals that will allow rusting to begin. If allowed to accumulate on the heating element, these deposits will lower the efficiency of the element and increase the power consumption of the unit. However, exercise extreme care if cleaning element or thermostat(s). These components can be easily damaged without proper care or caution.
8. If you suspect that you have “hard water”, you may need to change the water and clean the unit more frequently. Salt deposits are an indication of hard water that will eventually cause rust.

## Cleaning Tips

1. The interior of the unit should be cleaned, at least every two weeks, using a low abrasive bathroom cleaner with a soft cloth or green, blue, or white Scotch-Brite™ type scouring pad. Stainless steel wool may be used if necessary, but not regular, carbon steel wool. Check for low or no chlorine content in your cleaner and make sure that the residue is thoroughly rinsed away with water.
2. A strong solution of vinegar and water may be used to dissolve away deposits, which then must be thoroughly rinsed away. Results may vary.
3. For exterior cleaning, use stainless steel polish and a soft cloth only. Numerous industrial stainless steel polishes are available from various supply companies. Use of any cleaner except stainless steel polish on the exterior of the unit is not recommended. Abrasive cleaners and scouring pads can damage the finish of the unit.

## Care of HotPacs

Always return the Hydrocollator HotPac to the hot water of the heating unit after each treatment. There it is heated, kept clean, and ready for immediate use. The HotPac may be boiled; however, this hastens the deterioration of the pack. The pack may also be cleaned by scrubbing the pack with soap and water. The simplified method of keeping the pack clean is to keep it immersed in water.

The Hydrocollator HotPac, under constant daily use, should have a useful life of at least one year. When the pack begins to wear out, the filler oozes and leaks through the fabric of the cloth and sections of the pack appear to be loose and empty and will not retain heat properly. The pack should be replaced when it reaches this state. Original replacement Hydrocollator HotPacs may be ordered from an authorized DJO, LLC dealer.

**NOTE:** If HotPacs are to be stored for extended periods, they may be placed - while wet - in plastic bags and stored in a freezer.

DJO, LLC ("Company") warrants that the Hydrocollator Models SS-2, M-2, SS, E-1, E-2, and M-4 Heating Units ("Products") are free of defects in material and workmanship. This warranty shall remain in effect for one year (12 months) from the date of original consumer purchase. If these Products fail to function during the one year warranty period due to a defect in material or workmanship, at the Company's option, the Company or the selling dealer will repair or replace this Product without charge within a period of thirty days from the date on which the Product is returned to the Company or the dealer.

All repairs to the Product must be performed by a service center authorized by the Company. Any modifications or repairs performed by unauthorized centers or groups will void this warranty.

To participate in warranty coverage, this Product's warranty registration card (included with Product) must be filled out and returned to the Company by the original owner within ten (10) business days of purchase.

**This Warranty Does Not Cover:**

- Replacement parts or labor furnished by anyone other than the Company, the selling dealer or a certified Company service agent.
- Defects or damage caused by labor furnished by someone other than Company, the selling dealer or a certified Company service agent.
- Any malfunction or failure in the Product caused by product misuse, including, but not limited to, the failure to provide reasonable and necessary maintenance or any use that is inconsistent with the Product User Manual.

**COMPANY SHALL NOT BE LIABLE IN ANY EVENT FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES.**

Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

To obtain service from Company or the selling dealer under this warranty:

1. A written claim must be made within the warranty period to the Company or the selling dealer.

Written claims made to the Company should be sent to:

DJO, LLC  
1430 Decision St  
Vista, CA 92081 USA  
Phone: 1-800-592-7329 USA  
Phone: 1-423-870-2281 or 1-317-406-2250  
Fax: 1-317-406-2014

2. The Product must be returned to the Company or the selling dealer by the owner.

This warranty gives you specific legal rights and you may also have other rights which vary from state to state.

The Company does not authorize any person or representative to create for it any other obligation or liability in connection with the sale of the Product. Any representation or agreement not contained in the warranty shall be void and of no effect.

**THE FOREGOING WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESSED OR IMPLIED, INCLUDING ANY WARRANTY OR MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.**

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\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## REPLACEMENT PARTS

Hydrocollator® Heating Units

### Model M-2 220-240 VOLT – (Low Water)

Cat. No. – 2407 w/packs

Cat. No. – 2409 w/o packs

Key*	Part Description	Part No.
1	Element, Heating, 220 Volt~, 1000 Watt	22245
4	Thermostat, Hydraulic	10631
3	Lamp, Pilot, Green	22146
11	Cord Set (Country Specific)	See page 13
26	Switch	23406
28	Rack	22304
	Hose, Nylon 5/8 x 4' Coupled M-F	21635
7	Caster, 3 inch	21259
21	Handle, M/SS, Lid, Finished	20513
23	Handle, Side, Common (2)	20237
34	Lid, Assembly With Handle	22297
9	Cover, Junction Box, M-2, 220-240 Volt	22087
19	Drain, Copper M-2 Part A #25010	21286
5	Bracket, Bulb Thermostat, S/A (Lo-Water) (2)**	21023
35	Water Thief	21882
	Spacer, Thermostat (Lo-Water) (2)	72393
	Thermostat, (Lo-Water)***	20548
15	Lamp, Red, Indicator (Lo-Water)	22147
14	Fuse, 6.3 Amp. (2)	71772
	Fuse Holder (2)	72734
	Receptacle, Power 8843-FSG-W-40-60	74723
27	Harness Wiring	61579
31	Tank, Replacement with Drain	23373
18	Insert, For neck Contour Packs	21053
20	Lid, Rod Nuts (2)	21737
22	Lid, Rod	22303
29	Valve, Ball	21085
	Fit Bib, SS-2, M-2, M-4	23392

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

# REPLACEMENT PARTS

Hydrocollator® Heating Units

## Model M-4 – Standard

Cat. No. – 2502 w/packs

Key*	Part Description	Part No.
1	Heating Element M-4, 120 Volt, 1000 Watt	20570
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid	22348
7	Caster, 3 inch	21260
9	Junction Box Cover	22360
11	Power Cord	20568
39	Hose, Drain	21635
15	Pilot Lamp, Red	22149
16	Pilot Lamp, Clear	22148
19	Drain, Copper Tube Part A	21286
20	Lid Hinge Rod Nuts (2 required)	21737
21	Lid Knob/Handle	21025
22	Lid Hinge Rod	22361
23	Side Handle (Serial # Above 5149)	20237
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	61579
28	Rack, Stainless Steel	22374
29	Drain Valve, Ball - 1/2 in Replacement	21085
30	Drain, Hose, Bib	23392
31	Replacement Tank M-4	22343
32	M-4 Outer Jacket	23384
34	Handle, Lid Kit	22171
35	Water Thief	21882
36	Bracket, Element Support (2 each)	22356
26	On/Off Switch	23406

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## REPLACEMENT PARTS

Hydrocollator® Heating Units

### Model M-4 220-240 VOLT – (Low Water)

Cat. No. – 2506 w/packs

Cat. No. – 2508 w/o packs

Key*	Part Description	Part No.
1	Element, Heating, 240 Volt, 1500 Watt	22246
4	Thermostat, Hydraulic	10631
	Lamp, Green, 1607-5107, 220-240 Volt	22146
11	Cord Set (Country Specific)	See page 13
26	Switch	23406
28	Rack, Wire M-4	22374
39	Hose, Nylon 5/8 x 4' Coupled M-F	21635
7	Caster, 3 inch	21260
21	Handle, M/SS, Lid, Finished	20513
34	Lid, Sub-Assembly M-4, With Handle	22348
9	Cover, Junction Box, M-4	22360
19	Drain, Copper M-2 Part A #25010	21286
5	Bracket, Element Support	22356
35	Water Thief	21882
38	Breaker, Heinemann, 10 amp (2)	22173
	Thermostat, (Lo-Water)***	20548
15	Lamp, Red, Indicator (Lo-Water)	22147
	Receptacle, Power 8843-FSG-W-40-60	74723
31	Tank, M-4, Replacement with Drain	23384
20	Lid, Rod Nuts (2)	21737
22	Lid, Rod	22361
29	Valve, Ball	21085
	Fit Bib, SS-2, M-2, M-4	23392
5	Bracket, Bulb Thermostat, S/A (Lo-Water) (2)**	21023

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

# REPLACEMENT PARTS

Hydrocollator® Heating Units

## Model SS-2 – Standard

Cat. No. – 2302 w/packs

Key*	Part Description	Part No.
1	Element, Heating 120 Volt, 1000 Watt	10630
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid SS-2	22219
7	Caster, 3 inch	21259
9	Cover, Junction Box	22087
11	Power Cord	20568
39	Hose, Drain	21635
15	Pilot Lamp, Red	22149
16	Pilot Lamp, Clear	22148
19	Drain, Copper Tube Part A	21286
20	Lid Hinge Rod Nuts (2 each)	21737
21	Lid Knob/Handle	21025
22	Lid Rod Hinge	22231
23	Handle, Side (2)	20237
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	61579
28	Stainless, Wire Rack	22259
29	DrainValve, Ball - 1/2 in Replacement	21085
30	Drain, Hose, Bib	23392
31	Replacement Tank SS-2	23385
32	SS-2 Outer Jacket	22425
34	Handle, Lid Kit	22171
35	Water Thief	21882
26	On/Off Switch	23406

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

# REPLACEMENT PARTS

Hydrocollator® Heating Units

## Model SS-2 220-240 VOLT – (Low Water)

Cat. No. – 2307 w/packs

Cat. No. – 2309 w/o packs

Key*	Part Description	Part No.
1	Element, Heating, 240V~ CE	22245
4	Thermostat, Hydraulic	10631
3	Lamp, Pilot, Green	22146
11	Cord Set (Country Specific)	See page 13
26	Switch	23406
28	Rack	22259
39	Hose, Nylon 5/8 x 4' Coupled M-F	21635
7	Caster, 3 inch	21259
21	Handle, M/SS, Lid, Finished	20513
23	Handle, Side, Common (2)	20237
34	Lid, Assembly With Handle	22219
9	Cover, Junction Box, M-2, 220-240 Volt	22087
29	Drain, Copper M-2 Part A #25010	21286
5	Bracket, Bulb Thermostat, S/A (Lo-Water) (2)**	21023
35	Water Thief	21882
	Spacer, Thermostat (Lo-Water) (2)	72393
	Thermostat, (Lo-Water)***	20548
15	Lamp, Red, Indicator (Lo-Water)	22147
14	Fuse, 6.3 Amp. (2)	71772
	Fuse Holder (2)	72734
	Receptacle, Power 8843-FSG-W-40-60	74723
27	Harness Wiring	61579
31	Tank, Replacement with Drain	23372
18	Insert, For neck Contour Packs	21053
	Thermostat, LM5-2505	23393
20	Lid, Rod Nuts (2)	21737
22	Lid, Rod	22231
29	Valve, Ball	21085
	Fit Bib, SS-2, M-2, M-4	23392

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## REPLACEMENT PARTS

Hydrocollator® Heating Units

### Model SS 120 VOLT – Standard

Cat. No. – 2202 w/packs

Key*	Part Description	Part No.
1	Element, Heating 120 Volt, 1000 Watt	10630
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid	22219
9	Junction Box Cover	22223
11	Power Cord	20568
39	Hose, Drain	21635
15	Pilot Lamp, Red	22149
16	Pilot Lamp, Clear	22148
19	Drain, Copper Tube Part A	21288
20	Lid Hinge Rod Nuts (2 required)	21737
21	Lid Knob	21025
22	Lid Rod	22231
33	Thermo Low Water Kit***	20548
26	Overtemp Thermostat Control	20548
27	Wiring (Internal) Harness	21031
29	DrainValve, Ball - 1/2 in Replacement	21373
34	Handle, Lid Kit	22171
35	Water Thief	21882
37	Feet, Plastic	75538
26	On/Off Switch	23406

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## REPLACEMENT PARTS

Hydrocollator® Heating Units

### Model SS 220/240 VOLT – (Low Water)

Cat. No. – 2207 w/packs

Cat. No. – 2209 w/o packs

Key*	Part Description	Part No.
1	Element, Heating 1000 Watt	22245
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
9	Junction Box Cover	22223
39	Hose, Drain	21635
15	Pilot Lamp, Red	22147
19	Drain, Copper Tube Part A	21288
20	Lid Hinge Rod Nuts (2 required)	21737
22	Lid Rod	22231
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	61579
29	Valve, Boiler Drain, 1/2 I.D. Chrome	21373
	Carrier, Fuse	72733
	Holder, Fuse	72734
34	Handle, Lid finished	20531
35	Water Thief	21882
26	On/Off Switch	23406
	Lamp, Green	22146
	Cordset (Country Specific)	See page 13
28	Rack, Wire	22232
	Drain, Copper, Part B	21289
18	Insert, For Neck Contour Packs	21053
32	Tank, Replacement, w/Drain	23385
14	Fuse, Anti-Surge, 6.3 Amp (2)	71772

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## REPLACEMENT PARTS

Hydrocollator® Heating Units

### Model E-1 120 VOLT – Standard

Cat. No. – 2102 w/packs

Key*	Part Description	Part No.
1	Element, Heating 120 Volt, 1000 Watt	21007
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid	21004
9	Junction Box Cover	21005
11	Power Cord	20568
15	Pilot Lamp, Red	22149
16	Pilot Lamp, Clear	22148
20	Lid Hinge Rod Nuts (2 required)	21737
28	Rack, Stainless Steel	21062
21	Handle, Lid	21025
22	Hinge Rod, Lid	22668
23	Handle, Side Kit	21994
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	21031
26	On/Off Switch	23406

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## REPLACEMENT PARTS

Hydrocollator® Heating Units

### Model E-1 220/240 VOLT – (Low Water)

Cat. No. – 2107 w/packs

Cat. No. – 2109 w/o packs

Key*	Part Description	Part No.
1	Element, Heating 1000 Watt	22245
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid	21004
9	Junction Box Cover	21005
15	Pilot Lamp, Red	22147
20	Lid Hinge Rod Nuts (2 required)	21737
28	Rack, Stainless Steel	21062
21	Handle, Lid	21025
22	Hinge Rod, Lid	22668
23	Handle, Side Kit (2)	21994
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	61579
	Pilot Lamp, Green	22146
11	Cordset (Country Specific)	See page 13
18	Insert, For Neck Contour Packs	21053
	Receptacle, Power	74723
	Carrier, Fuse	72733
	Holder, Fuse	72734
14	Fuse, Anti-Surge, 6.3 Amp (2)	71772
	Washer, Nylon	21027

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## REPLACEMENT PARTS

Hydrocollator® Heating Units

### Model E-2 120 VOLT – Standard

Cat. No. – 2802 w/packs

Key*	Part Description	Part No.
1	Element, Heating 120 Volt, 1000 Watt	10630
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid	23457
9	Junction Box Cover	23460
11	Power Cord	20568
39	Hose, Drain	21635
15	Pilot Lamp, Red	22149
16	Pilot Lamp, Clear	22148
19	Drain, Copper Tube Part A	21288
21	E-2 Lid Handle	20512
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	21031
29	DrainValve, Ball - 1/2 in Replacement	21373
35	Water Thief	21882
26	On/Off Switch	23406

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

## REPLACEMENT PARTS

Hydrocollator® Heating Units

### Model E-2 220/240 VOLT – (Low Water)

Cat. No. – 2806 w/packs

Cat. No. – 2808 w/o packs

Key*	Part Description	Part No.
1	Element, Heating 1000 Watt	22245
4	Hydraulic Thermostat	10631
5	Bracket & Bulb Thermostat A/B Assembly (2 each)**	21023
6	Lid	23457
9	Junction Box Cover	23460
10	Cordset (Country Specific)	See page 13
39	Hose, Drain	21635
19	Drain, Copper Tube Part A	21288
21	E-2 Lid Handle	20512
33	Thermo Low Water Kit***	20548
27	Wiring (Internal) Harness	61579
29	DrainValve, Ball - 1/2 in Replacement	21373
35	Water Thief (2)	21882
26	On/Off Switch	23406
	Lamp, Green, Lens	22146
15	Lamp, Red, Lens	22147
	Carrier, Fuse (2)	72733
	Holder, Fuse (2)	72734
14	Fuse, Anti-Surge, 6.3 Amp (2)	71772
22	Lid, Rod	90796
18	Insert, For Neck Contour Packs	21053
28	Rack, Wire	22259

\* pages 6 and 7

\*\* Used to hold the Hydraulic Thermostat and Overtemp Thermostat to the Heating element.

\*\*\* Includes Gasket Thermostat, Nut 1/2-20 Brass, and Overtemp Thermostat.

**Moving  
Rehabilitation  
Forward™**

**DJO is an ISO 13485 Certified Company**

DJO, LLC  
1430 Decision St  
Vista, CA 92081 USA  
Phone: 1-800-592-7329 USA  
Phone: 1-423-870-2281 or  
1-317-406-2250  
Fax: 1-317-406-2014  
chattgroup.com

© 2009 DJO, LLC

MDSS GmbH  
Schiffgraben 41  
30175 Hannover  
Germany  
Telephone: +49-511-6262-8630  
Fax: +49-511-6262-8633

2 2 3 2 3

22323\_H