

The black urethane cover is very porous, so when the packs are stored for a long time on a shelf, the glycol leaches out of the pack. Customers are advised to either put the packs in a chilling unit right away, or rotate their stock often.

Instructions / Precautions

Application:

- For initial chilling of the ColPaC, place flat in chilling unit freezer for two hours. Do not chill below 0° F (-17° C).
- Remove ColPaC from freezer and place a hand towel between the skin and the ColPaC. Apply to area being treated.
- Treatment time should not exceed 30 minutes.

Precautions:

- Do not sit, lie or sleep on the ColPaC.
- Constantly monitor the ColPaC application to insure that the patient's skin is not being irritated by the cold.
- Do not attempt to use the ColPaC as a hot pack. No hot water. No microwave. Use only as a cold therapy.

Care of Your ColPaC®:

- Wipe all moisture from the ColPaC before returning to freezer.
- It is best not to fold the ColPaC for storage in freezer.
- ColPaCs are most serviceable when kept in a freezer set at 12° F (-11° C).
- Store in freezer when not in use.