



### Specifications:

**Speed:** 24 cycles per minute at 60Hz  
20 cycles per minute at 50Hz

**Motor:** 110v or 220v

**Capacity:** 16 tubes

**Width:** 5.5 in (13.3 cm)

**Length:** 13.5" (34.3 cm)

**Height:** 4 in (10.2 cm)

**Weight:** 4.82 lbs. (1.80 kg)

### Unpacking

Carefully unpack the Blood Rocker, saving packing materials in the unlikely event you need to return the rocker. Immediately call LW Scientific if you detect any damage. If you need to return the rocker, please use the original packing material.

### Operation

- Place samples to be mixed on the top of the rocker.
- Flip the Rocker Switch to the "on" position.

### Controls

Rocker Switch: Used to turn the blood rocker on and off.

### Setup

Place rocker on a flat, stable surface. Ensure that the controls are set in the "off" position. Plug into the appropriate electrical outlet.



### Notes for Safe Use:

- Do not operate unit in or near water or other liquids.
- Unplug unit before cleaning or repairing.
- Do not use if electrical cord is damaged.
- Replace cord if damaged.
- Always treat specimen samples as though they are infectious. The tube mat is washable. Do not autoclave.