NYOrtho

PATIENTSAFETY

The **NYOrtho Smoker's Apron** is ideal for patients requiring a protective covering to shield against a burning match or lit cigarette, while accommodating patient/resident rights.

· Certified flame-resistant fabric meets ASTM F2302.

NYOrtho Smoker's Apron meets compliance requirements for use in institutional facilities, such as nursing homes, assistive living, and similar facilities. Read all instructions and warnings before using Smoker's Apron. Keep instructions and warnings for future use.

• Flame-resistant apron protects against smoldering fires and cannot be ignited by a cigarette, cigar or burning match.

Side straps prevent cigarettes and ashes from falling between resident and the wheelchair

- · Wipe clean, lightweight and foldable
- Available oversized for Geri-chair (9531)
- Universal sizing



Keep instructions and warnings for future use. Instructions available for download at: https://www.nyorthousa.com/product/nyortho-smokers-apron

- A Before each use, inspect apron for broken stitches or parts, as well as torn, cut or frayed material. Do not use soiled or damaged material.
- B Seat resident in wheelchair and drape apron over the resident. Secure the apron by engaging the hook and loop neck strap located at the top of the apron. Adjust the neck strap until the apron reaches just below the neckline.
- C IMPORTANT: Engaging the side straps to prevent cigarettes and ashes from falling between resident and the wheelchair, ALWAYS drape the apron over the arm rests and wrap the side-straps around and under the arm rests and securely engage the side-straps' hook and loop to the underside of the apron. See fig. 1 & 2.

WARNING: Suffocation Hazard. Make sure the patient is securely positioned and cannot slide down or fall off the chair seat and become entrapped by the Smoker's Apron. If their body becomes suspended by the apron, suffocation could result.

After properly applying the apron, place resident in a safe open, area away from any other combustible items that could catch fire. Make sure resident in a location that is clearly visible to staff and without any visual obstruction. Closely supervise wheelchair occupant at all times.

WARNING: Fire Hazard. Although this apron is flame-resistant, cigarettes can lead to dangerous smoldering fires on seating, such as upholstererd dining chairs, wheelchairs, newspapers, and other nearby combustibles. The apron is not a substitute for proper supervision.

The webbing closure around the neck and binding around the circumference of the apron is fire rated CA117 and will melt / self-extinguish when exposed to ash or flame. Both of these components are not certified ASTM F2302.

Economy Smoker's Apron

Deluve Smoker's Anron										
		9531	White	30" x 44"	Geri-Chair	1/EA				
		9530-3	White	30" x 34"	Wheelchair	3/PK				
		9530	White	30" x 34"	Wheelchair	1/EA				

Deluxe Smoker's Apron

9532	Gray	30" x 34"	Wheelchair	1/EA
9532-3	Gray	30" x 34"	Wheelchair	3/PK

Care Instructions

The smoker's apron is a wipe clean fabric. Do NOT launder in washing machine. Do NOT use dryer. Wipe clean after each use. To clean the straps, hand wash with mild soap or detergent and rinse thoroughly. After cleaning, products MUST be rinsed with water to remove any residual chemicals. Make sure product is completely dry before use.