



Fall Management made simple, effective & affordable



Patterson Medical's new line of alarms combine affordability and ease of use to offer solutions that aren't just great for your patients, but great for your budget too.


PATTERSON
MEDICAL

Patterson Medical Patient Monitoring Alarms



Patterson Medical Alarm Monitors. For use with the Patterson Medical wheelchair sensor pads and bed sensor pads featured on the next page. The unit is easy to move from wheelchairs to beds with the included strap or optional bed bracket clip. State-of-the-art technology ensures patient safety and meets required protocols. These monitors, along with new Patterson Medical pads, activate an alarm when a patient stands up or shifts weight off pad. Unique status lights give caregivers peace of mind.

- Sensor Pad Connect (green flash): Indicates that the alarm and pad have been connected and the alarm system is ready to use. Green flash goes off when pad is lost, not there or faulty.
- Low Battery Alert (amber flash): If light is flashing, replace battery immediately.
- Alarm Alert (red flash): Patient has moved off the pad and needs assistance.
- The battery compartment cover slides then snaps in. Includes a mounting strap, mounting hardware and a 9V battery.
- Reset button needs to be pressed 3 times in 2 seconds to reset alarm. This helps to prevent patient tampering.

- Economy and Standard Alarm Monitors have two tones to choose from and the Deluxe Alarm Monitor has one tone and a voice record feature.
- Standard and Deluxe Alarm Monitors have a switch inside the battery compartment that can be positioned to NC to activate the nurse call feature. Alarm does not sound in room, but will signal at the nurse's station.

0815-62-537 Economy Alarm Monitor

0815-61-497 Standard Alarm Monitor

0815-61-505 Deluxe Alarm Monitor

Optional Accessories

0815-61-612 Alarm Monitor Bed Bracket Clip

0815-61-620 AC Adapter

A510-20-2 Nurse Call Cable

5634-58 T-Adapter

A510-20-3 Nurse Call Button Cord

Description	Pad Connect LED Indicator	Alarm LED Indicator	Low Battery LED Indicator	Reset Button	Hi/Lo Silent Switch	0/1sec/2sec Delay Switch	Tone Switch	Voice Record	Nurse Call Jack	AC Adapter Interface
0815-62-537 Economy Alarm Monitor	X	X	X	X	X	X	X			
0815-61-497 Standard Alarm Monitor	X	X	X	X	X	X	X		X	X
0815-61-505 Deluxe Alarm Monitor	X	X	X	X	X	X	X	X	X	X

See next page for Alarm Sensor Pads

Patterson Medical Magnet Alarm. Easy to operate, reliable and economical, too. Used on chairs with the cord attached to patient's clothes. The alarm is activated when the cord is pulled as a result of the patient starting to stand. The pull string has a 1" diameter magnet on one end and a spring-loaded metal clip on the other. Alarm triggers and red light is illuminated when magnet is pulled away from the alarm. Included for attachment of the unit is a spring-loaded clip for easy attachment to a wheelchair.

- The alarm unit measures 5" x 2" x 1.25", making it small and discreet.
- Durable design – doesn't need an expensive rubber protective. Passed drop test by independent lab simulating 500 drops.
- A volume adjustment switch inside the battery compartment can be set at high (105dB) or low (90dB).
- The included 9V battery has an extra secure fit. When the alarm is dropped, the battery does not fall out of the alarm and fly across the floor. Also, makes it more difficult for patients to disable the alarm. Red light flashes when battery is low.
- Tool free battery door – nurse or therapist won't have to search for a screwdriver just to change the battery.
- The length of the pull cord can be adjusted up to 30" so you can determine how far the patient is able to move without setting off the alarm.
- Replacement magnet, clip and cord sold separately.

0815-61-489 Magnet Alarm

0815-61-604 Replacement magnet, clip and cord sold separately (Pkg. of 5).



Patterson Medical Alarm Sensor Pads



Patterson Medical Alarm Sensor Pads. Vinyl-covered, water-resistant pads activate the alarm monitor when the resident rises or moves off the pad. Sensor pad set up instructions are printed right on the pad as well as a location to record the initial date of use.

Item No.	Chair Sensor Pads
0815-61-513	10" x 15", 90-Day
0815-61-521	10" x 15", 180-Day
0815-61-539	10" x 15", 1-Year
Item No.	Bed Sensor Pads
0815-61-547	10" x 30", 90-Day
0815-61-554	10" x 30", 180-Day
0815-61-562	10" x 30", 1-Year
0815-61-570	20" x 30", 90-Day
0815-61-588	20" x 30", 180-Day
0815-61-596	20" x 30", 1-Year
90-Day Sensor Pads	Ideal for hospitals and patients who have a short-term stay
180-Day Sensor Pads	Useful for most rehab stays because of its durability
One-Year Sensor Pads	Long-term stays or at-home use benefit from this most-durable pad

Patterson Medical Alarm Monitor & Sensor Pad Sets

Economy Alarm Monitor Plus Pad	
Bed Pad Alarm Sets	Description
0815-80-646	Economy Alarm Monitor & 10" x 30" Bed Pad, 90-Day
0815-80-679	Economy Alarm Monitor & 10" x 30" Bed Pad, 180-Day
0815-80-687	Economy Alarm Monitor & 10" x 30" Bed Pad, 1-Year
0815-80-695	Economy Alarm Monitor & 20" x 30" Bed Pad, 90-Day
0815-80-703	Economy Alarm Monitor & 20" x 30" Bed Pad, 180-Day
0815-80-711	Economy Alarm Monitor & 20" x 30" Bed Pad, 1-Year
Chair Pad Alarm Sets	Description
0815-80-729	Economy Alarm Monitor & 10" x 15" Chair Pad, 90-Day
0815-80-737	Economy Alarm Monitor & 10" x 15" Chair Pad, 180-Day
0815-80-745	Economy Alarm Monitor & 10" x 15" Chair Pad, 1-Year



Standard Alarm Monitor Plus Pad	
Bed Pad Alarm Sets	Description
0815-80-752	Standard Alarm Monitor & 10" x 30" Bed Pad, 90-Day
0815-80-760	Standard Alarm Monitor & 10" x 30" Bed Pad, 180-Day
0815-80-778	Standard Alarm Monitor & 10" x 30" Bed Pad, 1-Year
0815-80-786	Standard Alarm Monitor & 20" x 30" Bed Pad, 90-Day
0815-80-794	Standard Alarm Monitor & 20" x 30" Bed Pad, 180-Day
0815-80-802	Standard Alarm Monitor & 20" x 30" Bed Pad, 1-Year
Chair Pad Alarm Sets	Description
0815-80-810	Standard Alarm Monitor & 10" x 15" Chair Pad, 90-Day
0815-80-828	Standard Alarm Monitor & 10" x 15" Chair Pad, 180-Day
0815-80-836	Standard Alarm Monitor & 10" x 15" Chair Pad, 1-Year



Deluxe Alarm Monitor Plus Pad	
Bed Pad Alarm Sets	Description
0815-80-844	Deluxe Alarm Monitor & 10" x 30" Bed Pad, 90-Day
0815-80-851	Deluxe Alarm Monitor & 10" x 30" Bed Pad, 180-Day
0815-80-869	Deluxe Alarm Monitor & 10" x 30" Bed Pad, 1-Year
0815-80-877	Deluxe Alarm Monitor & 20" x 30" Bed Pad, 90-Day
0815-80-885	Deluxe Alarm Monitor & 20" x 30" Bed Pad, 180-Day
0815-80-893	Deluxe Alarm Monitor & 20" x 30" Bed Pad, 1-Year
Chair Pad Alarm Sets	Description
0815-80-901	Deluxe Alarm Monitor & 10" x 15" Chair Pad, 90-Day
0815-80-919	Deluxe Alarm Monitor & 10" x 15" Chair Pad, 180-Day
0815-80-927	Deluxe Alarm Monitor & 10" x 15" Chair Pad, 1-Year

Save Money with these Bed Pad & Chair Pad Alarm Sets



www.pattersonmedical.com

To order call:
1-800-323-5547
 In Canada call 1-800-665-9200

Fax toll free:
1-800-547-4333
 In Canada fax 1-877-566-8611

©2013 Patterson Medical Products, Inc.
 All rights reserved. Printed in USA. 081587401