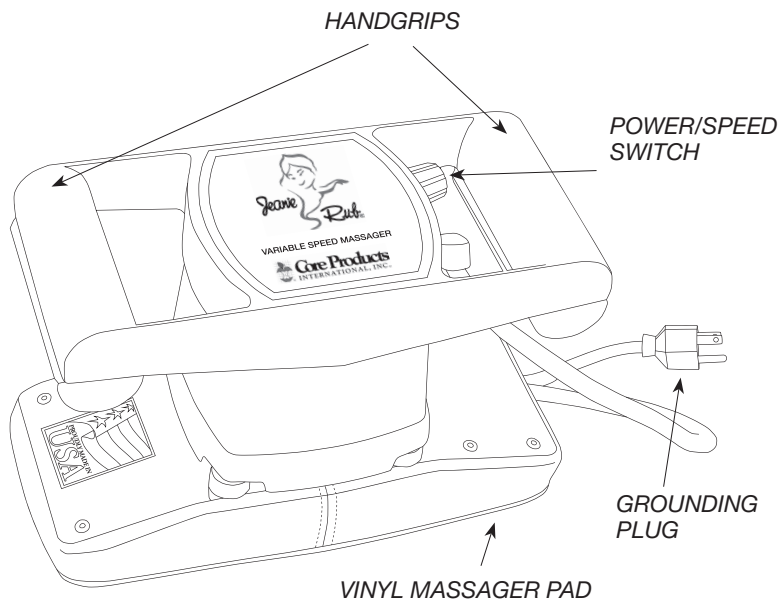
YOUR JEANIE RUB MASSAGER

Jeanie Rub Variable Speed Massager
Model: 3401
Universal motor
120 VAC / 60 Hz / 2.0 Amperes
Operating Range: 1100 - 4600 rpm
Rotary power switch/speed control
6.05 lbs. (2.7 kg)

All Jeanie Rub Massagers are built to ANSI Standards and certified by the Canadian Standards Association. The CSA mark on your massager ensures that it has been evaluated to meet or exceed the ANSI/UL/CSA consumer safety standards for use in the United States and Canada.

INDICATIONS FOR USE

The Jeanie Rub Massager delivers invigorating 1100 - 4600 rpm orbital massaging action. Use the Jeanie Rub massager on any part of the body, head to toe.

GENERAL WARNINGS AND PRECAUTIONS-DANGER:

To reduce the risk of electrical shock.

Connecting this product to an improperly grounded electric receptacle may result in fire or electric shock. Using this product near water may result in electric shock. If you suspect massager may have come into contact with water, do not use.

Grounding Instructions:

This product is equipped with a grounding plug which must be connected to a properly installed and grounded 120 volt receptacle. Do not modify the plug provided. Check with a qualified electrician or serviceman if you are in doubt as to whether the receptacle is grounded.

OPERATING INSTRUCTIONS

Note: Your Jeanie Rub Massager operates at speeds up to 4600 RPMs. If you are accustomed to older Jeanie Rub models, this faster speed may seem excessive. Please be assured this is completely normal as each unit is individually tested to meet our standards.

1. Plug power cord into 120 vac wall receptacle.
2. Firmly grasp massager handgrrips with both hands from above with thumbs toward center of unit.
3. Turn rotary power switch clockwise to begin operation.
4. Turn rotary power switch to adjust speed to a medium operating mode.
5. Place massager in contact with body.
 - Allow massager to do the work. It is not necessary to press down on massager to achieve results.
 - Find a speed that is most comfortable to the individual. Use lower speeds to ensure a more soothing massage. Use higher speeds to deliver a more invigorating massage.
 - Therapeutic benefits can be achieved in as little as 5-10 minutes per area, but may vary by individual.
 - For direct contact on skin, Core Products recommends using any of the Jeanie Rub covers to protect the vinyl pad of your massager. Using covers also allows the use of oils, lotions or creams. Using oils, lotions or creams without a cover may prematurely crack or erode the vinyl pad.
6. To stop operation, turn power/speed switch counter-clockwise.

WARNINGS

- This product is designed to operate on a standard grounded 120 vac receptacle.
- Use massager only for its intended use as described in this manual.
- Do not operate under a blanket or where ventilation can become blocked. Keep ventilation openings free of lint, hair, or other debris. Excessive heat buildup can result in fire or injury.
- This product should never be left unattended when plugged into a receptacle.
- Do not operate if cord has been damaged.
- Do not operate if massager is not functioning properly.
- Do not use outdoors.
- Do not place massager near heated surfaces.
- Do not use accessories other than those recommended by the manufacturer.
- Do not use for longer than 20 minutes to avoid overheating.
- Avoid using excessive pressure to avoid overheating.
- Do not use massager in damp area or wet area.
- Never insert objects into openings on massager.
- Unplug massager from receptacle when not in use.
- Allow adequate time for unit to cool down before storing.
- Do not carry massager by power cord.

PRECAUTIONS

- Use caution when wearing loose clothing or jewelry while operating massager. Loose clothing, jewelry, or hair may get caught in the moving parts of this product.
- Do not allow children to use massager.
- Do not massage any area of the body that is swollen, inflamed, hemorrhaging, or covered with skin lesions. Do not use on individuals with tumors, thrombophlebitis, varicose veins, or lung conditions such as tuberculosis, pneumothorax, tumors, or diseases of the chest wall.
- Do not use massager if any abnormal discomfort is experienced during operation.

CARE AND MAINTENANCE

To keep your massager working and looking its best, follow these simple steps.

1. Unplug your massager from receptacle before cleaning.
2. Wipe the vinyl pad of the massager with a damp cloth and mild detergent to clean. Do not use products containing alcohol, as this may cause the vinyl to prematurely crack or erode.
3. When cleaning around power switch or ventilation, use a dry cloth only. Never use a damp cloth to clean these areas.
4. This product is maintenance free. No periodic lubrication or servicing is required.
5. Allow adequate time for massager to cool down before storing.
6. When your massager is not in use for extended periods of time, store unit in area free of dust and moisture.

LIMITED ONE YEAR WARRANTY

Core Products International strives to provide the best quality products and the best possible service. Your purchase of the Jeanie Rub Massager comes with a Limited One Year Warranty. Defective products reported within one year of date of purchase will be replaced or repaired. This warranty covers mechanical or structural defects in workmanship within the coverage limitations listed below.

General Limitations:

- The warranty begins on the original purchase date or on the date of delivery.
- The warranty is valid only if the massager has not been misused or damaged in any way that is beyond the control of Core Products.
- Warranty does not apply to Jeanie Rub Massagers receiving unauthorized repairs.
- Warranty does not apply to Jeanie Rub Massagers purchased out of the box.
- Proof of purchase is required for all warranty repairs or returns.

Return Process:

Contact the Customer Service Department at Core Products at 1-800-365-3047 for all warranty repairs or returns. Core Products requires a Return Authorization (RA) number for any return to the factory. The consumer is responsible for packing the product in acceptable packaging. All returns require scheduling by the Customer Service Department at Core Products.

PLEASE SEND RETURNS TO:

Core Products International, 1505 Parker Drive, Chetek, WI 54728-7935

Out-of-Warranty Return Process:

If your Jeanie Rub Massager is ineligible for warranty service, but is repairable, contact the Customer Service Department at Core Products at 1-800-365-3047. Core Products requires a Return Authorization (RA) number for any return to the factory. The consumer is responsible for packing the product in acceptable packaging. Shipping and freight coverage is not covered when the Limited One Year Warranty has expired.

TROUBLESHOOTING

Special Motor Information:

The universal motor in the Jeanie Rub Massager incorporates carbon brushes. When used extensively at very low rpm levels, carbon can build up on the motor contacts, decreasing the massager's effectiveness. Core Products recommends that after each low rpm usage, the massager is run for a few minutes at a higher rpm level to ensure carbon buildup is cleaned from the motor contacts. This will prolong the life of the carbon brushes and optimize the massager's performance.

Core Products International, Inc

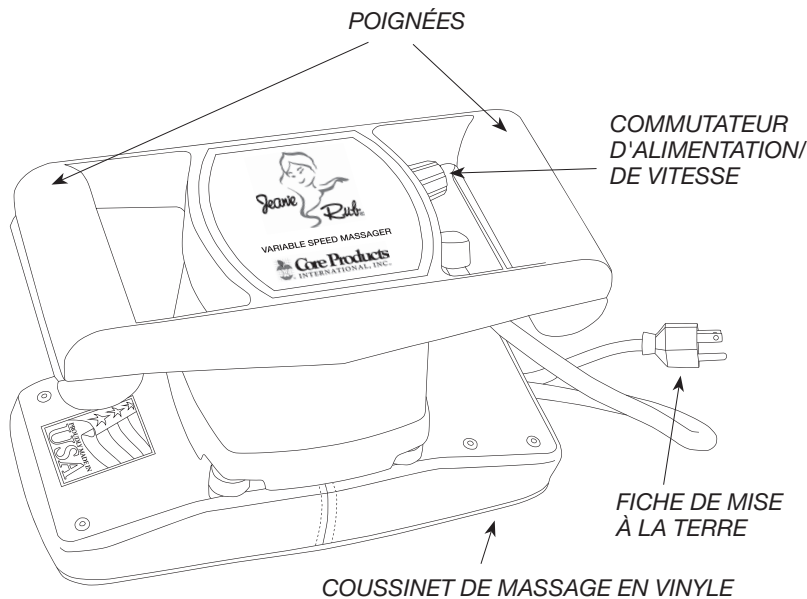
808 Propsect Ave

Osceola, WI 54020

715-294-2050 or 800-365-3047

customerservice@coreproducts.com

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