



Accutest® VALUPAK™ iFOBT Fecal Occult Blood Test PROCEDURE CARD

**READ PROCEDURE CARD BEFORE RUNNING TEST.
REFER TO PACKAGE INSERT FOR DETAILS.**

- 1) SHAKE THE SAMPLE COLLECTION TUBE VIGOROUSLY TO ENSURE A GOOD LIQUID SUSPENSION.

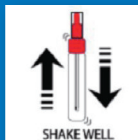


- 2) HOLDING THE TUBE UPRIGHT, UNSCREW THE CLEAR TIP COVER.

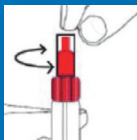

- 3) DISPENSE THREE (3) DROPS OF SUSPENSION INTO SAMPLE WELL.



- 4) READ RESULTS WITHIN 5-10 MINUTES AFTER ADDING THE EXTRACTION SOLUTION.

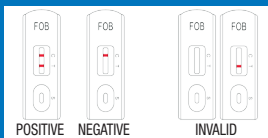

**Note: Do not interpret the results after more than 10 minutes.
Discard the test device after reading and recording the results.**



Jant Pharmacial Corporation
Encino, CA 91436

10210-JAN-QRG