

# ErgoPro II™

## Setup Guide

Congratulations on your purchase of the Stronglite Ergo Pro II™ portable massage chair. The following pages will guide you through the process of your massage chair setup and functionality.



**STRONGLITE™**  
SERVING MASSAGE PROFESSIONALS SINCE 1986

## SETUP PROCESS

The following steps will assist you with the setup of your Ergo Pro II™ portable massage chair.

### Unpacking the Chair

1



Remove all items from the box: Ergo Pro II, FacePillow and Deluxe Adjustable Platform. Lay the chair facing up.

### Releasing the Frame

2



Locate the elastic ball-lock under the Stronglite logo on the faceplate. Pull the ball upward.

3



Release the ball-lock away from the faceplate, allowing the chair to be opened.

### Standing the Chair Up

4



Pull the chair up, onto the back legs.

5



Anchor your foot on a back leg (A), near the wheel, & pull the front legs all the way forward till the chair stands upright (B).

### Locking the Frame

6



7



At the front of the chair, locate the frame lock lever underneath the arm rest. Carefully press the lever down completely till it locks in place.

**IMPORTANT:** This step prevents the metal frame from collapsing while in use.

### Arm Rest Adjustment

8



Locate arm rest lock lever (A). Lift armrest upward (B).

9



Fold lock lever over & down to secure arm rest.

### Chest Pad Adjustment

10



Locate chest pad lock lever (A). Make sure the lever is loose.

Chest Pad Adjustment (cont'd)



Adjust chest pad to desired location (B).



Fold lock lever over & down to secure chest pad.

FacePillow & Platform Adjustment



Locate platform lock lever in 'open' position (A).  
Fold down platform arms to desired location (B).  
Fold lock lever down to secure arms in place (C).

FacePillow & Platform Adjustment



Place FacePillow onto platform (A). Insert platform arms into inserts underneath the chest pad (B). Fold lock lever over & down to secure platform (C).



Use circular wheel tighteners for adjusting height level of the platform.

Setup Complete

Your Ergo Pro II is now ready for work!



ADDITIONAL FUNCTIONALITY

The following steps will help you customize the Ergo Pro II to fit all your clients.

Seat Angle Adjustment



The seat pivots up from the back legs. Locate and press the round metal pin inward to release the seat.



Lift the seat up (A). Pull the seat height lever down into the receiver slot (B).




The receiver location allows the seat to be locked in two angles: high or low.

## ADDITIONAL FUNCTIONALITY

The following steps will help you customize the Ergo Pro II to fit all your clients.

### Chair Height Adjustment - Back Legs

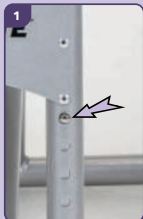

1  
Locate the round metal pins on each back leg under the faceplate.



2  
Press the round metal pins into the legs to allow for height adjustment.



3  
Adjust the legs to the desired height increment.

### Removable Pads




- FacePillow
- Arm Rest
- Chest Pad
- Seat Pad
- Knee Rests

All the pads and cushions for your Ergo Pro II can be removed and replaced!

### Removing the Knee Rests for Traditional Sitting Position



1  
Locate the round metal pins next to each support guide on the knee rests. Press each pin down to loosen the knee rest.

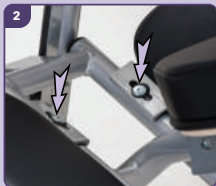

2  
Slide the knee rest forward until the support guide is centered within the circular opening.



Remove the knee rests. Your Ergo Pro II can now be used with a regular sitting position.

### Re-Packing Carry Case



Lay carry case down flat and open. Fold chair down into carry case. Wheels go through the holes in bottom of carry case.



Zip up the carry case and ensure the wheels are facing down toward the floor.

Your Ergo Pro II is now ready for easy transportation!



## Maintenance

We at Stronglite have designed and built the Ergo Pro II Chair to be long lasting and trouble free. All of the pivot joints are close tolerance to eliminate movement and reduce wear. Even so, it is possible for fasteners to become loose, parts to wear, and damage to occur.

It is up to you to maintain your chair in good condition. Tighten any loose fasteners and replace and/or repair any worn or damaged/missing parts. If you need replacement parts, contact Stronglite at 1-800-289-5487. See our warranty page, <http://www.stronglite.com/lifetime-warranty.html> for more information on repairs and replacement parts.

Do not allow clients to put all their weight on the knee rests, which are not designed to take the full brunt of the bodies weight.

**Warning: A chair with loose, damaged or missing parts can fail, possibly causing personal injury. Inspect your chair before each use and keep it in good condition.**

## Inspection

Before each use: Inspect your chair to see that all joints are as tight as possible without causing binding when the chair is folded up. Check to see that there are no damaged or missing parts. If you have any questions or need assistance please call 1-800-289-5487.

## Cleaning

The upholstery on your chair can be cleaned with mild soap and water and using a nonabrasive cloth. The powder coated aluminum and finished wood components can also be cleaned in this manner.

Warning: prolonged exposure to temperature extremes will cause damage to vinyl. Do not store your massage chair in your car or any other location exposed to hot temperatures.

## Warranty

Stronglite warrants the Ergo Pro II massage chair, excluding foam and vinyl, against defects in materials and craftsmanship for as long as you, the original purchaser, own the product, or for five years after model is discontinued. The foam and vinyl are warranted for 3 years from the original date of purchase. This warranty does not cover damage caused by accident, misuse or neglect, which includes tears in upholstery and damage caused by chemical based cleaners. This warranty is limited to factory-provided replacement parts, factory repair or replacement at the discretion of Stronglite.



# ADDITIONAL GREAT PRODUCTS BY STRONGLITE



- FacePillow™
- Strata™ FacePillow
- Carry Cases
- Stools
- Stool Covers
- Bolsters
- Bolster Covers
- Sheets & Linens
- Disposables
- Oils & Lotions
- Accessories

Strata™ FacePillow

FacePillow™



## Stronglite

990 Joshua Way  
Vista, CA 92081

(800) 289-5487  
(760) 597-3698  
(760) 599-7374 fax

Monday - Friday  
8:00 a.m. to 5 p.m. PST  
Saturday - Sunday  
Closed

Customer Service  
[info@stronglite.com](mailto:info@stronglite.com)

[Stronglite.com](http://Stronglite.com)  
Like us on Facebook