LUMISCOPE®

For The Quality Of Life™

Semi-Automatic Blood Pressure Monitor

Quick-Start Instructions



Please read before operating!!

**Refer to Instruction Manual for
Additional Details**

Model 1103

GF0800048RevB10


1. Open the battery cover and install four alkaline AA batteries in the battery compartment, ensuring that battery polarity is correct. Close the battery compartment cover.
2. Plug the end of the cuff tubing into the monitor connector as shown at right.

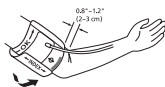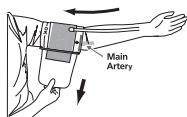3. See illustrations above. Apply the cuff around the upper arm as shown in the cuff label diagram. The cuff should not be too tight or too loose; two fingers should be easily inserted between cuff and upper arm. If the cuff is applied too tightly, or over thick clothing, high or inaccurate measurements may result.
4. Before taking a measurement, take 3 - 5 deep breaths and relax. In order to obtain accurate measurement results, remain seated, keep your arm and hands still, keep the cuff at heart level, and do not talk during measurement.
5. Press button "O/1" as shown at right.

O/1 (Off/On)

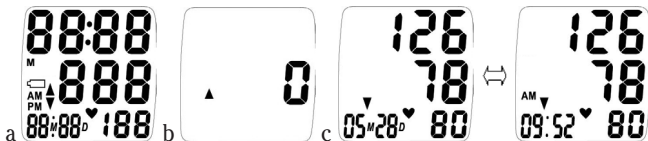6. All symbols will display for 2 seconds (a); you will hear two short beeps, then "0" will display with an upward arrow flashing (b). Pump the inflation valve until the reading reaches the recommended pumping value.
7. When measurement is complete, you will hear a long beep; the blood pressure and pulse measurements will then display. Press the rapid deflation valve to quickly deflate the cuff. The measurement date and time will display alternately (c).
8. Press button "O/1" to turn the device off.

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