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RapportTM Classic

Contents

- 2** Troubleshooting Guide
- 4** Warnings
- 6** How the Rapport™ Classic Vacuum Therapy Device (V.T.D.) Works
- 8** Choosing the Correct Constriction Ring
- 8** Choosing a Transfer Sleeve
- 9** Mounting the Constriction Ring
- 9** Fitting the Pump and the Penile Tube Together
- 10** Using the Pump to Create an Erection
- 12** Cleaning Instructions and Care of your Rapport™ Classic V.T.D.
- 14** Warranty

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Instructions for use

Troubleshooting Guide

Problem	Likely cause	Remedy	Comments
Handle sticking	The vacuum pump has been submerged in water or immersed in running water	Allow at least 24 hours to dry. Do not try to force the handle. Please do not immerse the vacuum pump in water. Clean as instructed	
No erection	No vacuum because of air leak caused by poor seal against the penis or seal between tube and pump	Use a smaller constriction ring, more lubricating jelly or trim pubic hair	The problem is often associated with inexperience, it can resolve itself as you become more adept at using the device
Partial erection	Not enough pumping strokes to create a full erection	Use a larger seal sleeve	With continued use the problem often resolves itself
Rapid loss of erection (in less than 5 minutes)	Vacuum effect lost because of air leak, poor seal against the penis, excessive pubic hair or constriction ring too large	Use a smaller constriction ring or more lubricating jelly	Experiment with varying sizes of constriction ring and / or seal sleeves
Delayed loss of erection (after 5 to 10 minutes)	Insufficient tension from constriction rings	Use smaller or a combination of constriction rings	The problem is often associated with inexperience, it can resolve itself as you become more adept at using the device
Pulling the skin of the scrotum	Lubricant on scrotum, improper pumping technique, sleeve too large	Remove lubricant from scrotum, use a smaller sleeve, use the modified pumping technique (ask your physician), or angle cylinder downward during pumping	
Discomfort or pain... during pumping	Pumping too rapidly	Pump slower and continue to use the device	With continued use the problem often resolves itself
from constriction ring	Constriction ring too small	Use a larger constriction ring and continue to use the device	With continued use the problem usually resolves itself
during intercourse	Inadequate lubrication or pressure on a sensitive area	Apply more lubrication or change your intercourse position	
during ejaculation	Long period of abstinence, infection or inflammation of the prostate gland	Continue to use the device	With continued use the problem often resolves itself – if pain or discomfort persists, consult your physician
Redness, irritation or bruising	Pumping too rapidly	Pump slower or use fewer strokes	Consult your physician if this problem persists
Penis feels cold (to you or your partner)	Constriction of blood flow to the penis or lubrication	Wash off lubricant or warm lubricant before use	
Penis pivots on its base	Constriction of blood flow to the penis	If manageable, apply the device after you have a partial erection	

Warnings

Do not submerge the vacuum pump in water or immerse in running water. Doing so may cause the handle to stick and prevent the vacuum pump from working correctly. If vacuum pump has been submerged refer to troubleshooting guide.

Cautionary points on the use of vacuum therapy devices.

- Use only the minimum amount of vacuum necessary to achieve an erection and increase the amount of vacuum pressure slowly. This will prevent bruising.
- If you experience any pain when you use the vacuum pump to create an erection, lower the vacuum pressure immediately by pressing the release valve.
- Never wear the constriction ring for more than 30 minutes and allow a minimum of 60 minutes between uses.
- Do not risk falling asleep while wearing a constriction ring.
- An instruction DVD is available, in addition to a video guide found at www.owenmumford.com, which shows a demonstration of the Rapport™ Classic V.T.D. being used to create an erection.
- Caution is advised when using a vacuum pump following removal of a penile implant because of possible surgical scarring and subsequent impairment.

- Misuse of a vacuum pump may aggravate already existing medical conditions such as Peyronie's disease, priapism and urethral stricture.
- Use of the vacuum pump may bruise or cause rupture of the blood vessels immediately below the surface of the skin, which may produce haemorrhage or haematoma.
- **If any of the above problems occur, discontinue the use of the vacuum pump and consult your doctor.**
- The vacuum pump does not guard against pregnancy. We advise you to continue to use your usual form of contraception.
- Be sure to start by using a larger size constriction ring to ensure that the constriction on the penis is no more than necessary.
- If your erection does not stay firm and hard the constriction ring may be too large and not enough blood is staying in the penis. Use the next size smaller.
- Always consult your healthcare professional before using the Rapport™ Classic V.T.D.

Owen Mumford will not be liable for any misuse of this product.

How the Rapport™ Classic Vacuum Therapy Device (V.T.D.) Works

An erection is created by means of a vacuum, which draws blood into the penis. The erection is maintained by placing a constriction ring at the base of the penis, which ensures that the blood remains in the penis until the ring is removed.





The constriction ring should be a snug, comfortable fit without being too tight.

The use of the Rapport™ Classic V.T.D. is not intended to replace your doctor's diagnosis of the underlying cause of your impotence, but to help achieve and maintain an erection. It is always best to discuss your individual problem with a doctor when using this treatment.

Initially it may take some practice to familiarise yourself with the Rapport™ Classic V.T.D., so please take the time to read this instruction book, and to practice the technique of creating and maintaining your erection, before you attempt sexual intercourse.

The instruction DVD and online instruction video guide shows how to put the Rapport™ Classic V.T.D. together and demonstrates how to create an erection.

The Rapport™ Classic V.T.D. is a complete system with everything you need for immediate use. Inside the discreet storage bag you will find: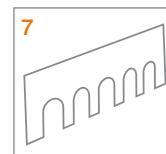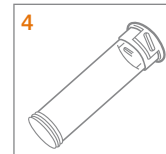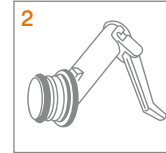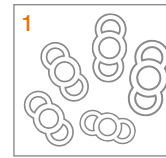

- 1** Five sizes of Rapport™ constriction rings. Five separately packaged and numbered rings, each a different size, from No. 4 the smallest up to No. 8 the largest. The centre ring is designed to encircle the base of the penis. The handles are for easy application and removal.
- 2** An easy to use Vacuum Pump which has a specially angled handle to make creating the erection simple. As a safety measure the vacuum pressure is controlled by a valve that will not allow the pressure to rise above 360mm of Hg. The vacuum pressure can be released at any time by pressing the pressure release valve on the top of the pump.
- 3** A loading cone to assist the mounting of the constriction rings onto the transfer sleeve.
- 4** A penile tube in durable clear material, which attaches to the pump unit at one end and the transfer sleeve with the constriction ring at the other.
- 5** Two transfer sleeves which give a choice of internal diameter. Both are specially shaped to hold the constriction ring and make it easy to transfer it onto the base of the penis.
- 6** A tube of lubricating jelly. This helps to create the vacuum seal and prevents the penis from adhering to the penile tube.
- 7** A card ring sizing guide for easy selection of the correct size constriction ring can be found at the back of this book.
- 8** Depending on which pack you have received, a How to Use DVD may be included. An online instruction video guide is available at www.owenmumford.com.

Choosing the Correct Constriction Ring

One of the most important parts of the Rapport™ Classic V.T.D. is the constriction ring. The ring must encircle the penis and fit comfortably tight.

The constriction ring works by retaining the blood in the penis to maintain a firm erection.

Using the card ring sizing guide provided, place your flaccid (non erect) penis into the slots until you find one which fits the circumference of your penis. Each slot has a ring size marked above it which corresponds to the ring you require.

If the Rapport™ Classic V.T.D. gives you a good erection, and with the constriction ring on your penis, your erection decreases prematurely, you will need to try a ring one size smaller.

If in use the constriction ring seems too tight, next time use one size larger to see if it will maintain your erection.

Try different sizes until you are satisfied with their fit. There are five sizes to choose from.

Choosing a Transfer Sleeve

There are two transfer sleeves provided with two internal diameters. Depending on the size of your penis you should choose which of the two is likely to be the most suitable. Remember that your erect penis will need to be withdrawn through the transfer sleeve.

The transfer sleeves are specially shaped so that the constriction ring can be easily loaded and held in the flat groove.

The unique shape also allows easy passage of the constriction ring off the transfer sleeve and onto the base of your penis.

Because the transfer sleeves are made of a soft material they help to make a good seal against your body to achieve the vacuum inside the penile tube which creates the erection.

Mounting the Constriction Ring


When you have selected the most suitable transfer sleeve place it firmly onto the clear end of the penile tube. (Fig. 1)

Fit the loading cone onto the transfer sleeve and apply a little water soluble lubricating jelly to the top of the cone.

Holding the constriction ring by the outer handles, simply stretch and pull the ring down the loading cone until it slips onto the transfer sleeve. Make sure that the constriction ring is located into the flat groove at the centre of the transfer sleeve. (Fig. 2)

The constriction ring is now mounted and ready for use.

Before using the Rapport™ Classic V.T.D. make sure you use the water soluble lubricating jelly to lubricate the end of the transfer sleeve. Also lightly lubricate the penis to prevent it from sticking to the inside of the penile tube during insertion or while using the pump. (Fig. 3)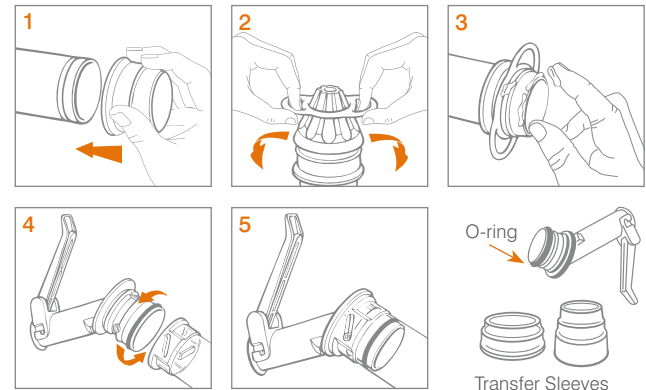

Fitting the Pump and the Penile Tube Together

The pump head has two small pegs each side above the O-ring; these should be positioned into the two grooves on the top of the penile tube.

The two parts are then locked together with a twist so that the two pegs slot down and locate at the lower end of the groove. (Fig. 4)

If these two parts are then locked together correctly it will ensure the unit is sealed tight enough to create a vacuum inside the penile tube. (Fig. 5)

Using the Pump to Create a Vacuum

- 1 Insert the penis into the penile tube and apply slight pressure to seal the transfer sleeve against your body at the base of the penis, to ensure a vacuum will be created. (Fig. 6)

It may be necessary to groom the hair at the base of the penis to ensure a vacuum is created.

Make sure the constriction ring on the transfer sleeve is in the correct position with the handles to the top and bottom.

Steady the tube with one hand to prevent any jerky movements and squeeze and release the pump handle with the other hand. This works the pump to create the vacuum necessary. (Fig. 7)

You may find that creating the erection will be easier and more comfortable if you stand up to use the pump. You can always steady the tube by placing it on a flat surface such as a table.

If you are unsteady when standing up, create the erection sitting down on the edge of a chair.

- 2 Continue pumping slowly to create the erection. If you feel any pain stop pumping and press the pressure release button on the pump to eliminate the vacuum. Contact your doctor.

Please take care not to pump too fast as this may bruise or cause a rupture of the blood vessels immediately below the surface of the skin.

- 3 **NOTE:** For your safety the vacuum pump is automatically released through the release valve at a pressure higher than 360mm of Hg.



- 4 When the desired erection has been achieved, continue to hold the tube tightly against your body. Hold the tube with one hand, still pushing it against your body and with the fingers of the other hand, ease the constriction ring off the transfer sleeve and onto the base of your penis. Always position the constriction ring with the handles to the top and bottom while on the base of your penis. (Fig. 8)

Because of the special shape of the transfer sleeve, when the constriction ring is eased out of the flat groove it will then slip easily off the transfer sleeve and onto your penis.

Some vacuum pressure may be lost during this transfer, but you will still need to press the release valve to lower the vacuum pressure to enable you to remove the penile tube. (Fig. 9)

If you find the placing of the constriction ring onto your penis difficult try releasing a little of the vacuum by pushing the penile tube away from your body slightly then rolling the constriction ring off the transfer sleeve.

- 5 With a constriction ring in place, a lasting erection should now be maintained and sexual intercourse can be accomplished.
- 6 **NOTE:** If a condom is to be worn, it should be put on as a last step. Do not wear a condom during the vacuum process. (Fig. 10)
- 7 To remove the constriction ring after intercourse, simply grasp the outer handles to stretch the central ring. As the blood is released from the penis it will decrease in size and the ring may be easily removed.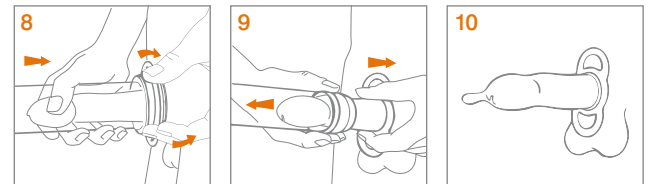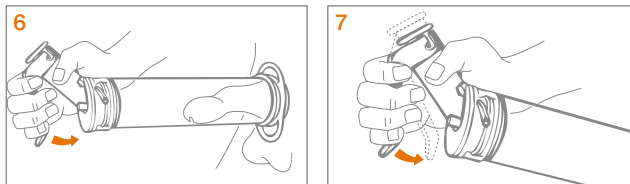

Cleaning Instructions and Care of your Rapport™ Classic V.T.D.

1 The Constriction Rings

After each use, the constriction ring should be washed with mild soap in warm water, rinsed well and dried thoroughly. Return the ring to its bag.


Make sure you only use water soluble lubricating jelly with the constriction rings. Your rings should last for longer if cared for properly.

2 The Penile Tube and Transfer Sleeve

After each use, the penile tube and transfer sleeve should be washed thoroughly with a mild soap in warm water. Rinse and dry well before replacing them into the carrying bag.

3 The Vacuum Pump


Very little maintenance is needed for the vacuum pump, but we recommend that you occasionally wipe the vacuum pump with a damp cloth to remove any dirt or lubrication. **Do not submerge the vacuum pump in water or immerse the vacuum pump under running water, as this may cause handle to stick and it will not be possible to create a vacuum.**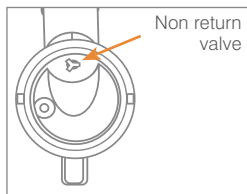

Please take care when wiping the underside of your vacuum pump as the non return valve (three-pronged rubber valve) can be pushed into the piston chamber with only a slight force. **If this valve is not in position, no vacuum will be achieved.**

It will be necessary from time to time to lubricate the O-ring on the pump head, particularly if you are finding it difficult to attach or remove the pump head to and from the penile tube.

After cleaning the pump head apply a little petroleum jelly to the O-ring (the coloured band) making sure you smear the lubricant all the way around the O-ring. This will ensure a total seal between the pump head and the penile tube and maintain the vacuum inside the penile tube.

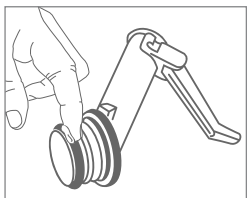NOTE: This lubrication of the O-ring with petroleum jelly is the **ONLY** use of petroleum jelly.

Do not allow petroleum jelly to come in contact with the constriction rings as it could damage them.

Aquagel® Lubricating Jelly

Ingredients – Sodium polyacrylate (carbomer), Glycerol, Hiquest (di-sodium salt of 1-hydroxy ethane-1, 1 diphosphonic acid; sequestrant), Sodium Nipasept (parabens mix – preservative), Purified water.

Made by ECOLAB.

ECOLAB, Lotherton Way, Garforth, Leeds LS25 2JY, England www.ecolab.co.uk

0086

STERILE R

Aquagel® is not a product or trademark owned by Owen Mumford. For more information please contact the relevant manufacturer.

Warranty

This warranty is offered to you as a purchaser ("**you**") by the Owen Mumford entity whose details are set out on the relevant device packaging ("**we**" or "**us**"). You cannot transfer this warranty to anyone else. This warranty is in addition to, and does not affect, your rights as a consumer or purchaser under the laws of your country.

This warranty applies for twelve months from the date of purchase (the "**warranty period**").

If the device is or becomes materially defective during the warranty period, please notify us providing details of the defect and we will provide you with a replacement device. Subject to the laws of your country, we shall have no further liability to you after we have provided you with a replacement device.

This warranty will not apply to defects which are attributable to:

- fair wear and tear;
- handling, storage or use of the device other than in accordance with our instructions, or improper handling, storage or use of the device; or
- damage or breakage caused by accident or by external causes beyond our control.

Owen Mumford will not be liable for any misuse of this product.

When you contact us, we may ask you for details of the batch number or other form of identification of the device in order for us to further investigate the defect. We may also ask you to return the device and any provision of a replacement device is subject to us receiving the defective device from you.

Owen Mumford Ltd, Primsdown Industrial Estate,
Worcester Road, Chipping Norton, Oxfordshire
OX7 5XP, United Kingdom
T : +44 (0)1608 645555 **F** : +44 (0)1608 646546
E : info@owenmumford.com

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