

Symbol Descriptions: English



In this document, indicates conditions or practices that could lead to illness, injury, or death ("WARNING").



In this document, indicates conditions or practices that could damage the equipment or other property ("CAUTION").



Consult Instructions for Use



Do not re-use, Single use device



Do not sterilize



Catalogue Number



Lot Number



Use By Date



Manufacture Date



Quantity



Manufacturer's contact information



Not made with natural rubber latex



Fragile



Temperature limit



Humidity limitation



For Professional Use



Do not use if package is damaged

STERILE Sterilized using ethylene oxide

Indications for Use

The single-use lighted laryngoscope is a device to aid the intubation of the trachea. A rigid laryngoscope blade restrains the tongue, allowing a clear view of the larynx and entrance to the trachea. The need to provide a guaranteed airway is fundamental to an anesthesiologist's task prior to an operation. Paramedics, A&E Consultants and General Practitioners may also be required to insert a tracheal tube in an emergency to keep the airway open where an unconscious patient is undergoing cardio-pulmonary resuscitation.

Warnings and Cautions

WARNING: Inspect each device for damage prior to use
WARNING: If a device is damaged, do not use any device from the same box.

WARNING: When in transit or storage, device may be subject to damage beyond the control of the manufacturer or supplier.

WARNING: Never use device with laser equipment.

WARNING: Treat used devices as bio hazardous infectious material. Dispose of used device in suitable disposal unit or in accordance with local regulations.

WARNING: Devices are not compatible with gamma radiation or autoclave sterilization and should never be re-sterilized by any method.

WARNING: No modification of this equipment is allowed

WARNING - Risk of reuse:

The strength of the device may be significantly compromised and the resulting risk to the patient and/or user is unknown.

Caution

The light may not function properly.

CAUTION: Use by trained personnel only.

CAUTION: U.S. federal law restricts this device to sale by or on the order of a physician.

CAUTION: Light source is single-use and of limited duration.

Do not activate light source until you are ready to use the device.

Warranty

Please visit www.obpmedical.com "Terms and Conditions" page for warranty information. OBP Medical should be notified immediately via phone or email of any defective


obpTM
simple, bright ideas.

SURE-SCOPE® Single-Use One Piece Laryngoscope with Integrated LED Light Source

REF C090001 thru C099999



OBP Medical
360 Merrimack Street
Building 9
Lawrence, MA 01843 USA
+ 1-978-291-6853
www.obpmedical.com

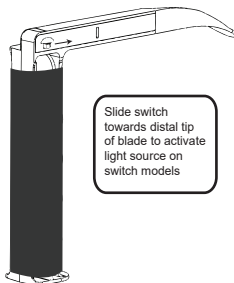

EC REF

Approved under
CE Marking
Directive 93/42/EEC
Medical Devices
The Netherlands

CE Marking
Directive 93/42/EEC
Medical Devices
The Netherlands



www.obpmedical.com

CE 2797

Patent information available at:
www.obpmedical.com/patents

PL-09-0001 REV G