



WHEELCHAIR SIDE WINGS

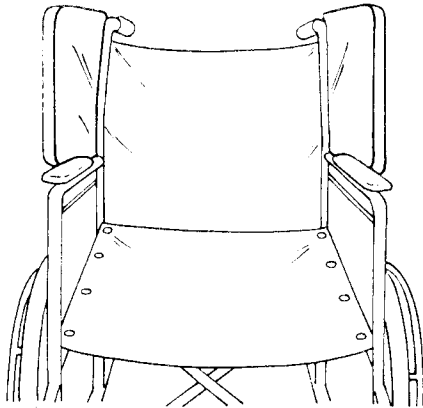

SIDE WINGS – STANDARD SUPPORT

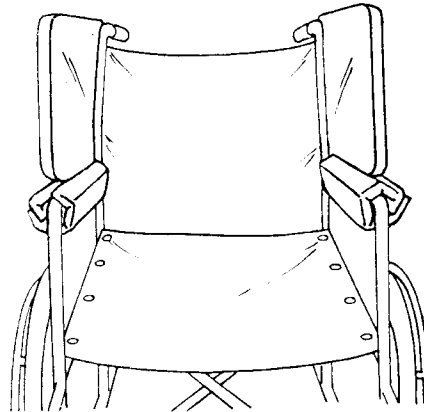

SIDE WINGS – XTRA SUPPORT

Reorder #	Description	Unit
□706320	Side Wings/Standard Support	Pair
□706322	Side Wings/X-tra Support	Pair



Purpose

Designed to provide support for residents who lean laterally.

Easy to Use

Installation instructions for Standard Support Side Wings

1. Remove the top and second-from-top screws from both sides of the wheelchair backrest. See illustration A.
2. Note that the Side Wings are marked LEFT and Right. Four sets of screws and washers are provided with the Side Wings. Place a washer on each screw. Insert the screws through the slots in the brackets, through the holes in the wheelchair upholstery, and into the screw holes in the backrest frame. Use two screws for each bracket. Tighten screws. The brackets have two parallel sets of slots that allow you to adjust the Side Wings for narrow-shouldered or wide-shouldered residents. See illustration B.

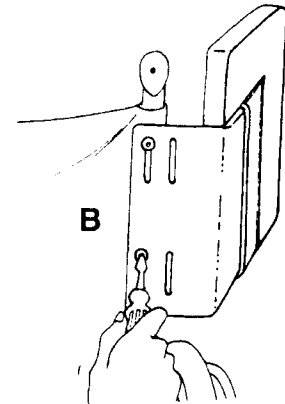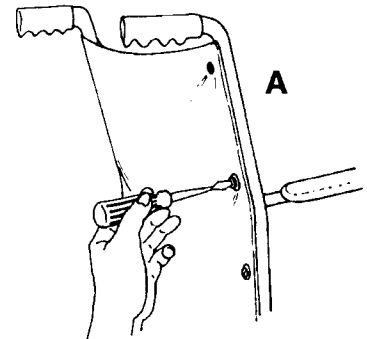

Installation instructions for X-tra Support Side Wings

1. Follow steps one and two above.
2. Wrap the armrest pads around the wheelchair armrests and secure the Velcro® closure. See illustrations C and D.

Easy to Care For

Skil-Care Wheelchair Side Wings wipe clean with a damp cloth. A spray-type cleaner may be used. If Velcro® closures on the X-tra Support model accumulate lint or particles, these may be removed using a stiff brush.

